Rumble Tuff®

WHISPER

**USB Electric Breast Pump
Instruction for use**

EN 01 - 38

ES 39 - 76

Model: 208DU-3

Congratulations on your new Rumble Tuff® Whisper Electric Breast Pump!
The Rumble Tuff® Electric Breast Pump is perfect for many occasions.

- Collecting breast milk while you're away from your baby
for example: when you're at work
- Relieving engorgement
for example: when your baby's feeding habits change because they're eating more solids
- Maintaining your milk supply if you have to temporarily stop breastfeeding
for example: to express unsuitable milk while you are undergoing medical treatment

The Rumble Tuff® Electric Breast Pump provides many of the features busy moms love.

- 2 cycles of expressions to mimic baby's nursing patterns.
 - Stimulation - Fast but shallow swallowing habit
 - Expression - Steady and strong swallowing pattern
- Milk back-flow protection
- Quiet motor for discretion
- Small and lightweight

Indication for Use

The Rumble Tuff® Electric Breast Pump is a personal use, electrically powered device used to express milk from the breasts of lactating women.
This device is not intended for hospital use.

Contraindications for use

There are no known contraindications for use with this product.

Table of Contents

Congratulations	1
Important Safeguards	4
Danger	4
Warning	4
Caution	5
Accessories and Parts	6
Included Parts and Accessories	6
Expression Collection Combo Kit	7
Bottle Assembly	8
How to Assemble the Expression Collection Combo Kit	8
Correct Breast Shield Fit	12
Cleaning Instructions	13
Sanitize Prior to First Use	13
Before Each Use	13
After Every Use	14
For Extra Protection, Sanitize	16
Store Safely Until Needed	17
Tubing Cleaning Instructions	17
About Your Rumble Tuff Whisper	18
Electric Breast Pump Main Unit	18
Expression Indicator Lights	18
How to Use Your Rumble Tuff Whisper	19
LED Light Sign Guide	21

Power Options	22
Using Your AC/DC Adapter	22
Using your Power Bank	23
Using your Rumble Tuff Sweet Essence	26
Storage and Preparation of Breast Milk	24
Troubleshooting.....	26
Interruption of Electrical Service.....	26
Radio Frequency Energy.....	26
Decreased Suction	26
Pump Doesn't Start	27
Breast Milk Buildup in the Breastshield Body.....	27
Where to Get Replacement Parts	27
Where to Purchase Additional Accessories	27
Condensation or Milk in Tubing.....	28
EMC/Technical Description	29
Warranty	35
Technical Data	36
Power Requirements.....	36
Technical Specifications	36
Symbols Legend	37

Important Safeguards

The Rumble Tuff® Electric Breast Pump is designed for lactating women to express and collect breast milk. It is intended for a single user and should never be shared or resold; doing so will pose health risks and void the warranty.

READ ALL INSTRUCTIONS BEFORE USING

DANGER:

To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING:

- Anti-Backflow is the essential performance of Rumble Tuff Breast Pumps. If any moisture/milk is found in the tubing while using or after using the pump, stop using the pump and contact Rumble Tuff Customer Service immediately.
- Use only the AC/DC Adapter that comes with the Whisper Breast Pump
- Never leave plugged-in product unattended.
- Plug the AC/DC Adapter into the electrical outlet, before connecting the Power Cord into your pump.
- Make sure the voltage of the AC/DC Adapter is compatible with the power source.
- Do not operate this product if the cord or plug is damaged, if it is not working properly, if it has been dropped or damaged, or if it has been dropped in water.
- Keep the cord away from heated surfaces.
- Keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- Do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or breast pump parts.
- Never put the pump unit into water or sterilizer as it can cause permanent damage to the pump.

- If you are infected with Hepatitis A, Hepatitis B, or have HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- Do not attempt to modify the product.
- Do not pump while driving.
- If there is any potential allergic reaction, stop using our products and find a medical advice immediately.

CAUTION:

- Do not use this product for any other purpose than as indicated in this manual.
- Make sure all of the parts are assembled correctly before operating.
- Stop using the breast pump immediately if your breasts or nipples experience pain or discomfort.
- Please wash hands before using this product.
- Do not drop or hit the product. Doing so may cause damage not covered by the warranty.
- Do not touch the Expression Collection Kits with bare hands immediately after sterilization by boiling water to avoid scalds and burns.
- Do not overboil product materials; to prevent premature aging.
- Do not use any accessories, parts, or attachments that are not recommended by Rumble Tuff. Doing so may affect the performance of your breast pump and void the warranty.
- Do not connect the pump or any of its accessories to any equipment not recommended by Rumble Tuff.
- Do not store the product in a dusty, or damp environment, or in direct sunlight.
- If irritation or discomfort occurs, discontinue use and see a doctor.

Note:


When disposing of the pump or its accessories, please consult with your local authorities for the proper method of disposal of potentially biohazardous parts and accessories, or to learn if there are any other regulations.

SAVE THESE INSTRUCTIONS


along with your receipt for future reference.

Accessories and Parts

Included Parts and Accessories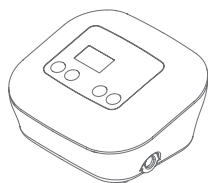


Breast Pump Main Unit

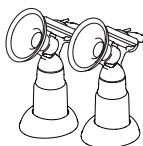Expression Collection
Combo Kits X2


Bottle Adapter
Kits X2

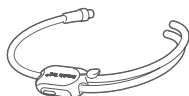Vacuum Tubing (Double)
with Splitter

AC/DC Adapter

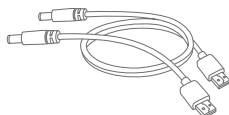
2 USB Cables
(45 cm & 190 cm)

Membrane Valve X4

Note: For ordering information about accessories, please visit www.rumbletuff.com/accessories; or contact our customer service. Toll-Free: (855) 228-8388

Expression Collection Combo Kit

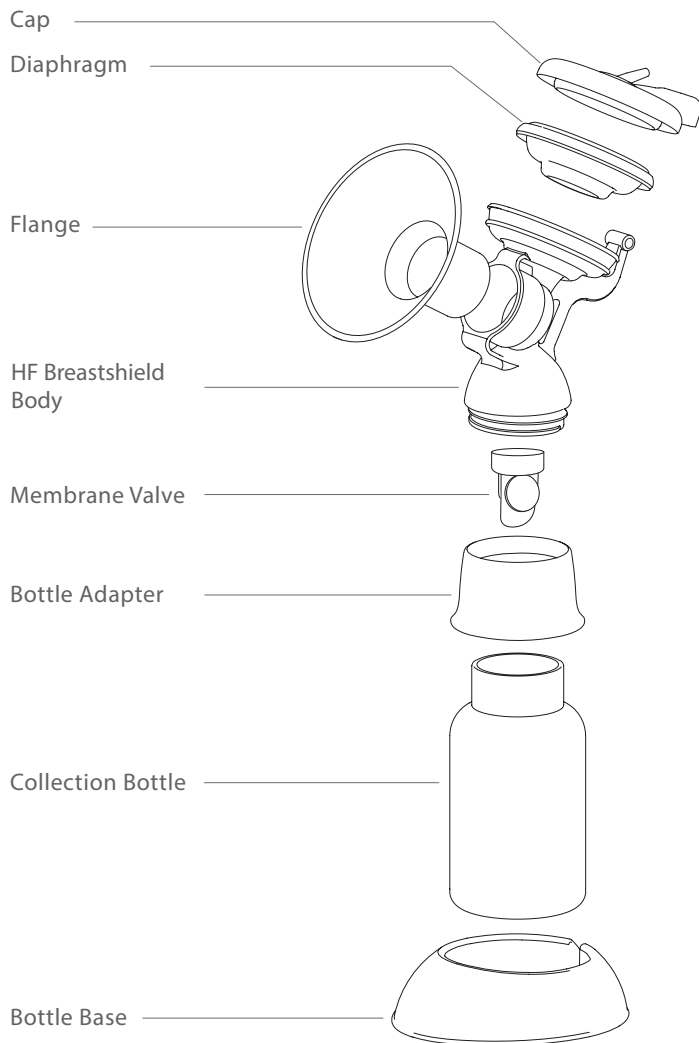Bottle Assembly

How to Assemble the Expression Collection Combo Kit


Hold the HF Breastshield Body so you can look into the bottle thread opening.


With the membrane facing towards you, firmly insert the Membrane Valve into the HF Breastshield Body and position valve as shown in the black checked illustration seen below.


Confirm that the Membrane Valve is securely installed.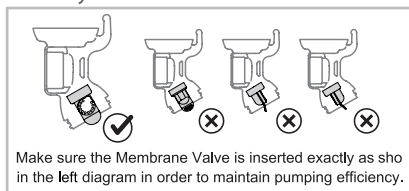

How to Assemble the Membrane Valve:

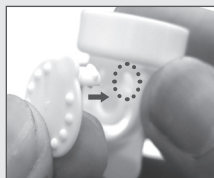Hold the Silicone Membrane and insert the small stem into the top-right opening of the Valve.

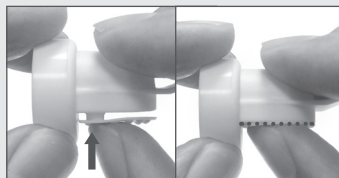
Press hard on the Membrane to ensure that the stem is completely inserted and is snug against the Valve opening.

Membrane Valve Assembly completed.

How to Assemble the Expression Collection Combo Kit (Continued)

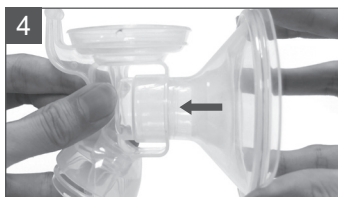
Insert the Flange into the HF Breastshield Body .

Place the Diaphragm in the Cup of the HF Breastshield Body .


Note: Do not reverse the order of steps 6 through 9. Doing so will result in suction loss when pumping.

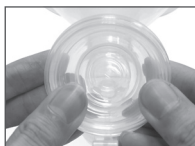Press around the edges of the Diaphragm for a secure fit.

Ensure that the Diaphragm is securely mounted in place. This is crucial in order to achieve proper suction and prevent milk back flow.

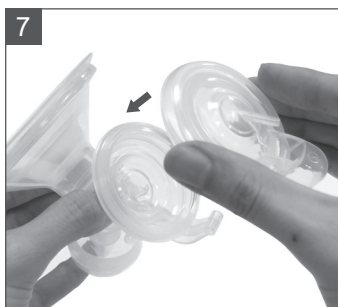Place the Cap on the Cup and Diaphragm.

Align the hinge cover over the hinge. Press tightly on the cap to seal.

Bottle Assembly

How to Assemble the Expression Collection Combo Kit (Continued)


The Electric Diaphragm Kit is assembled.

Twist the Bottle Adapter onto the HF Breastshield Body .

Twist on the Collection Bottle for a secure fit.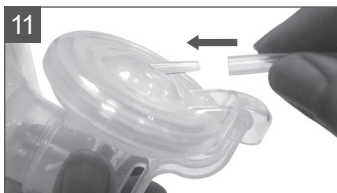

Connect the Tubing to the Tubing Connector located on the Cap.

Note: Make sure there is no moisture in the Tubing.

How to Assemble the Expression Collection Combo Kit (Continued)

12

The Expression Collection Combo Kit is assembled.

13

Make sure the Tubing Adapter is properly inserted into the Vacuum Inlet on the Electric Pump Main Unit.

To Convert Your Double Pump Into a Single Pump:

Remove one tube and close the cap.

1

Disconnect the tube from the Tubing Splitter on the side with the little Cap

2

Cap the opening of the Tubing Splitter

3


Ready to use as a single pump.

Note: Using the pump after converting from Single to Double or Double to Single may result in a temporary lack of suction or over-suction respectively. Please use caution and adjust the pump to your level of comfort for effective pumping.

Bottle Assembly

Correct Breast Shield Fit

- Measure nipples before pumping session.
- Please refer to rumbletuff.com/size-matters-quick-flange-breast-shield-guide/ for breast shield/flange size guidance.

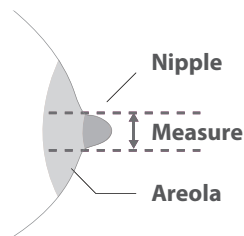The Collection Combo Kit come with 25mm breast shield/ flange size. Optional 13mm~ 32mm breast shield/ flange are available for purchase at rumbletuff.com.

too small

too large

correct fit

To allow for proper fit, breast shield / flange should be about 1 - 3mm larger than nipple diameter.

- Please refer rumbletuff.com/how-to-measure-your-nipples-for-the-proper-flange-fit-size/ options.

Sanitize Prior to First Use:

Before you use your Rumble Tuff® Whisper Electric Breast Pump for the first time, disassemble and boil all parts of the Expression Collection Combo Kit in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Remove all parts from the water immediately after sterilizing. To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- Completely air-dry or towel-dry the parts before reassembling.

Wet components may affect the performance of your Electric Breast Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water for at least 20 seconds.


2. **Assemble.** Assemble clean Express Collection Combo kits. Inspect each component for any cracks, tears, discoloration, or deterioration. If damage is found, please discontinue use and replace the part immediately. Inspect whether the kits or tubing has become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.

Special Cleaning Instruction for the Membrane Valve:

The Membrane is specially designed with a node on the stem for your convenience to clean. To clean the Membrane Valve, simply pull the Membrane from the Valve until it snaps slightly on the stem but does not completely detach from it.

Clean the Membrane Valve in warm, soapy water. Rinse thoroughly in clean, warm running water.

After cleaning, push the stem back into place and make sure the Membrane lays flat against the Valve opening.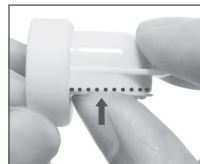

Cleaning Instructions

After every use:

1. **Store milk safely.** Cap milk collection bottle or seal milk collection bag, label with date and time, and immediately place in a refrigerator, freezer, or cooler bag with ice packs.
 - If milk collection container will be stored at a hospital or childcare facility, add name to the label.
2. **Clean pumping area.** A low-level disinfectant, such as Ethyl or Isopropyl Alcohol (70-90%) is recommended to wipe pump dials, power switch, and countertop.
3. **Take apart and inspect the Collection Kits.** Take apart breast pump tubing and set it aside. Separate all parts that come in contact with breast/breast milk (for example, Silicone Cushions, Flanges, Membrane Valves, Diaphragm and Milk Collection Bottles).
4. **Rinse Collection Kits.** To remove remaining milk, rinse Collection Kit parts, that come into contact with breast/breast milk, under warm, running water.
5. **Clean Collection Kits.** As soon as possible after pumping, clean Collection Kits that come into contact with breast/breast milk in one of the following ways.

Clean by hand

- (1). **Use a wash basin** Place Collection Kit parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the pump parts.
- (2). **Add soap and water** Fill wash basin with hot water and add soap.
- (3). **Scrub** Scrub items using a clean brush that is used only to clean infant feeding items.
- (4). **Rinse** Rinse by holding items under running water, or by submerging in fresh water in a separate basin which is used only for cleaning infant feeding items.
- (5). **Dry** Allow to air-dry thoroughly. Place Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Due to the potential of transferring germs, avoid using dish towels to rub or pat items dry.

Clean in a dishwasher

(1). **Wash** Place disassembled Collection Kits parts in dishwasher. To avoid small items ending up in the dishwasher filter, place them in a closed top basket. If possible, run the dishwasher using hot water and a heated or sanitizing drying cycle, which can help kill more germs.

(2). **Remove from dishwasher** Before removing and storing clean items, wash your hands with soap and water. If items are not completely dry, place them on a clean, unused dish towel or paper towel to air-dry thoroughly before storing or use the provided Drying Mess Bag to air dry all parts. Due to the potential of transferring germs, avoid using dish towels to rub or pat items dry.

6. **Clean wash basin and bottle brush.** If you use a wash basin or bottle brush when cleaning your Collection Kits parts, rinse them well and allow them to air-dry after each use. If they are dishwasher-safe, consider washing basin or bottle brush in a dishwasher with hot water and a heated or sanitizing drying setting; if not, wash by hand with soap and warm water.

7. **Tubing.** Inspect the tubing. If you find condensation in the air tube, wash and dry it immediately or replace the tubing. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connect. Instead contact Rumble Tuff Customer Service.

- E-mail: customerservice@rumbletuff.com
- Toll-Free: (855) 228-8388
- Website: www.rumbletuff.com

Cleaning Instructions

For Extra Protection, Sanitize

For extra germ removal, sanitize Collection Kit parts at least once daily. Sanitizing is especially important if your baby is less than 3 months old, born prematurely, or has a weakened immune system due to illness or medical treatment (such as chemotherapy for cancer).

If the parts are cleaned carefully after each use, daily sanitizing of Collection Kit parts may not be necessary for older, healthy babies.

Sanitize all items (even the bottle brush and wash basin!) by using one of the following options.

Note: If you use a dishwasher with hot water and a heating drying cycle (or sanitizing setting) to clean infant feeding items, a separate sanitizing step is not necessary.

1. **Clean first.** Collection Kit parts, bottle brushes, and wash basins should be sanitized only after they have been cleaned.
2. **Sanitize.** Sanitize the Collection Kits, bottle brushes, and wash basins using one of the following options.

Steam:

Use a microwave or plug-in steam system according to the manufacturer's directions.

Boil:

Place disassembled items into a pot and cover with water.

Put the pot over heat and bring to a boil for 10 minutes.

Remove items with clean tongs.

3. **Allow to air-dry thoroughly.** Place sanitized Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Due to the potential of transferring germs, avoid using dish towels to rub or pat items dry. You can also use the provided Drying Mesh Bag to air dry the Collection Kit parts.

Store Safely Until Needed

To help prevent germs and mold from growing, allow the clean Collection Kit parts, bottle brushes, and wash basins to air-dry thoroughly before storing. Once completely dry, the items should be stored in a clean, protected area to prevent contamination during storage.

- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Collection Kits.
- **Store safely.** Place reassembled Collection Kit in a clean, protected area such as inside an unused, sealable food storage bag. Store wash basins and bottle brushes in a clean area.



Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing is designed to not touch the pumped milk and does not need to be cleaned routinely.
- Keep a spare set of tubing on hand in case the set you are using gets soiled or damaged.
- If your tubing has water droplets in it at the end of a pumping session, disconnect the tubing from the Collection Kit, check to see if the Diaphragm needs to be replaced. Rinse the Tubing with clean, hot water. Hang to COMPLETELY AIR DRY BEFORE USING.
- If your tubing has mold in it, throw it away immediately because it is difficult to clean properly, and replace with a new set of tubing. Check to see if the Diaphragm needs to be replaced.
- If the outside of your tubing is soiled, it is recommended wiping it with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %).
- Do not insert any object into the Tubing.

About Your Rumble Tuff Whisper

Electric Breast Pump Main Unit

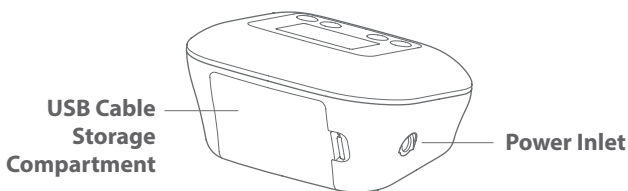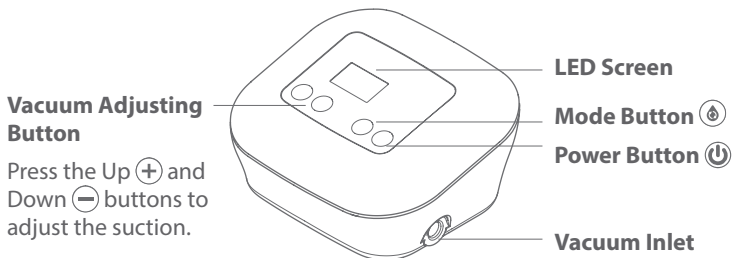Expression Indicator Lights

Stimulation (Single Digit Display)

- Helps stimulate milk flow and letdown.
- This is a fast, shallow suction pattern. When a baby has just latched on, the nursing pattern is fast and shallow to start the mother's milk flow.

Expression (Letter "F" before the single digit)

- Helps maximize milk expression.
- Longer and deeper suction pattern. When letdown begins (milk begins to flow), the baby often changes to a longer, deeper suction pattern.

About Your Rumble Tuff Whisper

How to Use Your Rumble Tuff Whisper

Hold the Expression Collection Combo Kit to your breast so that your nipple is centered in the Flange.

Press the Power Button  to begin pumping.

The last operating time will be displayed as soon as the pump is turned on. The pump automatically remembers the last suction level used in Stimulation mode and will start at that level.

While pumping, use the Vacuum Adjusting Buttons  and  to adjust the suction level.

When you feel your milk start to letdown, press the Mode Button  once to switch to the Expression mode. The pump automatically remembers the last suction level used in Expression mode and will start at that level.

About Your Rumble Tuff Whisper

How to Use Your Rumble Tuff Whisper (Continued)

While pumping, use the Vacuum Adjusting Buttons (+) and (-) to adjust the suction level.

When you finish your pumping session, press the Power Button to turn the pump off. (⏻)

After the pump has been turned off, the LED Display will remain lit to show the total minutes used in the last pumping session.

Note:

- Make sure there is no suction on the nipples before removing the breastshield. If necessary, break the suction by inserting a finger between the breastshield and the breast, similar to the way you break away from your baby's suction.
- If the Rumble Tuff® Electric Breast Pump is left on, but unattended for more than 30 minutes, the Pump will automatically shut-off.

About Your Rumble Tuff Whisper

LED Light Sign Guide

While Pump is On

LED Display	State
88 82 83 84 85 86 87 88 Stimulation LED (1 - 8)	Pumping in Stimulation
F0 F2 F3 F4 F5 F6 F7 F8 F9 Expression LED (F1 - F9)	Pumping in Expression

Power Options

Your Electric Breast Pump may be operated under any of the following power supply modes:

Using your AC/DC Adapter (included)


Danger:

*If the power adapter outer casings become loose or are separating, contact the device manufacturer immediately. **Separation of the casing can result in electric shock.***


Caution:

Electrical safety can only be achieved when using the AC/DC Adapter provided with the Pump. Use of any another Adapter poses a safety risk and/or the warranty may be voided.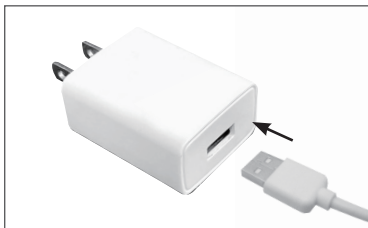


Connect the USB cable with the Adapter.

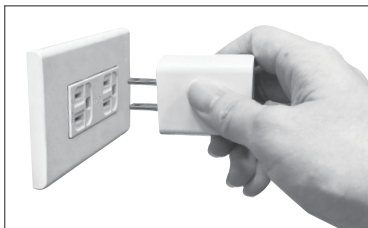
Plug the AC/DC Adapter into an electrical outlet.

Note: Do not leave the AC/DC Adapter (with or without the USB cable) in the electrical outlet after use.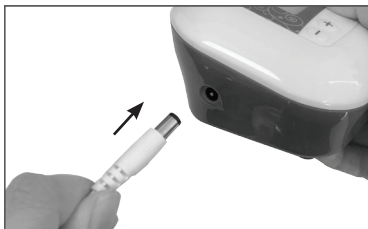


Plug the Power Cord into the Power Inlet on the Electric Pump Main Unit.

Using your Power Bank

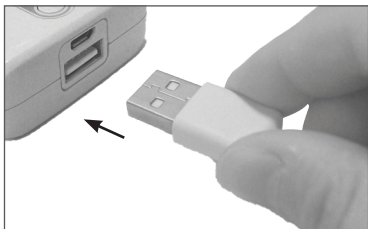
Connect the USB cable to the Power Bank.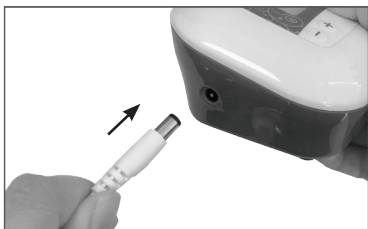

Plug the Power Cord into the Power Inlet on the Electric Pump Main Unit.

Turn on your Power Bank for power supply.

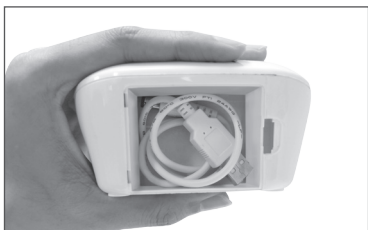Store the USB Cable in the storage compartment after use.

Note: To reduce noise, keep the storage compartment lid closed while pumping.

Caution:

Contact your PC or laptop manufacturer before connection. Rumble Tuff is NOT responsible for PC or laptop connection problems.

Storage and Preparation of Breast Milk

BEFORE EXPRESSING/PUMPING MILK

Wash your hands well with soap and water.

Inspect the pump kit and tubing to make sure it is clean.
Replace moldy tubing immediately.

Clean pump dials and countertop.

STORING EXPRESSED MILK

Use breast milk storage bags or clean food-grade containers with tight fitting lids.

Avoid plastics containing bisphenol A (BPA) (recycle symbol #7).

HUMAN MILK STORAGE GUIDELINES

TYPE OF BREAST MILK	STORAGE LOCATIONS AND TEMPERATURES		
	Countertop 77°F (25°C) or colder (room temperature)	Refrigerator 40 °F (4°C)	Freezer 0 °F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1–2 Hours	Up to 1 Day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

Storage and Preparation of Breast Milk

STORE

Label milk with the date it was expressed and the child's name if delivering to childcare.

Store milk in the back of the freezer or refrigerator, not the door.

Freeze milk in **small amounts of 2 to 4 ounces** to avoid wasting any.

When freezing leave an inch of space at the top of the container; breast milk expands as it freezes.

Milk can be stored in an insulated cooler bag with frozen ice packs for **up to 24 hours** when you are traveling.

If you don't plan to use freshly expressed milk **within 4 days**, freeze it right away.

THAW

Always thaw the oldest milk first.

Thaw milk under lukewarm running water, in a container of lukewarm water, or overnight in the refrigerator.

Never thaw or heat milk in a microwave. Microwaving destroys nutrients and creates hot spots, which can burn a baby's mouth.

Use milk **within 24 hours** of thawing in the refrigerator (*from the time it is completely thawed, not from the time when you took it out of the freezer*).

Use thawed milk **within 2 hours** of bringing to room temperature or warming.

Never refreeze thawed milk.

FEED

Milk can be **served cold, room temperature, or warm.**

To heat milk, place the sealed container into a bowl of warm water or hold under warm running water.

Do not heat milk directly on the stove or in the microwave.

Test the temperature before feeding it to your baby by putting a few drops on your wrist. It should feel warm, **not hot.**

Swirl the milk to mix the fat, which may have separated.

If your baby did not finish the bottle, leftover milk should be used **within 2 hours.**

CLEAN

Wash disassembled pump and feeding parts in a clean basin with soap and water. **Do not wash directly** in the sink because the germs in the sink could contaminate items.

Rinse thoroughly under running water. Air-dry items on a clean dishtowel or paper towel.

Using clean hands, store dry items in a clean, protected area.

For extra germ removal, sanitize feeding items daily using one of these methods:

- clean in the dishwasher using hot water and heated drying cycle (*or sanitize setting*).
- boil in water for 5 minutes (*after cleaning*).
- steam in a microwave or plug-in steam system according to the manufacturer's directions (*after cleaning*).

June 2019

Centers for Disease
Control and Prevention
National Center for Chronic
Disease Prevention and
Health Promotion

FOR MORE INFORMATION, VISIT:

<https://bit.ly/2dxVYLU>

296657-B

Interruption of Electrical Service

If you experience a power interruption due to a power surge or other factors, while operating your pump using the AC/DC Adapter, try the following:

- Unplug the AC/DC Adapter from the electrical outlet and wait at least for 5 seconds or until the power has resumed.
- Plug the AC/DC Adapter back into the electrical outlet.
- If the problem persists, try a different electrical outlet.

Radio Frequency Energy

This equipment generates, uses, and can radiate radio frequency energy. If not installed properly and used in accordance with the instructions, it may cause harmful interference with other devices in the vicinity. Try the following steps if interference occurs:

- Reorient or relocate the device receiving interference.
- Increase the distance between the equipment.
- Connect the equipment to an outlet on a circuit different from that which the other device(s) are connected.
- Consult the manufacturer or a field service technician of the device receiving interference for help.
- Please understand that there is no guarantee that interference will not occur.

Decreased Suction

- Make sure the tubing is securely attached to the pump and the Collection Combo Kit.
- If problem happens in one side of the Combo Kit, try to switch the two Collection Combo Kits to see if the problem is now switched to the different collection Combo Kit.
- If yes, the problem may be at the Tubing Splitter. Contact Technical Support for further assistance.
- If not, follow the following steps to troubleshoot the problem in the Expression Collection Combo Kit.
- Make sure the Expression Collection Combo Kit is assembled correctly (see pages 8– 12).
- Center your nipple in the Flange and make sure there is a complete seal between the Flange and your skin.
- Press the Diaphragm firmly into the Cup and make sure there is no gap (see page 9 step 6-7).

- Check the Membrane Valve and make sure that it is properly sealed and securely seated (see page 8).
- Inspect the Membrane and Valve very closely for tears or warping. This component is critical to achieve proper suction; even the tiniest tear will adversely affect pump performance. If damaged, replace the Membrane Valve with one of the spare valves provided with your Rumble Tuff Whisper .

Pump Doesn't Start

- Make sure the AC/DC Adapter is properly plugged in.
- Ensure the USB cable is well connected.
- If the Pump still does not work, contact Rumble Tuff Customer Service for assistance (see Page 22).

Breast Milk Buildup in the Breastshield Body

If breast milk doesn't appear to be draining fast enough into the Collection Bottle, reduce the suction setting on your Electric Breast Pump to allow the milk to drain before gradually increasing the suction to the original setting.

Where to Get Replacement Parts

If you are missing a part or if a part was damaged, please contact Rumble Tuff Customer Service.

- Email: customerservice@rumbletuff.com
- Toll-Free: (855) 228-8388
- Website: www.rumbletuff.com

Where to Purchase Additional Accessories

Additional accessories and replacements parts are available for purchase for use with your Rumble Tuff Whisper.

- Email: sales@rumbletuff.com
- Toll-Free: (855) 228-8388
- Fax: (801) 796-2688
- Online Store: www.rumbletuff.com/store

Condensation or Milk in Tubing:

If you find condensation in the tubing, wash and dry it immediately or replace the tubing. Rumble Tuff Whisper Electric Breast Pump has anti-backflow diaphragm on top of the Collection Kit to prevent milk over flow through the tubing into main breast pump unit. Do not attempt to wash, clean or sterilize the tubing or tubing connector if you see breast milk in either of these parts. Instead contact Rumble Tuff Customer Service.

- E-mail: customerservice@rumbletuff.com
- Toll-Free: (855) 228-8388
- Fax: (801) 796-2688
- Online Store: www.rumbletuff.com/store

Tips

- *If your breasts feel uncomfortable or in pain while using the Pump, try decreasing the suction level by pressing the Vacuum Adjusting Button down ⊖.*
- *You may also try pumping with/without the Silicone Cushion.*
- *If these tips do not help, turn the pump off and remove the Collection Bottles from your breasts for a few minutes. If you are still experiencing pain or discomfort, consult your healthcare professional or an international board certified lactation consultant.*
- *Your satisfaction is our highest priority. Please contact us if you experience any discomfort while using our pump.*

Position the device to ensure easy disconnection of the power cord from the supply mains.

To learn more about your pump, please visit www.rumbletuff.com or scan this QR Code with your smartphone or tablet.

The breast pump needs special precautions regarding EMC(Electromagnetic Compatibility) and needs to be installed and put into service according to the EMC information provided in the instructions for use. Portable and mobile RF communications can affect the breast pump.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Guidance and manufacturer's declaration – electromagnetic emissions		
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment .
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning :

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment. The Whisper Breast Pump has anti-backflow as essential performance, which is not influenced by electromagnetic. Further details describe on Page 4 Warning under Safeguard. This device was tested for immunity to electromagnetic disturbances and passed using the following criteria:

1. Change of operation mode except reset into standby mode (pump is switched off) .
2. Change of pressure value more than 10%, when the device is operated as intended.
3. Change in programmable parameters, including mode and suction level.
4. Error indicated and is not restartable.
5. Permanent hardware failure is induced in the device.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.

Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 2 kV, 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environ- ment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment .
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump require s continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery .
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50 or 60 Hz	30 A/m, 50 Hz / 60 Hz	It may be necessary to position the breast pump further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

NOTE : UT is the a.c. mains voltage prior to application of the test level .

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol : 
Radiated RF IEC 61000-4-3	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	

Note 1 : At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 : These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720						
1845	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1970						
2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

EMC tests have included the AC adapter.

Limited Consumer Product Warranty (United States)

This Rumble Tuff® Electric Breast Pump is warranted to the original consumer, the purchaser, to be free from defects in material and workmanship not commercially acceptable for a period of 90 days from the date of purchase with an extended ONE year warranty on the pump motor. This warranty applies to the original owner only. Warranty coverage terminates if you sell or otherwise transfer this product to another person. This warranty gives you specific legal rights and you may also have other rights, which vary by location.

RUMBLE TUFF MAKES NO EXPRESS WARRANTY OF ANY KIND REGARDING THIS PRODUCT OTHER THAN THOSE WARRANTIES SET FORTH HEREIN. ANY IMPLIED WARRANTY, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE TO THE EXTENT PERMITTED BY LAW, SHALL BE LIMITED IN DURATION TO A PERIOD OF 90 DAYS FROM THE DATE OF PURCHASE BY THE ORIGINAL PURCHASER.

In the event that this product is found by Rumble Tuff to not meet the above limited warranty standard, as the purchaser's sole and exclusive remedy, Rumble Tuff will repair, or, at the discretion of Rumble Tuff, replace this product without charge for such replacement parts or labor. **The owner shall bear all expenses related to returning this product to Rumble Tuff.** This warranty does not apply to any part of the product that has been subject to misuse, abuse, or alteration. Only Rumble Tuff brand replacement parts are recommended for use with Rumble Tuff breastfeeding products. Improper or incorrectly performed maintenance or repair, or use of non-Rumble Tuff brand replacement parts voids this warranty.

To ensure the pump always performs at its peak performance, we suggest once per year maintenance service on the pump. Maintenance service is not part of the warranty and can be obtained from Rumble Tuff certified technician. All maintenance fee and related shipping expenses are sole responsibility of the registered owner.

TO THE EXTENT PERMITTED BY LAW, Rumble Tuff® SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- The entire original Rumble Tuff® product
- The original receipt showing date of purchase
- Return Authorization Number (RA#) - Please contact Rumble Tuff® customer service by calling toll-free: (855) 228-8388 for a RA#
- Detailed description of the problem

You, the purchaser, are responsible for all shipping charges.

Do NOT send product without a RA#.

To obtain an RA# and shipping address, please contact Rumble Tuff® Customer Service Department by:

- Toll-free: (855) 228-8388
- E-mail: customerservice@rumbletuff.com
- In writing:
Customer Service Department
Rumble Tuff
865 N 1430 W
Orem UT 84057

Items returned without a RA# will not be processed and will be returned to sender (postage due).

Power Requirements:

- AC/DC Adapter (included) North American Plug Configuration:
- Model: GQ06-050100-ZU

In	Out
100V~240V~50/60 Hz	5V ===
max. 0.3A	1A

Technical Specifications:

- Vacuum:
 - Stimulation -68~-180mmHg (-9~-24kpa)
 - Expression -98~-263mmHg (-13~-35kpa)
- Ambient Temperature range: +10°C to 40°C (50°F to 104°F)
- Relative Humidity range: 20% to 80%
- Ingress protection level:IP21
- Transportation / Storage Environment : -10 to 50°c / 20 to 90% relative humidity

The meaning of the symbols found on the product and packaging are explained in below.

Symbols found on this instruction	
	Caution. Please consult the instruction for use for important cautionary information. ¹
	Non-ionizing electromagnetic radiation. Indicates that the equipment includes RF transmitters. ²
Symbols found on the main unit	
	Indicates On/Off button. ³
	Dispose the device in accordance with local regulation. ⁴
	Consult instruction for use. ⁵
	Type BF equipment (Breastshield). ⁶
	Indicates direct current. ⁷
	Polarity of power supply.

¹ ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1 : General requirements, Clause 5.4.4 Caution

² IEC 60878 Graphical symbols for electrical equipment in medical practice

³ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 29 Stand-by

⁴ EN 50419, Marking of Electrical and Electronic Equipment in accordance with Article 11(2) of Directive 2002/96/EC (WEEE)

⁵ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 11 Operating instructions

⁶ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 20 TYPE BF APPLIED PART

⁷ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 4 Direct current

⁸ Code of Federal Regulations, Title 47, Part 15b/ 15c

⁹ IEC 60417-5957, Graphical symbols use on equipment, for indoor use only

¹⁰ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 9 CLASS II equipment

Symbols Legend

Symbols found on the power adaptor	
	Indicates the product compliance with the requirements of the Federal Communications Commission. ⁸
	Indicates the device for indoor use only. ⁹
	Dispose the device in accordance with local regulation. ⁴
	The UL LISTED mark indicates that the product is manufactured in compliance with UL safety requirements for USA and Canada).
	Indicates direct current. ⁷
	Polarity of power supply.
	CLASS II Equipment. ¹⁰
Symbols found on the packaging	
	Dispose the device in accordance with local regulation. ⁴
	Type BF equipment (Breastshield). ⁶
	Consult instruction for use. ⁵

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