

Rumble Tuff® Baby Nasal Aspirator

BN302/BN303



Instant relief for baby's stuffy nose.

The Rumble Tuff® Baby Nasal Aspirator is the best way to provide your baby with instant, gentle relief for a runny or stuffy nose. Users can effortlessly monitor and control the suction level. The clear silicone tip is easily removable for cleaning and the dual-chamber design is hygienically safe.

How to Use:

1. Tuck nozzle tightly into baby's nostril to create a seal.
2. Put suction end into caregiver's mouth. Let lips rest against the flower-petal stopper.
3. Start sucking through the mouth piece. Maintain or increase suction strength until baby's mucus is trapped inside the collection chamber.
4. Gently remove nozzle from baby's nostril.

How to Clean:

1. Pull up on the nasal tip to remove collection cup from the aspirator. Pull tubing away from the other end of the aspirator.
2. Gently squeeze collection cup sides together to open the bottom of the cup.
3. Over a sink, wash collection cup with warm water. Rinse tubing, starting with the mouthpiece and letting the warm water flow through tubing and out into the sink.
4. Allow all parts to completely dry before reassembling and/or placing in the storage container.

Tips:

1. Put an over-the-counter saline drop in baby's nostril to moisten sensitive and dry nasal tissue. (You can make your own simple saline solution by adding 1/4 teaspoon of salt into 1 cup of clean water.)
2. The manual Baby Nasal Aspirator is more effective when baby is crying or when the mouth is open.

Warning/Caution:

- Wash all parts with warm water before and after each use. Pat dry with a cloth or paper towel. Let each part dry completely before reassembling and/or storing.
- Do NOT sterilize with boiling water or steam.
- Do not expose to extreme heat or cold. Do not expose to direct sunlight. Store your manual Baby Nasal Aspirator in a cool, dry place away from children. Use only under adult supervision. The nasal aspirator is not a toy.
- Fluid or moisture from the care-givers mouth may appear in the tubing. Please follow the cleaning instructions above.
- Dispose of Baby Nasal Aspirator after any visible wear or damage.

Specifications:

BPA Free.

Silicone nozzle.

Patents:

US #6,110,193

Germany #299 12 596.3

China #ZL 99 2 11211.7

Rumble Tuff®

865 N 1430 W Orem UT 84057

Ph: 801-609-8168 Fax: 801-796-2688

Email: customerservice@rumbletuff.com

www.rumbletuff.com

Made in China.