Rumble Tuff®

BREEZE

Electric Breast Pump Duo

Instructions & Assembly

Model: 209DA2

English 1-38
Español 39-80

Congratulations on your new Rumble Tuff® Breeze Electric Breast Pump Duo! The Rumble Tuff® Electric Breast Pump Duo is perfect for many occasions.

- Collecting breast milk while you're away from your baby
 - for example: when you're at work
- Relieving engorgement
 - for example: when your baby's feeding habits change because they're eating more solids
- Maintaining your milk supply if you have to temporarily stop breastfeeding
 - for example: to express unsuitable milk while you are undergoing medical treatment

The Rumble Tuff® Electric Breast Pump Duo provides many of the features busy moms love.

- 3 cycles of expressions to mimic baby's nursing patterns.
 - Reflex Expression - Latch on & Rooting habit
 - Stimulation - Fast but shallow swallowing habit
 - Expression - Steady and strong swallowing pattern
- Milk back-flow protection
- Quiet motor for discretion
- Small and lightweight
- 2 power options to fit your needs
 - AC/DC Adapter (included)
 - Built-in Li-Ion Rechargeable Battery
- Convert to Manual Pump in seconds (parts included)

Indication for Use

The Rumble Tuff® Electric Breast Pump Duo is a personal use, electrically powered device used to express milk from the breasts of lactating women. This device is not intended for hospital use.

Contraindications for use

There are no known contraindications for use with this product.

1. Introduction 1

2. Important Safeguards 3

3. Product Description 6

4. Start-up Procedure and Operation 9

 4.1 Cleaning 9

 4.2 Assembly 12

 4.3 Operating Breeze 15

 4.4 Power options 18

5. Special Features 20

6. Set the Clock and Alarm 21

7. Assemble the Manual Pump Diaphragm Kit 24

8. Breast Milk Storage and Thawing 27

9. Troubleshooting 28

10. Warranty 32

11. Legend and Symbols 33

12. Electromagnetic Compatibility (EMC) 34

13. Technical Data 38

The Rumble Tuff® Electric Breast Pump is designed for lactating women to express and collect breast milk. It is intended for a single user and should never be shared or resold; doing so will pose health risks and void the warranty.

Please read this document completely and follow the instructions exactly. You should always keep it within reach. To prevent personal injury and material damage, please observe the warning and safety information provided in this document.

DANGER	Can lead to serious injury or death
WARNING	Can lead to serious injury or death
CAUTION	Can lead to minor injury or material damage
<i>Note</i>	Useful breastfeeding and/or product information

DANGER:

To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING:

- Anti-Backflow is the essential performance of Rumble Tuff Breast Pumps. If any moisture/milk is found in the tubing while using or after using the pump, stop using the pump and contact Rumble Tuff Customer Service immediately.
- Use only the AC/DC Adapter that comes with the Whisper Breast Pump
- Never leave plugged-in product unattended.
- Plug the AC/DC Adapter into the electrical outlet, before connecting the Power Cord into your pump.
- Make sure the voltage of the AC/DC Adapter is compatible with the power source.
- Do not operate this product if the cord or plug is damaged, if it is not working properly, if it has been dropped or damaged, or if it has been dropped in water.
- Keep the cord away from heated surfaces.

2. Important Safeguards

- Keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- Do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or breast pump parts.
- Never put the pump unit into water or sterilizer as it can cause permanent damage to the pump.
- If you are infected with Hepatitis A, Hepatitis B, or have HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- Do not attempt to modify the product.
- Do not pump while driving.
- Using a breast pump on an aircraft is not recommended. The cabin pressure may affect the breast pumps performance.
- If there is any potential allergic reaction, stop using our products and find a medical advice immediately.

CAUTION:

- Do not use this product for any other purpose than as indicated in this manual.
- Make sure all of the parts are assembled correctly before operating.
- Stop using the breast pump immediately if your breasts or nipples experience pain or discomfort.
- Please wash hands before using this product.
- Do not drop or hit the product. Doing so may cause damage not covered by the warranty.
- Do not touch the Expression Collection Kits with bare hands immediately after sterilization by boiling water to avoid scalds and burns.
- Do not overboil product materials; to prevent premature aging.
- Do not use any accessories, parts, or attachments that are not recommended by Rumble Tuff. Doing so may affect the performance of your breast pump and void the warranty.

2. Important Safeguards

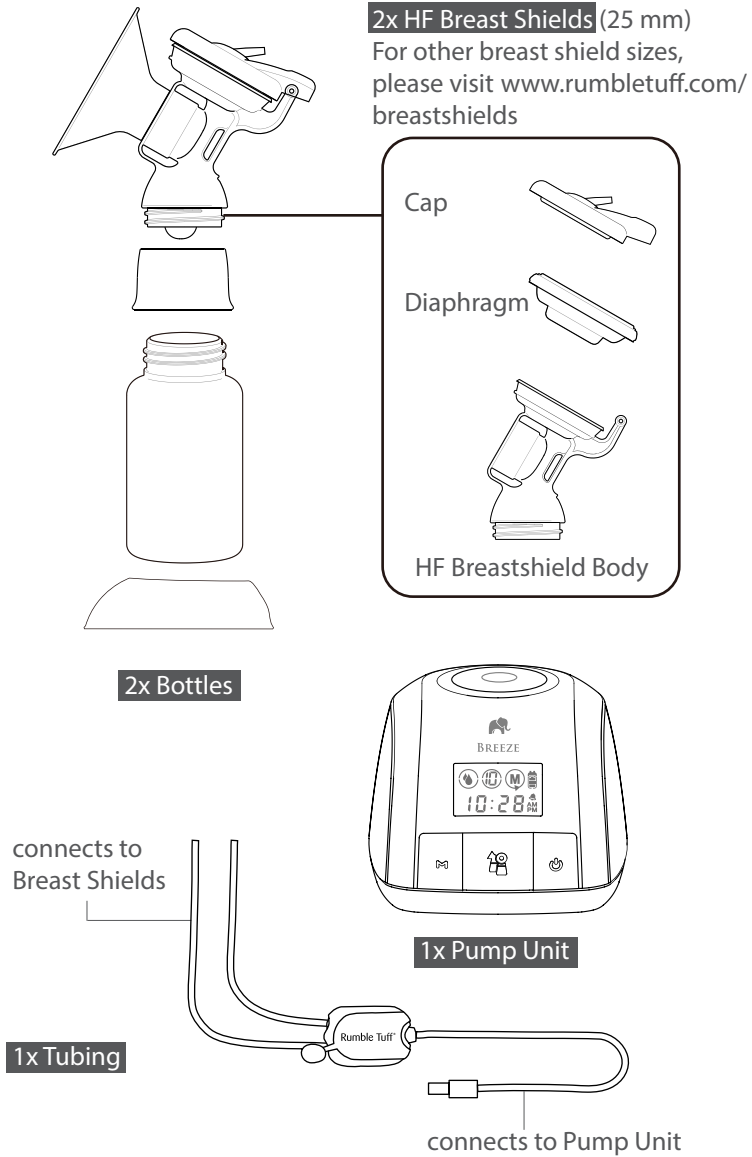
- Do not connect the pump or any of its accessories to any equipment not recommended by Rumble Tuff.
- Do not store the product in a dusty, or damp environment, or in direct sunlight.
- If irritation or discomfort occurs, discontinue use and see a doctor.

Note:

When disposing of the pump or its accessories, please consult with your local authorities for the proper method of disposal of potentially biohazardous parts and accessories, or to learn if there are any other regulations.


SAVE THESE INSTRUCTIONS
along with your receipt for future reference.

3. Product Description

3. Product Description

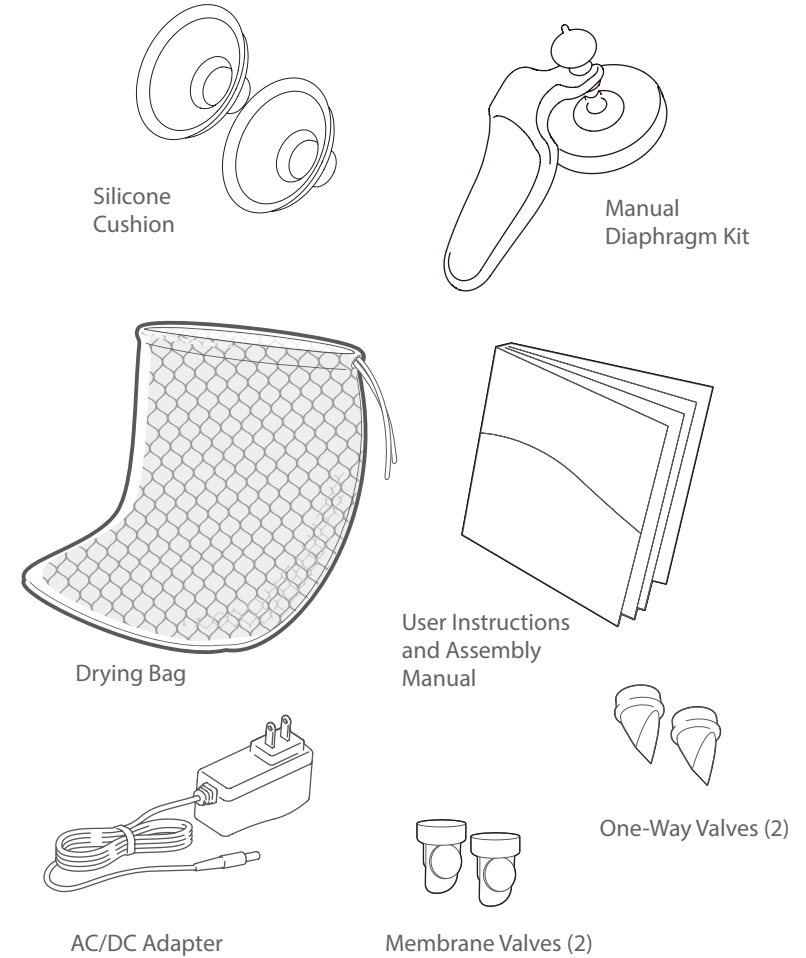
Note: For ordering information about accessories, please visit www.rumbletuff.com/accessories; or contact our customer service. Toll-Free: (855) 228-8388


3. Product Description

4. Start-up Procedure and Operation

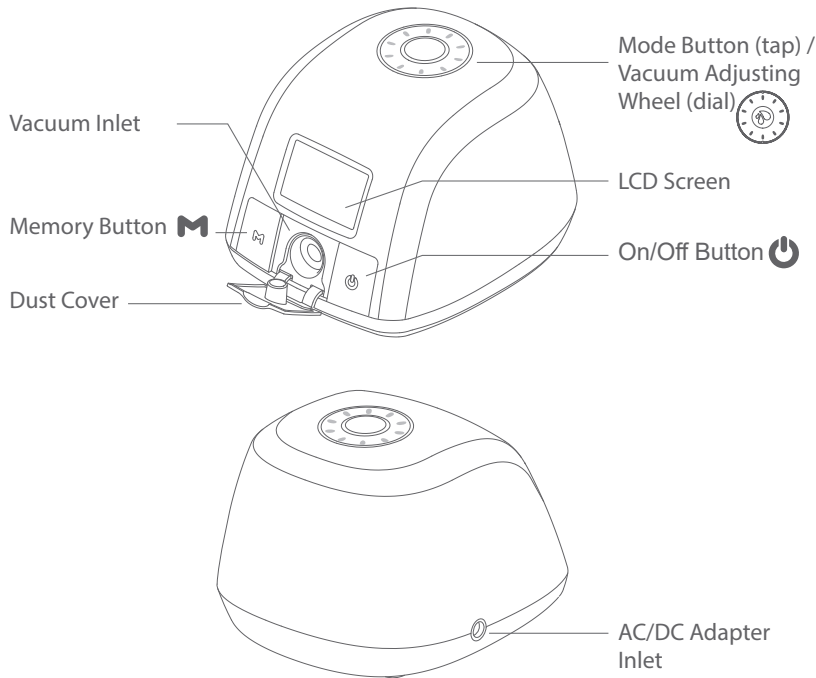Electric Breast Pump Main Unit

Expression Indicator Lights

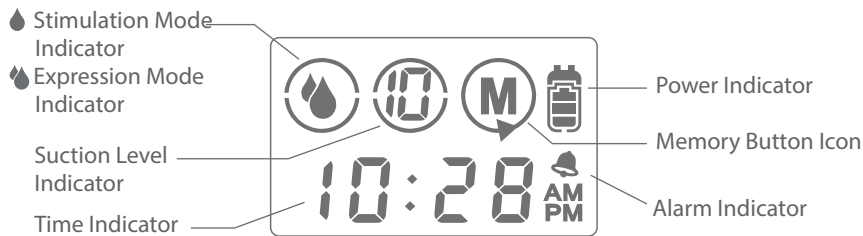4.1 Cleaning

Sanitize Prior to First Use:

Before you use your Rumble Tuff® Breeze Electric Breast Pump for the first time, disassemble and boil all parts of the Expression Collection Combo Kit in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Remove all parts from the water immediately after sterilizing. To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- Completely air-dry.

Wet components may affect the performance of your Electric Breast Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water for at least 20 seconds.
2. **Assemble.** Assemble the clean Express Collection Combo kit parts. Inspect each component for any cracks, tears, discoloration, or deterioration. If damage is found, please discontinue use and replace the part immediately. Inspect whether the kit parts or tubing have become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.
3. **Clean if using a shared pump.** Wipe pump dials, Power Button, and pump housing with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.

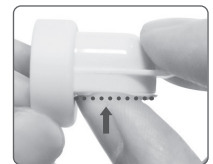
Special Cleaning Instruction for the Membrane Valve:

For your convenience in cleaning, the Membrane Valve is specially designed with a node on the stem of the Membrane.

To clean the Membrane, gently pull the Membrane away from the Valve until it snaps slightly on the stem but does not completely detach from it.

Clean the Membrane Valve in warm, soapy water, then rinse thoroughly in clean, warm water or warm running water.

After cleaning, push the stem back into place and make sure the Membrane lays flat against the Valve opening.

After Every Use:

1. **Store milk safely.** Cap the milk collection bottle or seal the milk storage bag, label with date and time, and immediately place in a refrigerator, freezer, or cooler bag with ice packs.
If the milk collection container will be stored at a hospital or childcare facility, it's recommended to add your name to the label.
2. **Clean pumping area.** Especially if using a shared pump, wipe the dials, power switch, and countertop with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %) is recommended.
3. **Take apart and inspect the Collection Kits.** Remove the breast pump tubing and set it aside. Separate all the parts that come in contact with breast/breast milk (for example: Silicone Cushions, Flanges, Membrane Valves, Diaphragm and Milk Collection Bottles).
4. **Rinse Collection Kits.** To remove any milk residue, rinse the Collection Kit parts that come into contact with breast/breast milk under running water
5. **Clean Collection Kits.** As soon as possible after pumping, clean Collection Kits that come into contact with breast/breast milk in one of the following ways.

Clean by hand

1. **Use a wash basin.** Place Collection Kit parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the pump.
2. **Add soap and water.** Fill wash basin with hot water and add soap.
3. **Scrub.** Scrub items using a clean brush that is used only to clean infant feeding items.
4. **Rinse.** Rinse by holding items under running water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. **Dry.** Allow to air-dry thoroughly. Place Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. Rumble Tuff also provides a Drying Mess Bag for your convenience to air dry all Collection Kit parts.

Clean in a dishwasher

1. Wash. Place disassembled Collection Kits parts in dishwasher. Be sure to place small items into a closed-top basket so they don't end up in the dishwasher filter. If possible, run the dishwasher using hot water and a heated drying cycle (or sanitizing setting); this can help kill more germs.
 2. Remove from dishwasher. Wash your hands with soap and water before removing and storing cleaned items. If items are not completely dry, place items on a clean, unused dish towel or paper towel to air-dry thoroughly before storing or use the provided Drying Mess Bag to air dry all parts. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.
6. **Clean wash basin and bottle brush.** If you use a wash basin or bottle brush when cleaning your Collection Kits parts, consider washing them every few days, either in a dishwasher with hot water and a heated drying cycle, if they are dishwasher-safe, or by hand with soap and warm water. Rinse them well and allow them to air-dry after each use.
 7. **Tubing**
 - Wipe the outside of the Tubing with a clean, damp cloth. Avoid getting water or other fluids inside the Tubing.
 - If liquid is found inside the Tubing, rinse the Tubing with clean, hot water. Hang to AIR DRY COMPLETELY BEFORE USING. Do NOT insert any object into Tubing.

Store Safely Until Needed

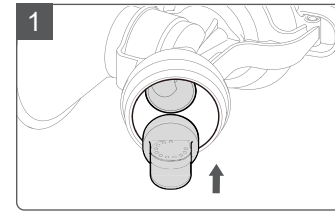
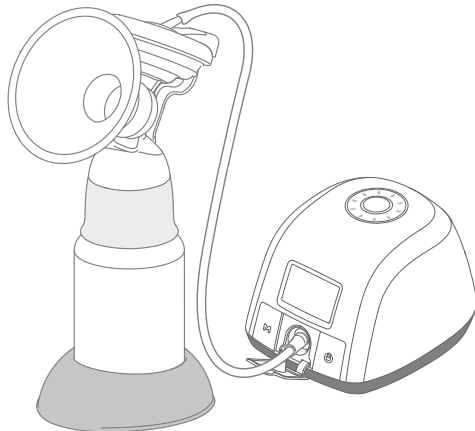
To help prevent germs and mold from growing, allow the clean Collection Kit parts, bottle brushes, and wash basins to air-dry thoroughly before storing. Once completely dry, and to prevent contamination during storage, the items should be stored in a clean, protected area.

- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Collection Kit parts.
- **Store safely.** Place reassembled Collection Kit in a clean, protected area such as inside an unused, sealable food storage bag. Store wash basins and bottle brushes in a clean area.


Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing is designed to not touch the pumped milk and does not need to be cleaned routinely.
- Keep a spare set of tubing on hand in case the set you are using gets soiled or damaged.
- At the end of a pumping session, if your tubing has water droplets in it, disconnect the tubing from the Collection Kits, check to see if the Diaphragm needs to be replaced.
- If your tubing has mold in it, throw it away immediately because it is difficult to clean properly. Replace it with a new set of tubing, and check to see if the Diaphragm needs to be replaced.
- If the outside of your tubing is soiled, wipe it with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %) is recommended.
- Do not insert any object into the Tubing.


4.2 Assembly

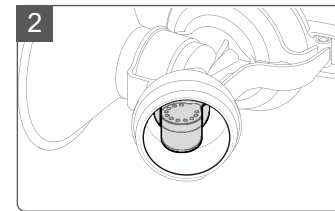With the membrane facing towards you, firmly insert the Membrane Valve into the HF Breast Shield Body and position valve as shown in diagram 2.

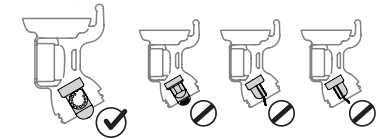Confirm that the Membrane Valve is securely installed.


Make sure the Membrane Valve is inserted exactly as shown in the left diagram in order to maintain pumping efficiency.


Verify that the membrane of the Membrane Valve is flat. A curled membrane will result in suction loss.


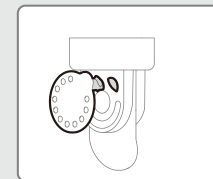
How to Assemble the Membrane Valve:

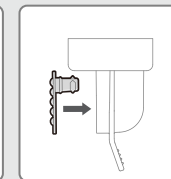Hold the Silicone Membrane and insert the small stem into the top-right opening of the Valve.

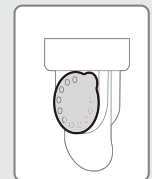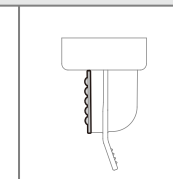
Press gently and firmly on the Membrane to ensure that the stem is completely inserted and is snug against the Valve opening.


Membrane Valve Assembly completed.

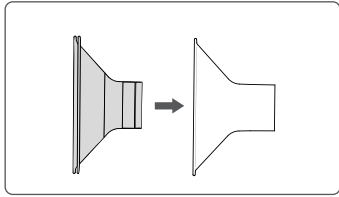4. Start-up Procedure and Operation


If desired, place the Silicone Cushion into the Flange of the Breastshield.

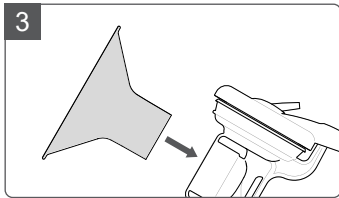Note: Your Electric Breast Pump will work with or without the Silicone Cushion. The usage of the cushion is entirely optional and based on your comfort and preference.

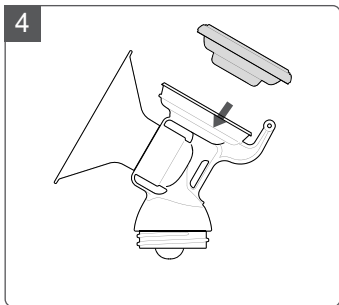
3 Insert the Flange into the Breastshield Body.


4 Place the Diaphragm in the Cup of the Breastshield Body.

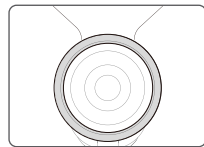
⚠ Note: Do not reverse the order of steps 4 through 5. Doing so will result in suction loss when pumping.

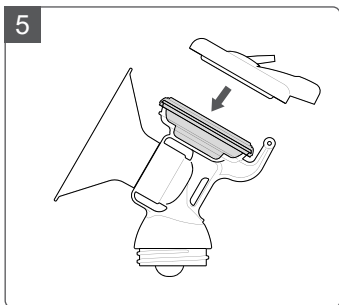Press around the edges of the Diaphragm for a secure fit.

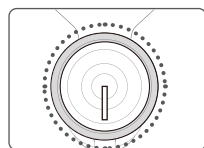
5 Place the Cap on the Cup and Diaphragm.

Align the hinge cover over the hinge. Press tightly on the cap to seal.

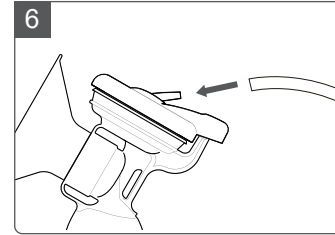
4. Start-up Procedure and Operation


6 Connect the Tubing to the Tubing Connector located on the Cap.

Note: Make sure there is no moisture in the Tubing.

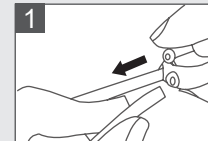
To Convert Your Double Pump Into a Single Pump: Remove one tube and close the cap.

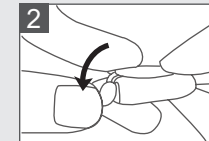1 Disconnect the tube from the Tubing Splitter on the side with the little Cap

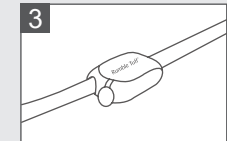
2 Cap the opening of the Tubing Splitter


3 Ready to use as a single pump.

Note: Using the pump after converting from Single to Double or Double to Single may result in a temporary lack of suction or over-suction respectively. Please use caution and adjust the pump to your comfortable and effective level.

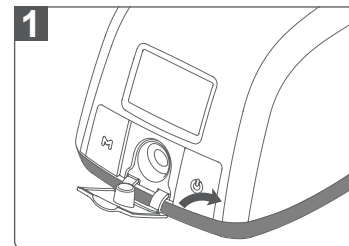4.3 Operating Breeze

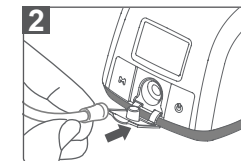
1. Open the dust-proof cover and plug in the tubing adapter.


2. Make sure the Tubing Adapter is properly inserted into the Vacuum Inlet on the Electric Pump Main Unit.

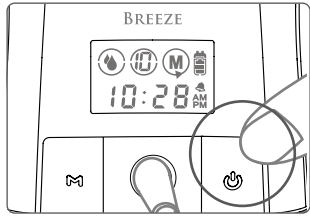
4. Start-up Procedure and Operation

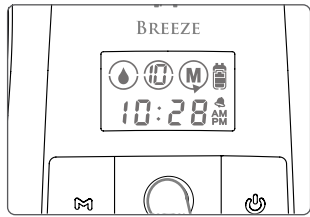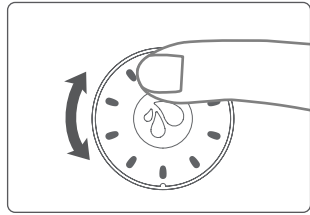Press the Power Button  to begin pumping.

The LCD Screen shows the time of your last pumping session for a few seconds. The pump will automatically start from the Reflex Expression for the flashing 15 seconds, then switch to the Stimulation Mode  automatically.

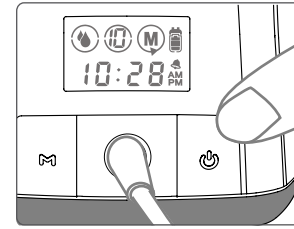
Adjust the suction in the Stimulation Mode to your comfort level during pumping by rotating the Vacuum Adjusting Wheel  clockwise (up/+) or counter wise (down/-)

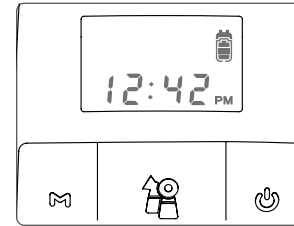
Depending on your preference and your baby's sucking habit, you can choose to express your milk in either the Stimulation Mode  or Expression Mode  settings. To switch from Stimulation Mode to Expression Mode, press the Mode Button  once.

To have Reflex Expression, restart the device by pressing Power Button  to turn off and turn it on again.

4. Start-up Procedure and Operation

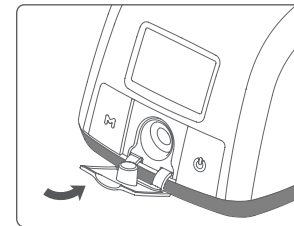
When your pumping session is finished, turn the pump off by pressing the Power Button. 

After the pump has been turned off, the LCD Screen will remain on for a few seconds. The display will show the total amount of pumping time, then automatically switch to Standby/Clock mode, or Alarm Clock Mode if the Alarm was turned on.

 After disconnecting the tubing from the Pump Unit, please make sure to cover up the Vacuum Inlets with the Dust Proof Cover.

Note:

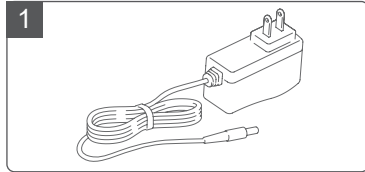
- Make sure there is no suction on the nipples before removing the breastshield. If necessary, break the suction by inserting a finger between the breastshield and the breast, similar to the way you break away from your baby's suction.
- If the Rumble Tuff® Electric Breast Pump Duo is left on, but unattended for more than 30 minutes, the Pump will automatically shut-off.

4. Start-up Procedure and Operation


4. Start-up Procedure and Operation

4.4 Power options

Your Electric Breast Pump may be operated under any of the following power supply modes:

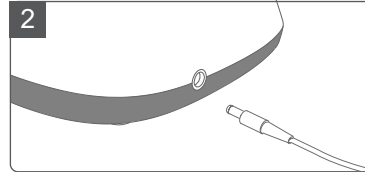1 Plug the AC/DC Adapter into an electrical outlet.

2 Attach the power cord to the breast pump.

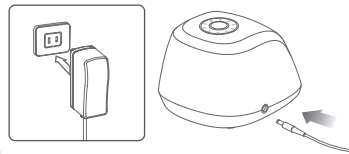
Danger:
If the power adapter outer casings become loose or are separating, contact the device manufacturer immediately. **Separation of the casing can result in electric shock.**

Caution:
Electrical safety can only be achieved when using the AC/DC Adapter provided with the Pump. Use of any another Adapter poses a safety risk and/or the warranty may be voided.

Built-in Li-Ion Rechargeable Battery

Your Rumble Tuff Breeze contains a built-in Li-Ion rechargeable battery.

- Fully charge the battery for 12 hours before the first use on battery power.
- If needed, you may use the pump with the power cord plugged in to an external power source. The battery will charge as soon as the pump is plugged in, but not while the pump is in use.
- After the initial charge, each subsequent charge will take approximately 2.5 hours.
- For best results, keep your battery fully charged and recharge the battery before it completely drains.
- Keep the pump in a cool place.
- When the battery low  is light, recharge the battery as soon as possible by plug in the AC/DC Adapter to the wall outlet and other end of the power cord to the pump.

- Your pump may not run immediately on battery power if it has been stored in a heated location. If this is the case, you can allow the battery to cool down while still using it on an external power source.

Note :

The Li-Ion Rechargeable Battery used in this device may present a risk of fire or chemical burn if mistreated.

Do NOT attempt to open the pump unit to replace the battery or for any other reason, doing so will void the warranty.

Do NOT disassemble the battery.

Do NOT place battery near a heat source or fire.

Dispose of used battery properly.

Do NOT dispose of battery in fire.

Keep away from children.

Battery Charge Status Codes

 External power source connected

 Battery charge status

 Low battery, needs charging
(flashing)

 Battery drained.
(flashing) Fully charge the battery for 6 hours.

Operating time remaining (approx.):

 Full Bar: 2hours  2 Bar: 1.5hours

 1 Bar: 1hour  1 Bar Flash: 30minutes
(flashing)

- Keep the battery charged at 50% capacity and at room temperature (20-25°C/68-77°F) for long-term storage.

- We recommend batteries be charged once every 3 months to prevent over-discharge.

- Battery performance will deteriorate after being used or stored for a long period of time

- Battery life: Continuous charge and discharge for 500 cycles, the capacity will be $\geq 70\%$ of initial capacity.

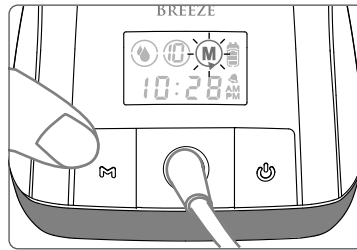
5. Special Features

Peak Enhanced Waveform

Specially programmed expression pattern extends the time your nipple stays engaged in the tunnel allowing your spray to continue longer with each cycle. This allows you to collect more milk in less time.

Memory Function

Memory function allows you to record and recall the pumping session that works best for you. By pressing the Memory Button **M**, the pump will remember the cycles and suction levels of any pumping session you choose.

Note 1: Press Memory Button for 2 sec. to record.

Note 2: Breeze will remember the last suction level automatically without pressing the Memory Button.

Mode Button/Vacuum Adjusting Wheel

Simply tap the Mode/Vacuum wheel to switch between stimulation and expression modes. The wheel allows easy adjustment of suction levels. This emanates mild blue light to increase the visibility when operating at night.

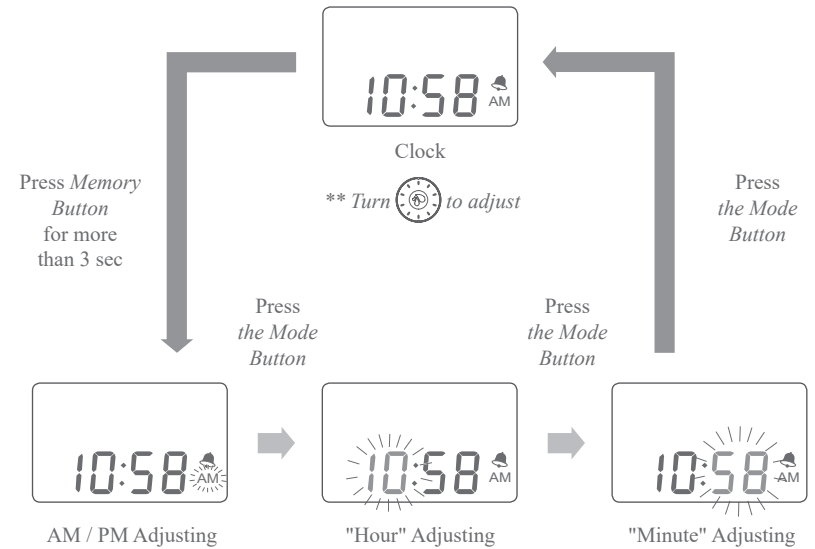
Long-life Rechargeable Battery

Allows for easy pumping in any location, no outlet needed. Quickly charges in only two hours.

6. Set the Clock and Alarm

How to Adjust the Clock

1. While in the Clock/Standby Mode, press and hold the Memory Button **M** for 3 seconds to go into Clock Setting mode. A beep will sound and the AM/PM icon will flash.
2. To change between AM/PM, turn the Vacuum Adjusting Wheel  clockwise or counterclockwise
3. Press the Mode Button  once to change from AM/PM to the hour. The hour number will flash.
4. To change the hour, turn the Vacuum Adjusting Wheel  clockwise, to increase (+), or counterclockwise, to decrease (-), the hour.
5. Press the Mode Button  again to change the minutes. The minute number will flash.
6. To change the minutes, turn the Vacuum Adjusting Wheel  clockwise, to increase (+), or counterclockwise, to decrease (-), the minutes.
7. Press the Mode Button  one last time to save your settings.

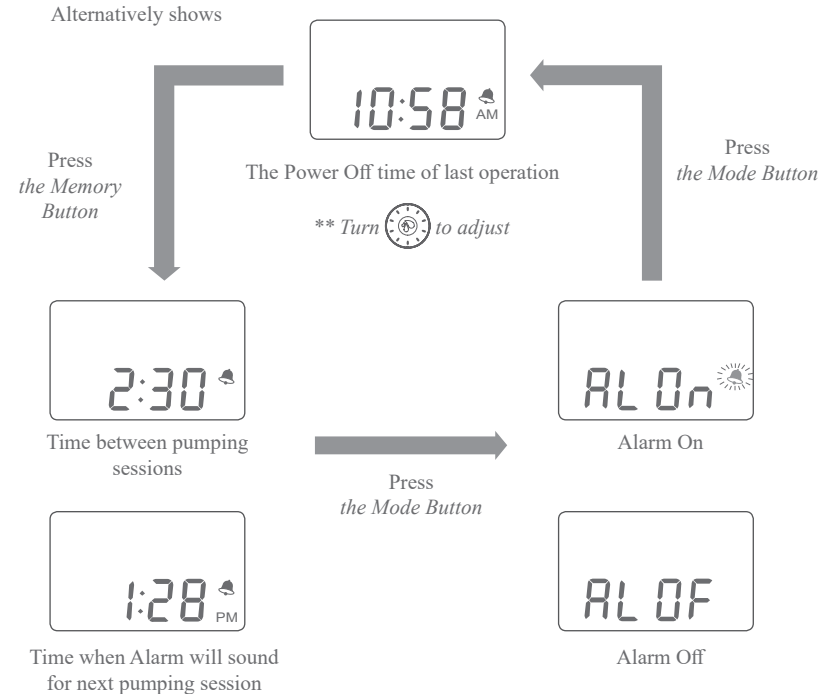
The Alarm Clock Mode is designed to remind you when it's time for your next pumping cycle. By keeping a regular expression pattern, your body maintains a good milk supply. After you set the time interval between each pumping session, the alarm countdown will automatically start after the previous pumping cycle has ended. An alarm will sound when the pre-set time has passed.

1. In Clock/Standby Mode, press the Memory Button **M** once to go into the Alarm Adjusting Mode.
2. LCD Screen will alternately display two sets of time: Alarm Interval time (which is the time between each pumping session) and Next Alarm Time. (The time of your next pumping session). E.g. If the Power Off time of your last session is 10:58 AM and you want to do the next expression session in 2 hours and 30 minutes, then set the Alarm Interval Time as 2 hours and 30 minutes. The Next Alarm Time will be at 1:28 PM. (As shown on P. 25)
3. Turn the Vacuum Adjustment Wheel  clockwise (+) or counterclockwise (-) to adjust the Alarm Interval Time. The adjustment increases (+) or decreases (-) in 15 minute increments. The Next Alarm Time will be adjusted automatically according to the last time the pump was used.



Note: The Alarm Interval Time is the time from the end of the last session to the next alarm set off time. It is NOT the time from the beginning of one expression session to the beginning of the next one.

4. Press the Mode Button  to switch to the Alarm On/Off setting.
5. Turn the Vacuum Adjustment Wheel  clockwise (+) or counterclockwise (-) to turn the Alarm Function On/Off.
6. Press the Mode Button  again to save your setting.


Note: If no adjustments are made within 10 seconds in the Alarm Adjusting Mode, the setting will automatically save and the LCD Screen will return to Clock/Standby Mode.

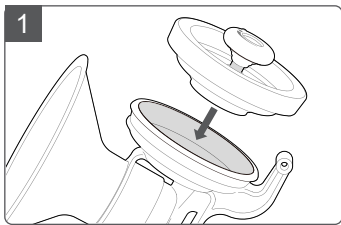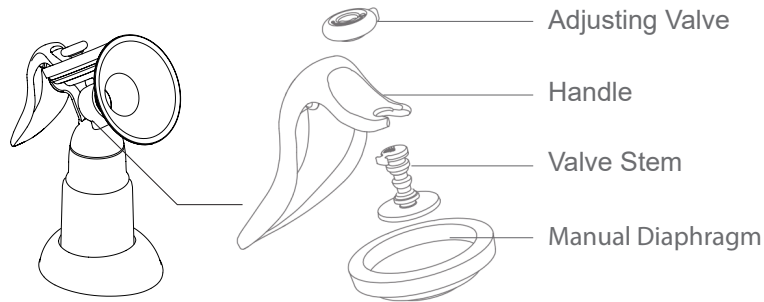7. Assemble the Manual Pump Diaphragm Kit

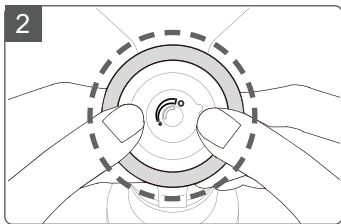
With the Cup in one hand, place the Manual Silicone Diaphragm in the Cup (matching the shape of the Cup).


Press around the edges of the Diaphragm for a secure fit. Make sure the Diaphragm is evenly mounted on the Cup and there is no gap in between them.

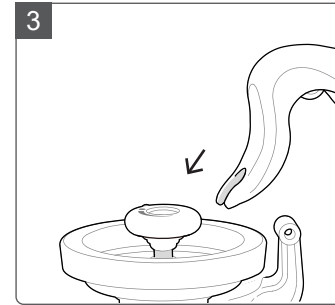
7. Assemble the Manual Pump Diaphragm Kit

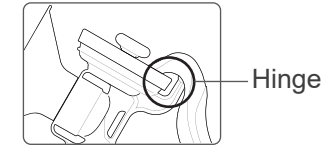Push the Handle down onto the hinge of the Cup until it "snaps" into place.

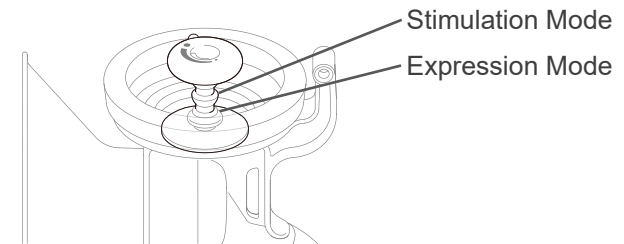Note: There are two recesses for attaching the handle to the Manual Pump. You can toggle between these two settings by pushing the Adjusting Valve Stem away from the "U" shape tip of the Handle to dislocate the connection. Reattach the tip of the Handle using a different recess on the Adjusting Valve Stem.


Stimulation Mode:

This setting is for fast, shallow suction. Often when a baby is hungry or has just "Latched-On", their nursing pattern is fast and shallow to start the mother's milk flow.

Expression Mode:

This setting is for longer, deeper suction. When the mother's milk begins to flow into the baby's mouth, they often naturally change to a longer deeper suction pattern.

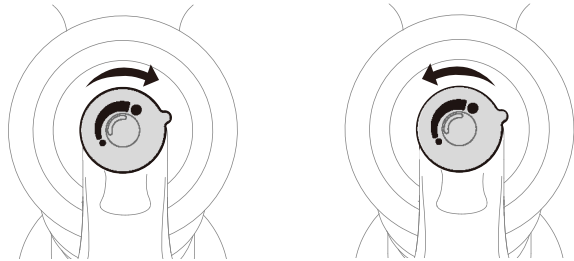7. Assemble the Manual Pump Diaphragm Kit

Increase

Decrease

Note: If an adjustment is necessary, turn the round wheel of the Adjusting Valve to a comfortable suction level.

Caution:

Do not over-turn the Adjusting Valve, as it will cause the Adjusting Valve to bulge and leak air, creating a suction problem. Make sure the gap opening on top of the Adjusting Valve is lined up properly on the top of the Valve Stem so the Adjusting Valve turns smoothly. If necessary, re-adjust the Adjusting Valve by turning it back and forth a few times.

8. Breast Milk Storage and Thawing

Storage Location and Temperatures			
TYPE OF BREAST MILK	Countertop 77°F (25°C) or colder (room temperature)	Refrigerator 40°F (4°C)	Freezer 0°F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1–2 Hours	Up to 1 Day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

Source: CDC - Human Milk Storage Guidelines
(https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm)

Note:

1. Always make sure to check the date on the milk's container before using. Do not use undated milk, or milk that has been stored too long.
2. Thaw frozen milk in the refrigerator or under cool, running water.
3. Heat the milk under warm, running water and gently swirl the container to mix the milk.
4. If warm water is not available, heat a pan of water on the stove. Once the water is warm, not boiling, remove the pan from the stove and place milk container in the pan. Never warm the milk container directly on the stove.
5. Never microwave breast milk because microwaves can cause dangerous hot spots that could burn you or your baby.
6. Always test the temperature of the milk on the inside of your wrist before feeding it to a baby. The milk should feel warm, not hot.
7. After thawing, milk should be stored in the refrigerator for no more than 24 hours.
8. Never refreeze thawed milk. Throw away previously frozen milk that is not used within 24 hours.

Problem	Solution
No power	<ul style="list-style-type: none"> • Make sure it's the correct power adaptor for Rumble Tuff model. • Try different wall outlet, if it's still not working, please call Rumble Tuff customer service at Toll-Free:(855)228-8388.
Interruption of electrical service	<p>During normal operation of your Rumble Tuff electric breast pump, if you experience an interruption of electrical service, due to a power outage or other factors, try the following:</p> <ul style="list-style-type: none"> • Unplug the AC/DC Adapter from the electrical outlet and wait for 5 seconds. • Plug the AC/DC Adapter back into the electrical outlet. • If the problem persists, try a different electrical outlet.
Radio frequency energy	<p>This equipment generates, uses, and can radiate radio frequency energy. If not installed properly and used in accordance with the instructions, it may cause harmful interference with other devices in the vicinity. Try the following steps if interference occurs:</p> <ul style="list-style-type: none"> • Reorient or relocate the device receiving interference. • Increase the distance between the equipment. • Connect the equipment to an outlet on a circuit different from that which the other device(s) are connected. • Consult the manufacturer or a field service technician of the device receiving interference for help. • Please understand that there is no guarantee that interference will not occur.
Pump doesn't run when power is "on"	<p>If the Power Symbol flashes  or you see " <i>Err1</i> " and hear 4 Beeps(under Alarm ON), contact the local DME dealer/retailer you purchased the unit from.</p>

No suction or low suction

- Make sure the tubing is securely attached to the pump and the Collection Kits.
- If problem happens in one side of the collection bottle, try to switch the tubing to different vacuum inlet to see if the problem is now switched to the different collection bottle.
- If yes, the problem may be at the pump unit. Contact Technical Support for further assistance
- If not, follow the following steps to troubleshoot the problem in the Expression Collection Combo Kit.
- Make sure the Expression Collection Combo Kit is assembled correctly. (see pages 13 – 16)
- Center your nipple in the Flange and make sure there is a complete seal between the Flange and your skin Press the Diaphragm firmly into the Cup and make sure there is no gap. (see page 15 step 5)
- Check the Membrane Valve and make sure that it is properly sealed and securely seated. (see page 14)
- Inspect the Membrane and Valve very closely for tears or warping. This component is critical to achieve proper suction; even the tiniest tear will adversely affect pump performance. If damaged, replace the Membrane Valve with one of the spare valves provided with your Rumble Tuff Breeze.

Breast milk buildup in the Breastshield Body

If breast milk doesn't appear to be draining fast enough into the Collection Bottle, reduce the suction setting on your Electric Breast Pump to allow the milk to drain before gradually increasing the suction to the original setting.

The breast pump generates only constant vacuum instead of sucking cycles

Contact the Rumble Tuff Customer Service.

The pump unit's exterior got wet

Unplug the pump unit from the power source and turn it off. Dry off the exterior of the pump unit.

The pump unit has been submerged in water	Unplug the pump unit from the power source and turn it off. Contact the Rumble Tuff Customer Service.
The breast pump is not responding as expected	Press the On/Off button to switch the breast pump off, then restart the breast pump. If there is no change, contact the Rumble Tuff Customer Service.
No response while operating Adjusting Knob and Mode button	You CANNOT adjust the suction level or switch between Stimulation Mode and Expression Mode while in My Favorite Pattern. If you have to, stop My Favorite Pattern by pressing the M Button.
Where to get replacement parts	If you are missing a part or if a part was damaged, please contact Rumble Tuff Customer Service. <ul style="list-style-type: none"> • E-mail: customerservice@rumbletuff.com • Toll-Free: (855) 228-8388 • Website: www.rumbletuff.com
Where to purchase additional accessories	Additional accessories and replacements parts are available for purchase for use with your Rumble Tuff® Breeze. <ul style="list-style-type: none"> • E-mail: sales@rumbletuff.com • Fax: 801-796-2688 • Toll-Free: (855) 228-8388 • Online Store: www.rumbletuff.com/store
Condensation or milk in Tubing	If you find condensation in the tubing, wash and dry it immediately or replace the tubing. Rumble Tuff Breeze Electric Breast Pump has anti-backflow diaphragm on top of the Collection Kit to prevent milk over flow through the tubing into main breast pump unit. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connect. Instead contact Rumble Tuff Customer Service. <ul style="list-style-type: none"> • E-mail: sales@rumbletuff.com • Fax: 801-796-2688 • Toll-Free:(855) 228-8388 • Online Store: www.rumbletuff.com/store

Tips

- If your breasts feel uncomfortable or are in pain while using the Pump, try decreasing the suction level by adjusting the Vacuum Adjusting Wheel counter wise (down/-). 
- You may also try pumping with/without the Silicone Cushion.
- If these tips do not help, turn the pump off and remove the Collection Bottles from your breasts for a few minutes. If you are still experiencing pain or discomfort, consult your healthcare professional or an international board certified lactation consultant.
- Your satisfaction is our highest priority. Please contact us if you experience any discomfort while using our pump.

Position the device to ensure easy disconnection of the power cord from the supply mains.

To learn more about your pump, please visit www.rumbletuff.com or scan this QR Code with your smartphone or tablet.

The breast pump needs special precautions regarding EMC(Electromagnetic Compatibility)and needs to be installed and put into service according to the EMC information provided in the instructions for use. Portable and mobile RF communications can affect the breast pump.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Limited Consumer Product Warranty (United States)

This Rumble Tuff® Electric Breast Pump is warranted to the original consumer, the purchaser, to be free from defects in material and workmanship not commercially acceptable for a period of 120 days from the date of purchase with an extended ONE year warranty on the pump motor. This warranty applies to the original owner only. Warranty coverage terminates if you sell or otherwise transfer this product to another person. This warranty gives you specific legal rights and you may also have other rights, which vary by location.

RUMBLE TUFF MAKES NO EXPRESS WARRANTY OF ANY KIND REGARDING THIS PRODUCT OTHER THAN THOSE WARRANTIES SET FORTH HEREIN. ANY IMPLIED WARRANTY, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE TO THE EXTENT PERMITTED BY LAW, SHALL BE LIMITED IN DURATION TO A PERIOD OF 120 DAYS FROM THE DATE OF PURCHASE BY THE ORIGINAL PURCHASER.

In the event that this product is found by Rumble Tuff to not meet the above limited warranty standard, as the purchaser's sole and exclusive remedy, Rumble Tuff will repair, or, at the discretion of Rumble Tuff, replace this product without charge for such replacement parts or labor. **The owner shall bear all expenses related to returning this product to Rumble Tuff.** This warranty does not apply to any part of the product that has been subject to misuse, abuse, or alteration. Only Rumble Tuff brand replacement parts are recommended for use with Rumble Tuff breastfeeding products. Improper or incorrectly performed maintenance or repair, or use of non-Rumble Tuff brand replacement parts voids this warranty.

To ensure the pump always performs at its peak performance, we suggest once per year maintenance service on the pump. Maintenance service is not part of the warranty and can be obtained from Rumble Tuff certified technician. All maintenance fee and related shipping expenses are sole responsibility of the registered owner.

TO THE EXTENT PERMITTED BY LAW, Rumble Tuff® SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- The entire original Rumble Tuff® product
- The original receipt showing date of purchase
- Return Authorization Number (RA#) - Please contact Rumble Tuff® customer service by calling toll-free: (855) 228-8388 for a RA#
- Detailed description of the problem

You, the purchaser, are responsible for all shipping charges.

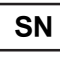
Do NOT send product without a RA#.

To obtain an RA# and shipping address, please contact Rumble Tuff® Customer Service Department by:

- Toll-free: (855) 228-8388
- E-mail: customerservice@rumbletuff.com
- In writing:
Customer Service Department
Rumble Tuff
865 N 1430 W
Orem UT 84057

Items returned without a RA# will not be processed and will be returned to sender (postage due).

The following tables explain the meaning of the symbols found on the product parts and its packaging.

Symbol	Description
	Manufacturer
	Serial number.
	Danger, Warning, Caution
Note	Notice or Tips
	Refer to instruction manual.
	Type BF applied part.
	CLASS II equipment.
	DC
	This product should not be mixed with other commercial waste for disposal.
	Power
	Consult instruction for use.
	CE mark
IP22	Degree of protection against harmful ingress of water or particulate matter.

12. Electromagnetic Compatibility (EMC)

This appliance conforms to the following standards:

IEC 60601-1-11 Medical electrical equipment – Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of IEC 60601-1-2(EMC), IEC 60601-1(Safety) standards. And the manufacturer is ISO 13485 certified.

Guidance and manufacturer's declaration – electromagnetic emissions		
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment .
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning:

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used.

12. Electromagnetic Compatibility (EMC)

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 2 kV, 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump requires continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery.

12. Electromagnetic Compatibility (EMC)

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50 or 60 Hz	30 A/m, 50 Hz / 60 Hz	It may be necessary to position the breast pump farther from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.
---------------------------------------------------------	---------------------	-----------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

NOTE : UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz – 80 MHz z 6 V in ISM and amateur radio bands between 0.15 MHz z and 80 MHz 80% AM at 1 kHz	3 V 0.15 MHz – 80 MHz z 6 V in ISM and amateur radio bands between 0.15 MHz z and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
	Radiated RF IEC 61000-4-3	10 V/m (minimum) 80 MHz – 2.7 GHz 80% AM at 1 kHz	

12. Electromagnetic Compatibility (EMC)

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio. (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30cm) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810						
870	800 to 960	GSM 800/900, TETRA 800, IDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
930						
1720						
1845	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1970						
2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

EMC tests have included the AC adapter.

Model	209DA2
Power Source	AC/DC Adaptor: Input: 100 ~ 240V, 50/60Hz Output: 9V 1.3A
	Non-replaceable Li-Ion battery; 7.4V 2200mAh
Unit Dimensions	129 x 114 x 88 mm
Unit Weight	432 g
Suction	Approx. -70 ~ -250 mmHg
Vacuum Levels	9 levels
Operating Environment	Temperature range: 41 ~ 104°F(5 ~ 40°C) Humidity range: 15% ~ 90%
Transport/Storage Environment	Temperature range: -4 ~ 122°F(-20 ~ 50°C) Humidity range: 15% ~ 90%
Expected Service Life	Your device has been designed and developed with the greatest possible care and has an expected service life of 2000 hours.
Ingress Protection Level	IP22
Classification	Internal powered equipment, Type BF
Protection Class	Class II

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