Rumble Tuff®

EZ Express

Electric Breast Pump Duo

Instructions & Assembly

Model: 200DP

English 1-36
Español 37-73

Congratulations on your new Rumble Tuff® EZExpress Electric Breast Pump Duo! The Rumble Tuff® Electric Breast Pump Duo is perfect for many occasions.

- Collecting breast milk while you're away from your baby
 - for example: when you're at work
- Relieving engorgement
 - for example: when your baby's feeding habits change because they're eating more solids
- Maintaining your milk supply if you have to temporarily stop breastfeeding
 - for example: to express unsuitable milk while you are undergoing medical treatment

The Rumble Tuff® EZExpress Electric Breast Pump Duo provides many of the features busy moms love.

- Milk backflow protection
- Quiet, small and portable
- 4 speed modes and 9 levels of suction.
- Pause anytime
- 10 second delayed start
- Automatically power off
- Memory of last session time
- Convert to Manual Pump in seconds (parts included)
- 2 power options: AC/DC Adaptor & 4 AA Batteries

Indication for Use

The Rumble Tuff®EZExpress Electric Breast Pump Duo is a personal use, electrically powered device used to express milk from the breasts of lactating women. This device is not intended for hospital use.

Contraindications for Use

There are no known contraindications for use with this product.

Table of Contents

1. Introduction	1
2. Important Safeguards	3
3. Product Description	6
4. Start-up Procedure and Operation	9
4.1 Cleaning	9
4.2 Assembly	13
4.3 Operating EZExpress	17
4.4 Power options	19
5. Special Features	20
6. Assemble the Manual Pump Diaphragm Kit	21
7. Breast Milk Storage and Thawing	24
8. Troubleshooting	25
9. Warranty	29
10. Legend and Symbols	30
11. Electromagnetic Compatibility (EMC)	31
12. Technical Data	36

2. Important Safeguards

The Rumble Tuff®EZExpress Electric Breast Pump is designed for lactating women to express and collect breast milk. It is intended for a single user and should never be shared or resold; doing so could pose health risks and possibly void the warranty.

Please read this document completely and follow the instructions exactly. You should always keep it within reach. To prevent personal injury and material damage, please observe the warning and safety information provided in this document.

 DANGER	Can lead to serious injury or death
 WARNING	Can lead to serious injury or death
 CAUTION	Can lead to minor injury or material damage
Note	Useful breastfeeding and/or product information

DANGER:

To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING:

- Anti-Backflow is the essential performance of Rumble Tuff Breast Pumps. If any moisture/milk is found in the tubing while using or after using the pump, stop using the pump and contact Rumble Tuff Customer Service immediately. Toll-Free: (855) 228-8388
- Use only the AC/DC Adapter that comes with the EZExpress Pro Breast Pump
- Never leave plugged-in product unattended.
- Plug the AC/DC Adapter into the electrical outlet, before connecting the Power Cord into your pump.
- Make sure the voltage of the AC/DC Adapter is compatible with the power source.

2. Important Safeguards

- Do not operate this product if the cord or plug is damaged, if it is not working properly, if it has been dropped or damaged, or if it has been dropped in water.
- Keep the cord away from heated surfaces.
- Keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- Do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or breast pump parts.
- Never put the pump unit into water or sterilizer as it can cause permanent damage to the pump.
- If you are infected with Hepatitis A, Hepatitis B, or have HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- Do not attempt to modify the product.
- Do not pump while driving.
- Using a breast pump on an aircraft is not recommended. The cabin pressure may affect the breast pumps performance.
- If there is any allergic reaction, stop using this product and consult your physician immediately.

CAUTION:

- Do not use this product for any other purpose than as indicated in this manual.
- Make sure all of the parts are assembled correctly before operating.
- Stop using the breast pump immediately if your breasts or nipples experience pain or discomfort.
- Please wash hands before using this product.
- Do not drop or hit the product. Doing so may cause damage not covered by the warranty.
- To avoid scalds and burns, do not touch the Expression Collection Kits with bare hands immediately after sterilization by boiling water.
- To prevent premature aging, do not overboil product materials.

2. Important Safeguards


- Do not use any accessories, parts, or attachments that are not recommended by Rumble Tuff. Doing so may affect the performance of your breast pump and void the warranty.
- Do not connect the pump or any of its accessories to any equipment not recommended by Rumble Tuff.
- Do not store the product in a dusty, or damp environment, or in direct sunlight.
- If irritation or discomfort occurs, discontinue use and consult your physician immediately.

Note:


When disposing of the pump or its accessories, please consult with your local authorities for the proper method of disposal of potentially biohazardous parts and accessories, or to learn if there are any other regulations.

SAVE THESE INSTRUCTIONS
along with your receipt for future reference.


3. Product Description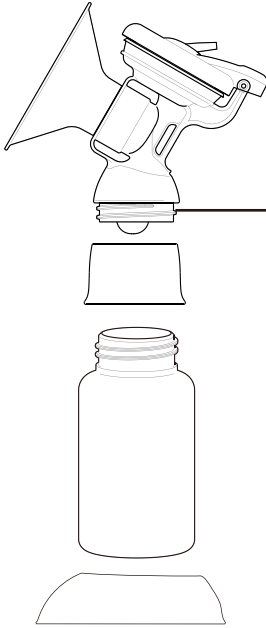

2x HF Breast Shields (25 mm)
For other breast shield sizes,
please visit www.rumbletuff.com/breastshields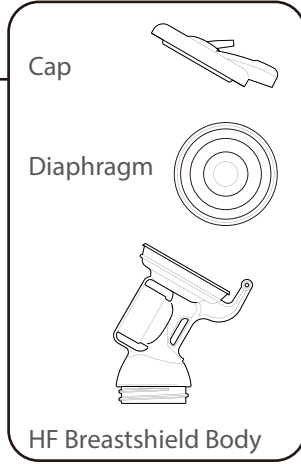


2x Bottles

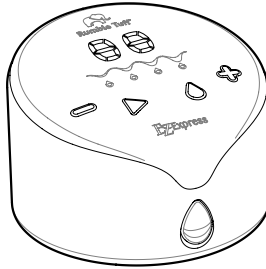1x Pump Unit

connects to Breast Shields


1x Tubing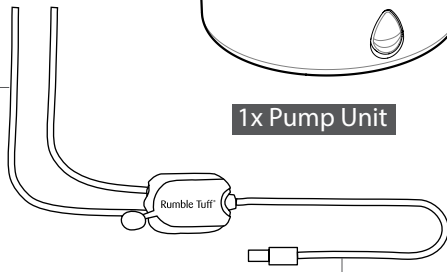

connects to Pump Unit


3. Product Description

Note: For ordering information about accessories, please visit www.rumbletuff.com/accessories; or contact our customer service. Toll-Free: (855) 228-8388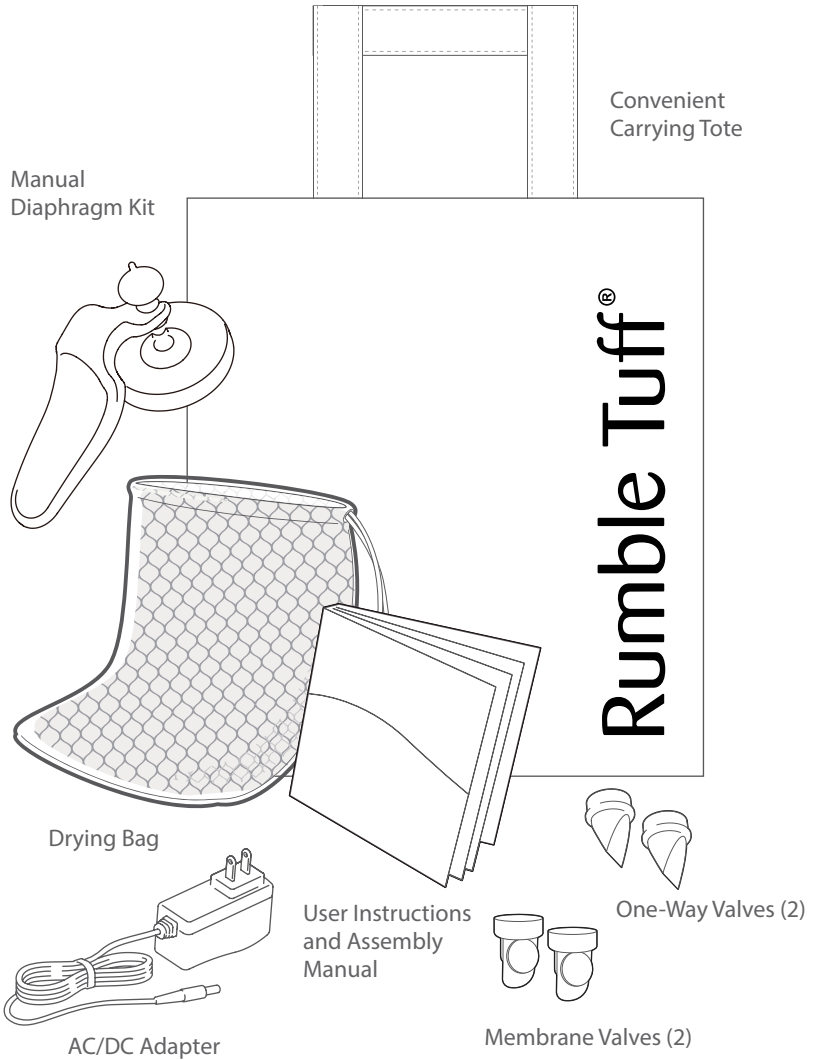


3. Product Description

LED Indicator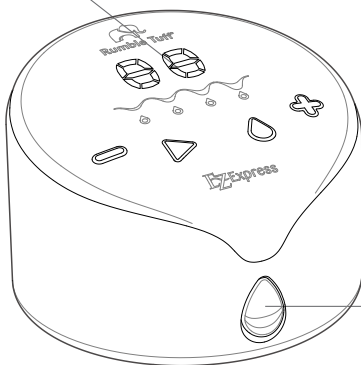

Vacuum Inlet with Dust Proof Cover

On/Off Button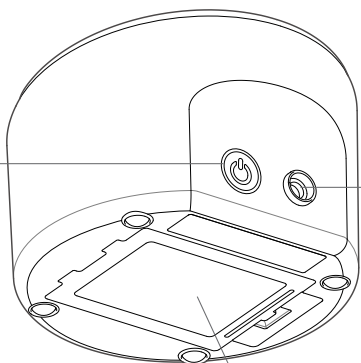

AC/DC Adapter Inlet

Battery Cover

4. Start-up Procedure and Operation

4.1 Cleaning

Sanitize Prior to First Use:

Before you use your Rumble Tuff®EZExpress Electric Breast Pump for the first time, disassemble and boil all parts of the Expression Collection Combo Kit in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Remove all parts from the water immediately after sterilizing. To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- Completely air-dry.

Wet components may affect the performance of your Electric Breast Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water for at least 20 seconds.
2. **Assemble.** Assemble the clean Express Collection Combo kit parts. Inspect each component for any cracks, tears, discoloration, or deterioration. If damage is found, please discontinue use and replace the part immediately. Inspect whether the kit parts or tubing have become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.
3. **Clean if using a shared pump.** Wipe pump dials, Power Button, and pump housing with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.


Special Cleaning Instruction for the Membrane Valve:

For your convenience in cleaning, the Membrane Valve is specially designed with a node on the stem of the Membrane.

To clean the Membrane, gently pull the Membrane away from the Valve until it snaps slightly on the stem but does not completely detach from it.

Clean the Membrane Valve in warm, soapy water, then rinse thoroughly in clean, warm water or warm running water.

After cleaning, push the stem back into place and make sure the Membrane lays flat against the Valve opening.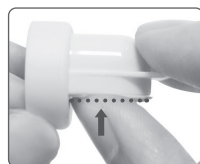

4. Start-up Procedure and Operation

After Every Use:

1. **Store milk safely.** Cap the milk collection bottle or seal the milk storage bag, label with date and time, and immediately place in a refrigerator, freezer, or cooler bag with ice packs.
If the milk collection container will be stored at a hospital or childcare facility, it's recommended to add your name to the label.
2. **Clean pumping area.** Especially if using a shared pump, wipe the dials, power switch, and countertop with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %) is recommended.
3. **Take apart and inspect the Collection Kits.** Remove the breast pump tubing and set it aside. Separate all the parts that come in contact with breast/breast milk (for example: Silicone Cushions, Flanges, Membrane Valves, Diaphragm and Milk Collection Bottles).
4. **Rinse Collection Kits.** To remove any milk residue, rinse the Collection Kit parts that come into contact with breast/breast milk under running water
5. **Clean Collection Kits.** As soon as possible after pumping, clean Collection Kits that come into contact with breast/breast milk in one of the following ways.

Clean by hand

1. **Use a wash basin.** Place Collection Kit parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the pump.
2. **Add soap and water.** Fill wash basin with hot water and add soap.
3. **Scrub.** Scrub items using a clean brush that is used only to clean infant feeding items.
4. **Rinse.** Rinse by holding items under running water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. **Dry.** Allow to air-dry thoroughly. Place Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. Rumble Tuff also provides a Drying Mess Bag for your convenience to air dry all Collection Kit parts.

4. Start-up Procedure and Operation

Clean in a dishwasher

1. **Wash.** Place disassembled Collection Kits parts in dishwasher. Be sure to place small items into a closed-top basket so they don't end up in the dishwasher filter. If possible, run the dishwasher using hot water and a heated drying cycle (or sanitizing setting); this can help kill more germs.
 2. **Remove from dishwasher.** Wash your hands with soap and water before removing and storing cleaned items. If items are not completely dry, place items on a clean, unused dish towel or paper towel to air-dry thoroughly before storing or use the provided Drying Mess Bag to air dry all parts. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.
6. **Clean wash basin and bottle brush.** If you use a wash basin or bottle brush when cleaning your Collection Kits parts, consider washing them every few days, either in a dishwasher with hot water and a heated drying cycle, if they are dishwasher-safe, or by hand with soap and warm water. Rinse them well and allow them to air-dry after each use.
7. **Tubing**
- Wipe the outside of the Tubing with a clean, damp cloth. Avoid getting water or other fluids inside the Tubing.
 - If liquid is found inside the Tubing, rinse the Tubing with clean, hot water. Hang to AIR DRY COMPLETELY BEFORE USING. Do NOT insert any object into Tubing.

Store Safely Until Needed

To help prevent germs and mold from growing, allow the clean Collection Kit parts, bottle brushes, and wash basins to air-dry thoroughly before storing. Once completely dry, and to prevent contamination during storage, the items should be stored in a clean, protected area.

- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Collection Kit parts.
- **Store safely.** Place reassembled Collection Kit in a clean, protected area such as inside an unused, sealable food storage bag. Store wash basins and bottle brushes in a clean area.

4. Start-up Procedure and Operation


Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing is designed to not touch the pumped milk and does not need to be cleaned routinely.
- Keep a spare set of tubing on hand in case the set you are using gets soiled or damaged.
- At the end of a pumping session, if your tubing has water droplets in it, disconnect the tubing from the Collection Kits, check to see if the Diaphragm needs to be replaced.
- If your tubing has mold in it, throw it away immediately because it is difficult to clean properly. Replace it with a new set of tubing, and check to see if the Diaphragm needs to be replaced.
- If the outside of your tubing is soiled, wipe it with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %) is recommended.
- Do not insert any object into the Tubing.


4. Start-up Procedure and Operation


4.2 Assembly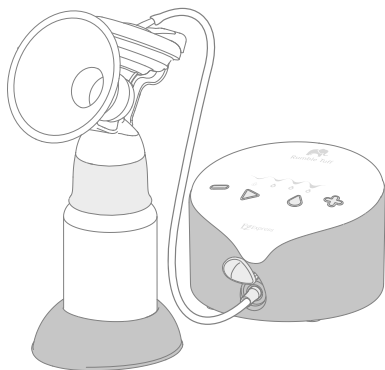


Verify that the membrane of the Membrane Valve is flat. A curled membrane will result in suction loss.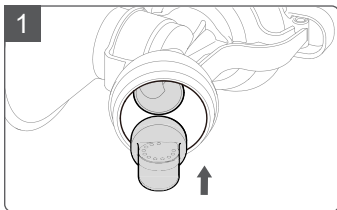

With the membrane facing towards you, firmly insert the Membrane Valve into the HF Breast Shield Body and position valve as shown in diagram 2.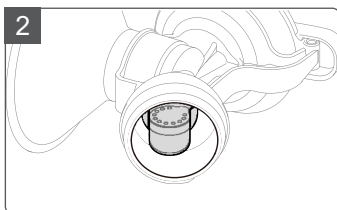


Confirm that the Membrane Valve is securely installed.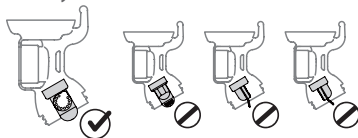


Make sure the Membrane Valve is inserted exactly as shown in the left diagram in order to maintain pumping efficiency.

4. Start-up Procedure and Operation



How to Assemble the Membrane Valve: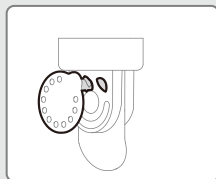


Hold the Silicone Membrane and insert the small stem into the top-right opening of the Valve.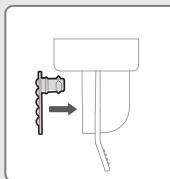

Press gently and firmly on the Membrane to ensure that the stem is completely inserted and is snug against the Valve opening.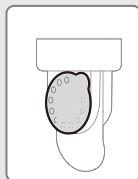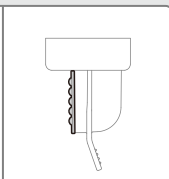


Membrane Valve Assembly completed.

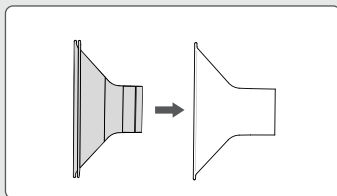If desired, place the Silicone Cushion into the Flange of the Breastshield.


Note: Your Electric Breast Pump will work with or without the Silicone Cushion. The usage of the cushion is entirely optional and based on your comfort and preference.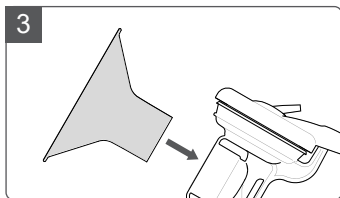

Insert the Flange into the Breastshield Body.

4. Start-up Procedure and Operation


4

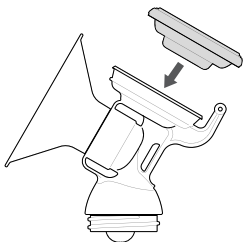Place the Diaphragm in the Cup of the Breastshield Body.


Note: Do not reverse the order of steps 4 through 5. Doing so will result in suction loss when pumping.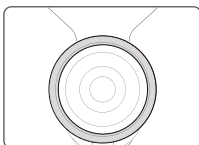


Press around the edges of the Diaphragm for a secure fit.

5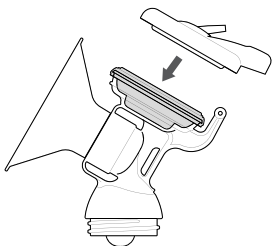


Place the Cap on the Cup and Diaphragm.

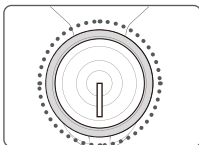Align the hinge cover over the hinge. Press tightly on the cap to seal.


6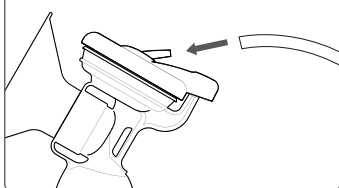


Connect the Tubing to the Tubing Connector located on the Cap.

Note: Make sure there is no moisture in the Tubing.

4. Start-up Procedure and Operation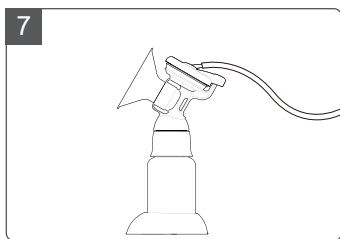


The Expression Collection Combo Kit is assembled.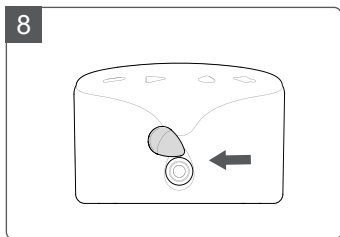


Open the Dust Proof Cover by pulling the cover out and turning it 90° to either side. Plug in the tubing adapter.

To Convert Your Double Pump Into a Single Pump:


Remove one tube and close the cap.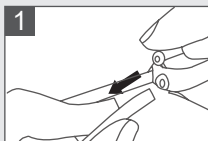

Disconnect the tube from the Tubing Splitter on the side with the little Cap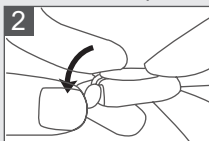

Cap the opening of the Tubing Splitter

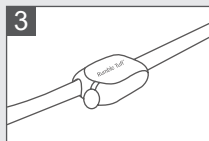
Ready to use as a single pump.

Note: Using the pump after converting from Single to Double or Double to Single may result in a temporary lack of suction or over-suction respectively. Please use caution and adjust the pump to your comfortable and effective level.

4. Start-up Procedure and Operation

4.3 Operating EZExpress

LED Display

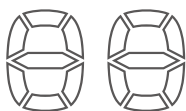
Multifunction Display

Mode Indicator

Depending on the state of the device, the multifunction display will show the following information:

- Session time in seconds (first minute of pumping), then in minutes.
- L1 to L9 for the vacuum level (along with the vacuum level display).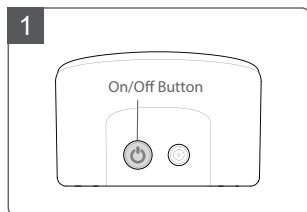

Press the On/Off Button  to begin pumping.

Note: The pump is now in Standby Mode.


Press the Play/Pause Button  to begin pumping. At the first 1~59 seconds, the LED Indicator shows the time by second; after the 59 seconds, The LED Indicator shows the time by minute.

4. Start-up Procedure and Operation


Depending on your preference and your baby's nursing habit, you can choose to express your milk in 4 different modes. To switch from one mode to the others, press the Mode Button.

**Note: Mode 1 & 2 – Stimulation;
Mode 3 & 4 – Expression.**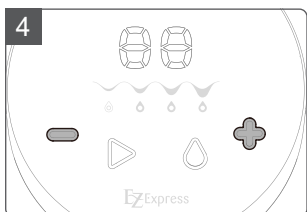


Adjust the vacuum level by pressing the Vacuum Adjusting Buttons; + for up, - for down. There are 9 suction levels for each Mode.

Note: (+) light will be dimmed on level 9, level indicator will flash twice if press (+) again; and (-) light will be dimmed on level 1, the level indicator will flash twice if press (-) again.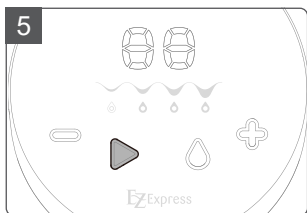

When your pumping session is over, press the Play/Pause Button ▶ to stop the pump.
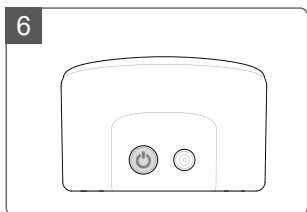


Then, press the On/Off Button ⏻ on the back to switch the pump off.

4. Start-up Procedure and Operation

4.4 Power options

Your Electric Breast Pump may be operated under any of the following power supply modes:

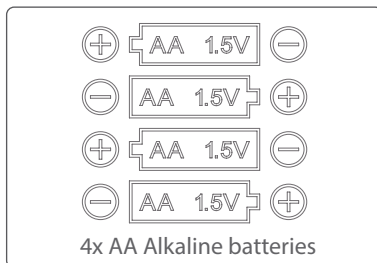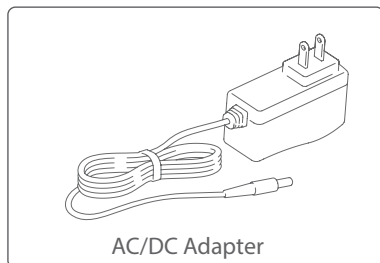Danger:

If the power adapters outer casing becomes loose or starts to separate, contact the device manufacturer immediately. Separation of the casing can result in electrical shock.

Caution:


Electrical safety can only be achieved when using the AC/DC adapter provided with the Pump. Use of any another Adapter poses a safety risk and/or the warranty may be voided.

Note:

Rumble Tuff suggests using rechargeable AA batteries which have a rated capacity of 2000 mAh or more.


5. Special Features

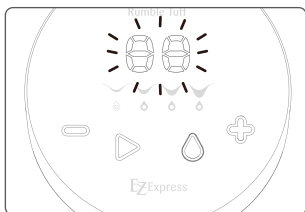Pausing


1. During the pumping process, you can press the Play/Pause Button  any time to pause, the timer will pause for two minutes and the LED Indicator will flash.
2. Press the Play/Pause Button  again to resume pumping.
3. If pumping is not resumed within two minutes, the breast pump will automatically switch off.

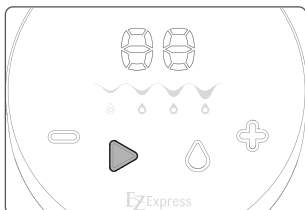Delayed Start

The pump features a 10 second delay start function, allowing user to adjust the Collection Kits. To activate the delay function, "press and hold" the Play/Pause Button  for 2 seconds in either the Standby or the Pause Mode. At the end of 10 seconds, the pump will start pumping automatically. To skip the delay, press the Play/Pause Button  once; the pump will start pumping immediately.



Auto-Off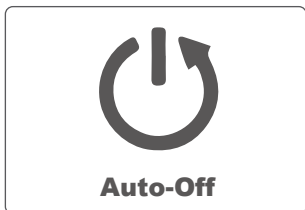


If there is no interaction with the pump, under the following conditions, it will shut off automatically:

1. On Standby Mode: 5 minutes.
2. On Pause Mode: 2 minutes.
3. Total pumping session reaches: 30 minutes.

6. Assemble the Manual Pump Diaphragm Kit

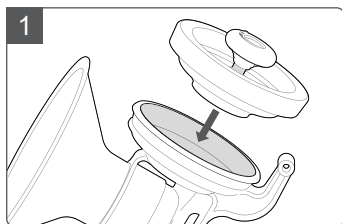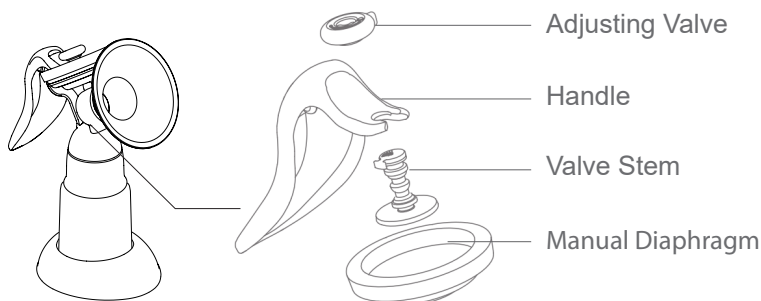
With the Cup in one hand, place the Manual Silicone Diaphragm in the Cup (matching the shape of the Cup).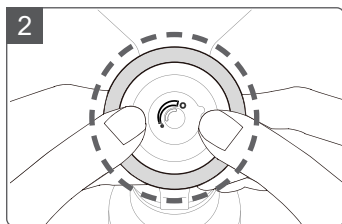


Press around the edges of the Diaphragm for a secure fit. Make sure the Diaphragm is evenly mounted on the Cup and there is no gap in between them.

6. Assemble the Manual Pump Diaphragm Kit

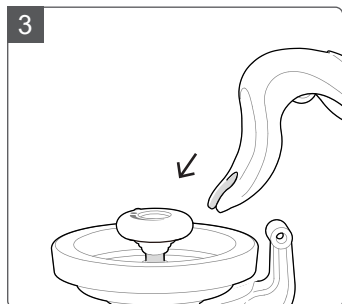
Push the Handle down onto the hinge of the Cup until it "snaps" into place.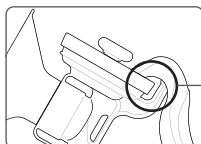

Hinge

Note: There are two recesses for attaching the handle to the Manual Pump. You can toggle between these two settings by pushing the Adjusting Valve Stem away from the "U" shape tip of the Handle to dislocate the connection. Reattach the tip of the Handle using a different recess on the Adjusting Valve Stem.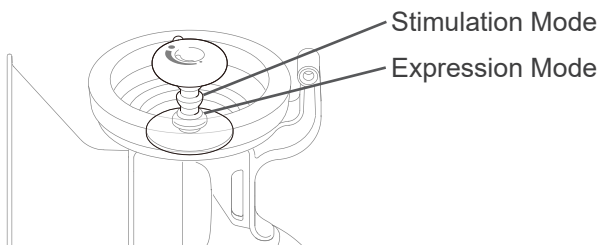


Stimulation Mode:

This setting is for fast, shallow suction. Often when a baby is hungry or has just "Latched-On," their nursing pattern is fast and shallow to start the mother's milk flow.

Expression Mode:

This setting is for longer, deeper suction. When the mother's milk begins to flow into the baby's mouth, they often naturally change to a longer deeper suction pattern.

6. Assemble the Manual Pump Diaphragm Kit

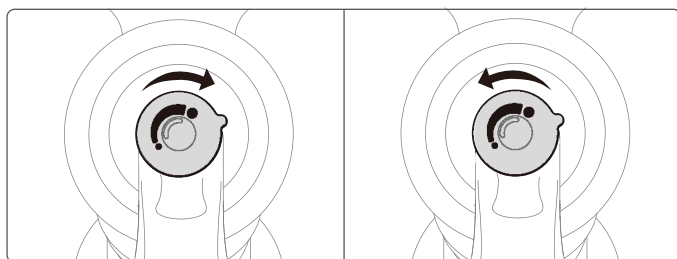Note: If an adjustment is necessary, turn the round knob of the Adjusting Valve to a comfortable suction level.

Caution:

Do not over-turn the Adjusting Valve, as it will cause the Adjusting Valve to bulge and leak air, creating a suction problem. Make sure the gap opening on top of the Adjusting Valve is lined up properly on the top of the Valve Stem so the Adjusting Valve turns smoothly. If necessary, re-adjust the Adjusting Valve by turning it back and forth a few times.

7. Breast Milk Storage and Thawing

Storage Location and Temperatures			
TYPE OF BREAST MILK	Countertop 77°F (25°C) or colder <small>(room temperature)</small>	Refrigerator 40°F (4°C)	Freezer 0°F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1-2 Hours	Up to 1 Day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding(baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

Source: CDC - Human Milk Storage Guidelines
(https://www.cac.gov/breastfeeding/recommendations/handling_breastmilk.htm)

Note:

1. *Always make sure to check the date on the milk's container before using. Do not use undated milk, or milk that has been stored too long.*
2. *Thaw frozen milk in the refrigerator or under cool, running water.*
3. *Heat the milk under warm, running water and gently swirl the container to mix the milk.*
4. *If warm water is not available, heat a pan of water on the stove. Once the water is warm, not boiling, remove the pan from the stove and place milk container in the pan. Never warm the milk container directly on the stove.*
5. *Never microwave breast milk because microwaves can cause dangerous hot spots that could burn you or your baby.*
6. *Always test the temperature of the milk on the inside of your wrist before feeding it to a baby. The milk should feel warm, not hot.*
7. *After thawing, milk should be stored in the refrigerator for no more than 24 hours.*
8. *Never refreeze thawed milk. Throw away previously frozen milk that is not used within 24 hours.*

Problem	Solution
No power	<ul style="list-style-type: none">• Make sure it's the correct power adaptor for Rumble Tuff model.• Try different wall outlet, if it's still not working, please call Rumble Tuff customer service at Toll-Free:(855)228-8388.
Interruption of electrical service	<p>During normal operation of your Rumble Tuff electric breast pump, if you experience an interruption of electrical service, due to a power outage or other factors, try the following:</p> <ul style="list-style-type: none">• Unplug the AC/DC Adapter from the electrical outlet and wait for 5 seconds.• Plug the AC/DC Adapter back into the electrical outlet.• If the problem persists, try a different electrical outlet.
Radio frequency energy	<p>This equipment generates, uses, and can radiate radio frequency energy. If not installed properly and used in accordance with the instructions, it may cause harmful interference with other devices in the vicinity. Try the following steps if interference occurs:</p> <ul style="list-style-type: none">• Reorient or relocate the device receiving interference.• Increase the distance between the equipment.• Connect the equipment to an outlet on a circuit different from that which the other device(s) are connected.• Consult the manufacturer or a field service technician of the device receiving interference for help.• Please understand that there is no guarantee that interference will not occur.
Low battery	<p>Power LED flashes when Battery power is lower than 4.5V; When power is lower than 4V, the pump will automatically turn off.</p>

8. Troubleshooting

No suction
or low suction

- Make sure the tubing is securely attached to the pump and the Collection Kits.
- If problem happens in one side of the collection bottle, try to switch the tubing to different vacuum inlet to see if the problem is now switched to the different collection bottle.
- If yes, the problem may be at the pump unit. Contact Technical Support for further assistance
- If not, follow the following steps to troubleshoot the problem in the Expression Collection Combo Kit.
- Make sure the Expression Collection Combo Kit is assembled correctly. (see pages 13 – 16)
- Center your nipple in the Flange and make sure there is a complete seal between the Flange and your skin Press the Diaphragm firmly into the Cup and make sure there is no gap. (see page 15 step 5)
- Check the Membrane Valve and make sure that it is properly sealed and securely seated. (see page 14)
- Inspect the Membrane and Valve very closely for tears or warping. This component is critical to achieve proper suction; even the tiniest tear will adversely affect pump performance. If damaged, replace the Membrane Valve with one of the spare valves provided with your Rumble Tuff EZExpress.

Breast milk
buildup in the
Breastshield Body

If breast milk doesn't appear to be draining fast enough into the Collection Bottle, reduce the suction setting on your Electric Breast Pump to allow the milk to drain before gradually increasing the suction to the original setting.

The breast pump
generates only
constant vacuum
instead of
sucking cycles

Contact the Rumble Tuff Customer Service.

The pump unit's
exterior got wet

Unplug the pump unit from the power source and turn it off. Dry off the exterior of the pump unit.

8. Troubleshooting

<p>The pump unit has been submerged in water</p>	<p>Unplug the pump unit from the power source and turn it off. Contact the Rumble Tuff Customer Service.</p>
<p>The breast pump is not responding as expected</p>	<p>Press the On/Off button to switch the breast pump off, then restart the breast pump. If there is no change, contact the Rumble Tuff Customer Service.</p>
<p>Where to get replacement parts</p>	<p>If you are missing a part or if a part was damaged, please contact Rumble Tuff Customer Service.</p> <ul style="list-style-type: none">• E-mail: customerservice@rumbletuff.com• Toll-Free: (855) 228-8388• Website: www.rumbletuff.com
<p>Where to purchase additional accessories</p>	<p>Additional accessories and replacements parts are available for purchase for use with your Rumble Tuff® EZExpress.</p> <ul style="list-style-type: none">• E-mail: sales@rumbletuff.com• Fax: 801-796-2688• Toll-Free: (855) 228-8388• Online Store: www.rumbletuff.com/store
<p>Condensation or milk in Tubing</p>	<p>If you find condensation in the tubing, wash and dry it immediately or replace the tubing. Rumble Tuff EZExpress Electric Breast Pump has anti-back-flow diaphragm on top of the Collection Kit to prevent milk over flow through the tubing into main breast pump unit. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connect. Instead contact Rumble Tuff Customer Service.</p> <ul style="list-style-type: none">• E-mail: sales@rumbletuff.com• Fax: 801-796-2688• Toll-Free:(855) 228-8388• Online Store: www.rumbletuff.com/store

8. Troubleshooting

- *If your breasts feel uncomfortable or if you are in pain while using the Pump, try decreasing the suction level by pressing the Vacuum Adjusting Button down. *
- *You may also try pumping with/without the Silicone Cushion.*
- *If these tips do not help, turn the pump off and remove the Collection Bottles from your breasts for a few minutes. If you are still experiencing pain or discomfort, consult your healthcare professional or an international board certified lactation consultant IBCLC.*
- *Your satisfaction is our highest priority. Please contact us if you experience any discomfort while using our pump.*

Position the device to ensure easy disconnection of the power cord from the outlet.

Limited Consumer Product Warranty (United States)

This Rumble Tuff® EZExpress Electric Breast Pump is warranted to the original consumer, the purchaser, to be free from defects in material and workmanship which are not commercially acceptable for a period of 120 days from the date of purchase and an extended two year warranty on the pump motor. This warranty applies to the original owner only. Warranty coverage terminates if you sell or otherwise transfer this product to another person. This warranty gives you specific legal rights and you may also have other rights, which vary by location.

RUMBLE TUFF MAKES NO EXPRESS WARRANTY OF ANY KIND REGARDING THIS PRODUCT OTHER THAN THOSE WARRANTIES SET FORTH HEREIN. ANY IMPLIED WARRANTY, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE TO THE EXTENT PERMITTED BY LAW, SHALL BE LIMITED IN DURATION TO A PERIOD OF 120 DAYS FROM THE DATE OF PURCHASE BY THE ORIGINAL PURCHASER.

In the event that this product is found by Rumble Tuff® to not meet the above limited warranty, as the purchaser's sole and exclusive remedy, Rumble Tuff® will repair, or, at the discretion of Rumble Tuff®, replace this product without charge for such replacement parts or labor. **The purchaser shall bear all expenses related to returning this product to Rumble Tuff®.** This warranty does not apply to any part of the product that has been subject to misuse, abuse, or alteration. Only Rumble Tuff® brand replacement parts are recommended for use with Rumble Tuff® breast feeding products. Improper or incorrectly performed maintenance or repair, or use of non-Rumble Tuff® brand replacement parts voids this warranty.

TO THE EXTENT PERMITTED BY LAW, Rumble Tuff® SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- The entire original Rumble Tuff® product
- The original receipt showing date of purchase
- Return Authorization Number (RA#) - Please contact Rumble Tuff® customer service by calling toll-free (855) 228-8388 for a RA#
- Detailed description of the problem

You, the purchaser, are responsible for all shipping charges.

Do NOT send product without a RA#.

To obtain an RA# and shipping address, please contact Rumble Tuff® Customer Service Department by:

- Toll-free: (855) 228-8388
- E-mail: customerservice@rumbletuff.com
- In writing:

Customer Service Department
Rumble Tuff
865 N 1430 W
Orem UT 84057

Items returned without a RA# will not be processed and will be returned to sender (postage due).

10. Legend and Symbols

The following tables explain the meaning of the symbols found on the product parts and its packaging.

Symbol	Description
	Manufacturer
	Serial number.
	Danger, Warning, Caution
Note	Notice or Tips
	Refer to instruction manual.
	Type BF applied part.
	CLASS II equipment.
	DC
	This product should not be mixed with other commercial waste for disposal.
	Power
	Consult instruction for use.
	CE mark
IP22	Degree of protection against harmful ingress of water or particulate matter.

11. Electromagnetic Compatibility (EMC)

This appliance conforms to the following standards:

IEC 60601-1-11 Medical electrical equipment – Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of IEC 60601-1-2(EMC), IEC 60601-1(Safety) standards. And the manufacturer is ISO 13485 certified.

Guidance and manufacturer's declaration – electromagnetic emissions

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .

Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment .
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning:

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used.

11. Electromagnetic Compatibility (EMC)

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 2 kV, 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT; 1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT; 1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump requires continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery.

11. Electromagnetic Compatibility (EMC)

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50 or 60 Hz	30 A/m, 50 Hz / 60 Hz	It may be necessary to position the breast pump farther from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.
---	---------------------	-----------------------	--

NOTE : UT is the a.c. mains voltage prior to application of the test level.

11. Electromagnetic Compatibility (EMC)

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	10 V/m (minimum) 80 MHz – 2.7 GHz 80% AM at 1 kHz	10 V/m (minimum) 80 MHz – 2.7 GHz 80% AM at 1 kHz	

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

11. Electromagnetic Compatibility (EMC)

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30cm) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720						
1845	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1970						
2450						
5240	5100 to 5800	Bluetooth, WLAN, 802.11 b/g/n , RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5500						
5785						
		WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9

EMC tests have included the AC adapter.

12. Technical Data

Model	200DP
Power Source	AC/DC Adaptor: Input: 100 ~ 240V, 50/60Hz Output: 12V 1A
	4 AA Alkaline Batteries
Unit Dimensions	116 x 116 x 77 mm
Unit Weight	290 g
Suction	Approx. -70 ~ -250 mmHg
Vacuum Levels	9 levels
Operating Environment	Temperature range: 41 ~ 104°F(5 ~ 40°C) Humidity range: 15% ~ 90%
Transport/Storage Environment	Temperature range: -4 ~ 122°F(-20 ~ 50°C) Humidity range: 15% ~ 90%
Expected Service Life	Your device has been designed and developed with the greatest possible care and has an expected service life of 2000 hours.
Ingress Protection Level	IP22
Classification	Internal powered equipment, Type BF
Protection Class	Class II

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