
# Printing Instructions

## Desktop Printing Instructions

- 1) Click the  icon in the upper right of your screen.
- 2) Select the **More Settings** dropdown menu.
- 3) Select **Actual Size** under the **Scale (%)** section (make sure there is no border on the document).
- 4) Click **Print** in the bottom left corner.

## iPhone Printing Instructions

- 1) Click the  button.
- 2) Click **Print** in the menu.
- 3) Click **Layout**.
- 4) Click **Border**.
- 5) Select **Single Hairline** (make sure there is no border on the document).
- 6) Make sure you have a printer selected.
- 7) Click **Print** in the top right corner.

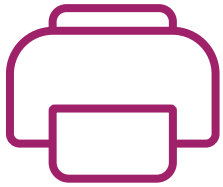# Printing Instructions

To ensure the ruler in this printable is the correct size for cutting out, make sure to follow the instructions below.

Set your printer to print on **LETTER SIZE** paper at **100% SCALE**. There should be no white border shown on the print preview.

To ensure accurate sizing,  
place a US penny here.


# Flange Size Guide

We at Rumble Tuff understand that finding the perfect flange size for your breast pump can be a bit overwhelming, but don't worry, we're here to help!

To ensure that you get the best experience and comfort while using your breast pump, it's important to measure your nipple correctly. Flanges are not "one size fits all" and having the right size makes a big difference.

## What is a Flange?

The flange is the part of the breast pump kit that fits over your nipple and areola. Before using your pump, you should measure your nipples to make sure you're using the flange size that works best for you. This guide will help you learn the correct way to measure your nipples for a perfect fit. See the flange size examples (**diagram 1**).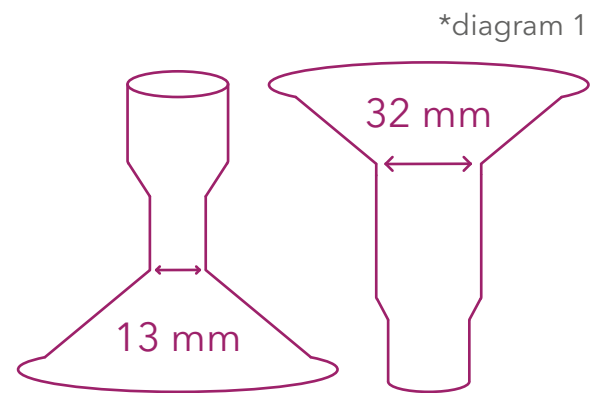

## What do I Measure?

Firstly, it's important to understand the anatomy of your breasts such as: the difference between a nipple and an areola. Please reference **diagram 3**.


There are 2 things to keep in mind when measuring to find your correct flange size.

- 1) When measuring your nipple size be sure to measure the base of your nipple (as seen in **diagram 3**).
- 2) Once you have found your nipple measurement. Be sure to add 1-3 millimeters (mm) to get the correct size flange to fit your nipple (see **diagram 2**).

### Sizing Examples

if nipple size is:	try flange size:
12 mm	15mm
14mm	17mm
16mm	19mm
19mm	21mm

\*diagram 2

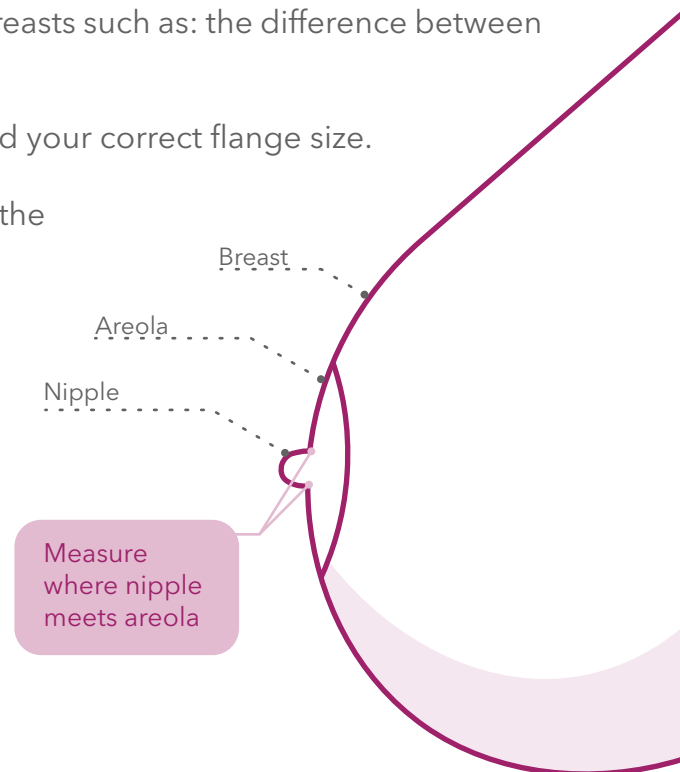\*diagram 3

# Step 1: Get Ready to Measure

In order to find the correct flange size you'll need to measure your nipples in millimeters. Below are 2 options you can use to help measure.

## Option 1



Cut out the Rumble Tuff milimeter ruler (**diagram 5**).

## Option 2

Find household items to help measure (**diagram 4**).

### Nipple Measurement Tips:

- use a mirror
- ask a friend/partner for help
- use a ruler (measure in mm)
- mark with paper then measure
- use the Rumble Tuff flange ruler included with each pump

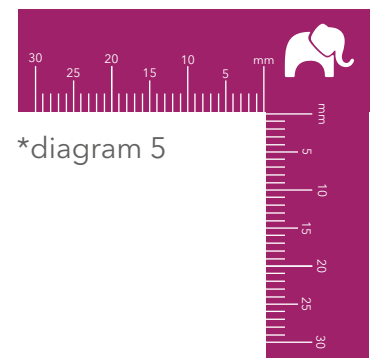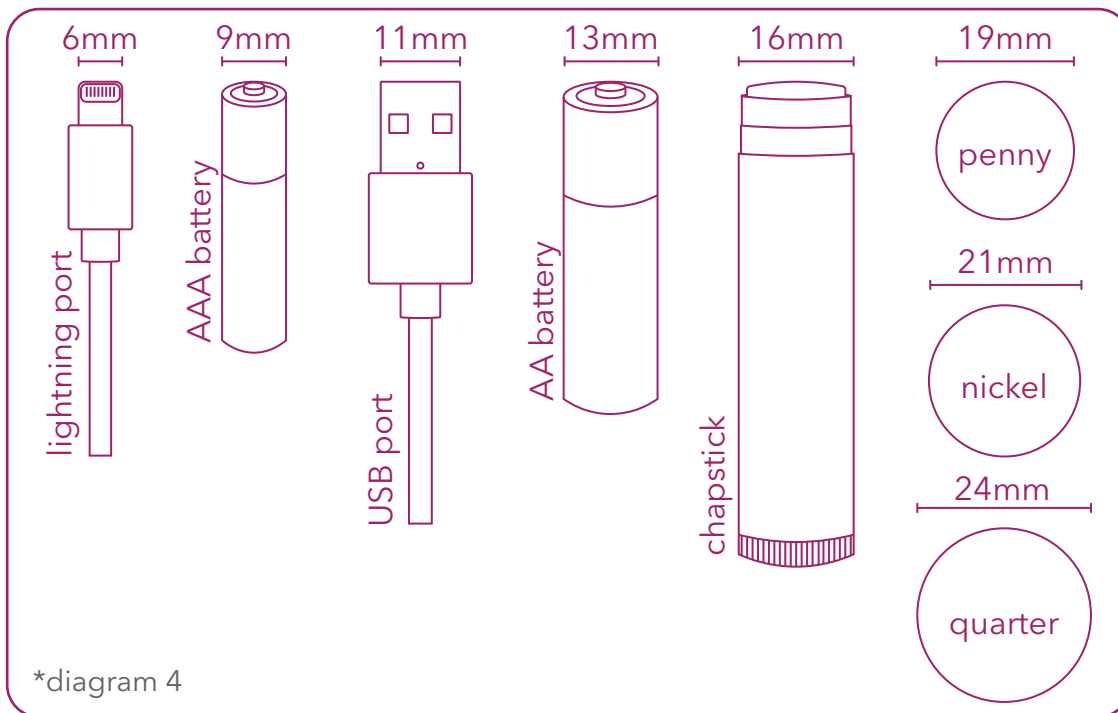# Step 2: Hygiene

Make sure to wash your hands and any items you might use before measuring.

# Step 3: Measure

Using either the Rumble Tuff Ruler (**diagram 5**) or any household items you might have (**diagram 4**) measure at the base of your nipple. Once you have the measurement in millimeters (mm) be sure to add 1-3 mm to ensure a comfortable fit (see **diagram 2** for examples).

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