

LED Indicator

LED指示燈

Decrease Suction Button

降低吸力調節鍵

Start/Pause Button

開始/暫停鍵

Increase Suction Button

增加吸力調節鍵

Mode Button

模式鍵

Vacuum Inlet

管接頭插入口



Front/正面



Open the Dust Cover to access the Vacuum Inlet

打開防塵蓋插入管接頭

Back/背面

On/Off Button

電源鍵

AC/DC Adapter Plug In

AC/DC適配器插孔



**⚠ Before using your pump for the first time, please read the User Manual for more detailed instructions.**

在您第一次擠乳前，請先閱讀產品說明書

## Wash 清潔

Before first use and after each use  
首次使用前及每次使用後



## Sanitize 消毒

Before first use and once per day  
首次使用前及一天一次



### Note:

- Do not wash or boil the tubing. If it becomes visibly dirty, wipe it clean with a warm, damp cloth.
- Before use, wash and sanitize all parts that may come in contact with the breast or breast milk.
- For more detailed use and care instructions, please refer to the User Manual.
- For best results, pump parts should be completely dry before use.

### 注意事項：

- 請勿清洗或煮沸消毒矽膠套管，若有明顯髒污請以溫濕布擦拭。
- 使用前請清洗與消毒所有會接觸乳房和乳汁的部件
- 更多相關詳細資訊請參閱產品說明書
- 為求最佳效果，使用前請確保所有部件都已完全乾燥狀態


• Scan code to access Bottle Assembly/Inspection Video

• 請掃描QR Code觀看瓶組組裝說明影片



## Expression Collection Combo Kit

集乳瓶組配件

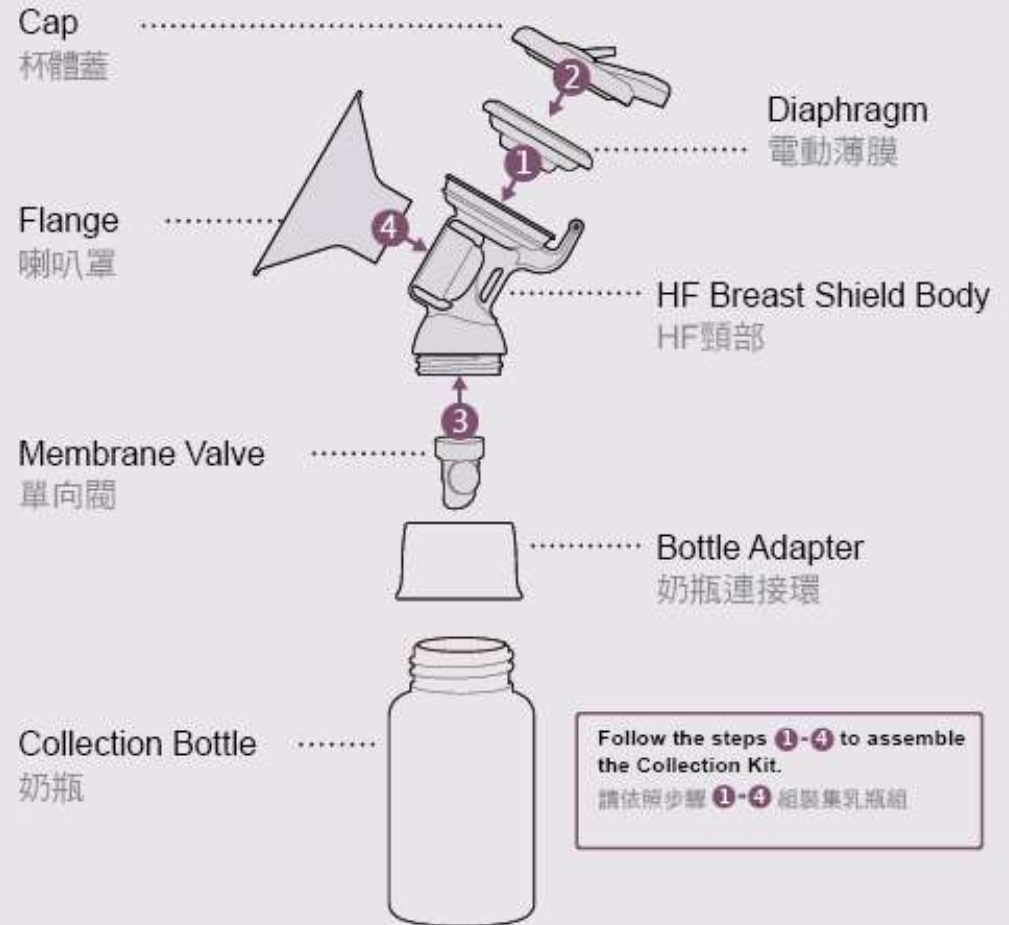

※ Please refer to user manual page 13~16 for detailed information.  
詳細資訊請參閱產品說明書第13~16頁