



# The Cleaning and Disinfecting Instruction of Rumble Tuff Multi-User Electric Breast Pump

The following cleaning instruction is for the Hospital, Rental Agency or lactation professional who rents out the Rumble Tuff Multi-User Electric Breast Pump.



## Notice:

The Rumble Tuff Multi-User Electric Breast Pump is designed and intended for multiple users provided it is being used by one user at a time and the Collection Bottle Kits are NOT shared. The pump console must be thoroughly cleaned and disinfected before it is being distributed to the next user.

The Rumble Tuff breast pumps are designed with an Anti-backflow Diaphragm above the Collection Bottle. Under normal usage, milk will not flow through the Tubing to the Pump Console. If milk is reportedly pumped into the Pump Console, remove the pump from the circulation and contact Rumble Tuff Customer Service for replacement.

## Inspection Instruction:

1. Inspect the Pump Console and the Power Cord for dirt or milk residue.
2. Inspect the Vacuum Inlets for milk residue inside the inlet. If residue is found inside, please report to Rumble Tuff Customer Service. Quarantine the pump and do not put the pump into circulation. Do not attempt to disassemble the Pump Console or clean the inside. Doing so may increase the risk of cross contamination and void the warranty.

## Cleaning:

1. Use a clean, damp cloth (use basic washing liquid or non-abrasive detergent if needed)
2. Wipe the Pump Console housing, Power Cord, and Case to clean off any milk residue or dirt.

## Disinfecting:

1. Use an alcohol-based disinfectant disposable wipe, CaviCide (recommended) or Isopropyl Alcohol (70% concentration or above).
2. Wipe the Pump Console housing, Power Cord, and Case.

## Storing:

Make sure the Pump Console and Power Cord are completely dry before storing them inside the Case.

Rumble Tuff Customer Service:

Phone: 1-866-228-8388

E-Mail: [customerservice@rumbletuff.com](mailto:customerservice@rumbletuff.com)