Rumble Tuff®

EZ-GO

Electric Breast Pump

Instructions & Assembly

Model: 200DP-C

English 1-35
Español 36-70

Congratulations on your new Rumble Tuff EZ Express Electric Breast Pump (Go-Cups Edition)! The Rumble Tuff Electric Breast Pump is perfect for various occasions.

- Collecting breast milk while you're away from your baby.
for example: when you're at work.
- Relieving engorgement.
for example: when your baby's feeding habits change because they're eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding.
for example: to express unsuitable milk while you are undergoing medical treatment.

The Rumble Tuff EZ Express Electric Breast Pump Duo& Go-Cups provides many of the features that busy moms love.

- Quiet, small and portable
- 4 speed modes and 9 levels of suction
- Pause anytime
- 10 second delayed start
- Automatically power off
- Memory of last session time
- 2 power options: AC/DC Adaptor & 4 AA Batteries
- Pump on the go
- Only 4 parts for easy assembly and cleaning
- Tubing can be inserted at any angle
- Easy pour

Indication for Use

The Rumble Tuff EZ Express Electric Breast Pump is a personal use, electrically powered device used to express milk from the breasts of lactating women. This device is not intended for hospital use.

Table of Contents

1. Introduction	1
2. Important Safeguards	3
3. Product Description	6
4. Start-up Procedure and Operation	
4.1 Cleaning	8
4.2 Assembly	12
4.3 Placement In Bra	14
4.4 Correct Breast Shield Fit	15
4.5 Positions For Expressing	16
4.6 Disconnecting Collection Cups And Milk Transfer.....	16
4.7 Cup Test And Trim Tubing	18
4.8 Operating EZ Express	19
4.9 Power options	21
5. Special Features	22
6. Breast Milk Storage and Thawing	23
7. Troubleshooting	24
8. Warranty	28
9. Legend and Symbols	29
10. Electromagnetic Compatibility (EMC)	30
11. Technical Data	35

2. Important Safeguards

The Rumble Tuff EZ Express Electric Breast Pump is designed for lactating women to express and collect breast milk. It is intended for a single user and should never be shared or resold; doing so could pose health risks and possibly void the warranty.

Please read this document completely and follow the instructions exactly. You should always keep it within reach. To prevent personal injury and material damage, please observe the warning and safety information provided in this document.

 DANGER	Can lead to serious injury or death
 WARNING	Can lead to serious injury or death
 CAUTION	Can lead to minor injury or material damage
<i>Note</i>	Useful breastfeeding and/or product information

DANGER:

To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING:

- Use only the AC/DC Adapter that comes with the EZ Express Breast Pump.
- Never leave plugged-in product unattended.
- Plug the AC/DC Adapter into the electrical outlet, before connecting the Power Cord into your pump.
- Make sure the voltage of the AC/DC Adapter is compatible with the power source.

- Do not operate this product if the cord or plug is damaged, not working properly, dropped, damaged, or immersed in water.
- Keep the cord away from heated surfaces.
- Keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- Do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or parts.
- Never submerge the pump unit into water or sterilizer as it can cause permanent damage to the pump.
- If you have Hepatitis A, Hepatitis B, or have HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- Do not attempt to modify the product.
- Do not pump while driving.
- Using a breast pump on an aircraft is not recommended. The cabin pressure may affect the breast pump's performance.
- If there is any potential allergic reaction, stop using our products and seek medical advice immediately.
- DO NOT boil tubing or adapters.
- If you experience pain or have lactation-related questions while using this device, seek professional advice from a lactation consultant to ensure you are using Rumble Tuff Go-Cups properly.
- Anti-Backflow is essential to the performance of Rumble Tuff Breast Pumps.
- If any moisture/milk is found in the tubing while using or after using the pump, stop using the pump and contact Rumble Tuff Customer Service immediately. Toll-Free: (855) 228-8388

CAUTION:

- Do not use this product for any other purpose than as indicated in this manual.
- Make sure all of the parts are assembled correctly before operation .
- If irritation or discomfort occurs, stop using the breast pump immediately and consult a doctor.
- Ensure all parts are clean and dry before use.
- Exposure to harsh detergents may cause silicone to appear cloudy over time.
- Replace damaged or worn parts immediately.
- Replace valves every 1-2 months or if loss of suction is detected. Two spare valves are included.
- Do not drop or hit the product. Doing so may cause damage not covered by the warranty.
- To avoid scalds and burns, do not touch the Go-Cups with bare hands immediately after sterilization by boiling water.
- To prevent premature aging, do not over boil product materials.
- Do not use any accessories, parts, or attachments that are not recommended by Rumble Tuff. Doing so may affect the performance of your breast pump and void the warranty.
- Do not connect the pump or any of its accessories to any equipment not recommended by Rumble Tuff.
- Do not store the product in a dusty, or damp environment, or in direct sunlight.

Note:


When disposing of the pump or its accessories, please consult with your local authorities for the proper method of disposal of potentially biohazardous parts and accessories, or to learn if there are any other regulations.

SAVE THESE INSTRUCTIONS
along with your receipt for future reference.

3. Product Description


1x Pump Unit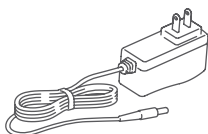


AC/DC Adapter

2X Diaphragm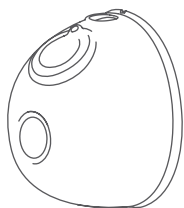

2x Cup

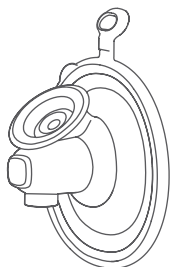
2X Breast Shield / Flange


2x insert

4X Silicone Lip Valve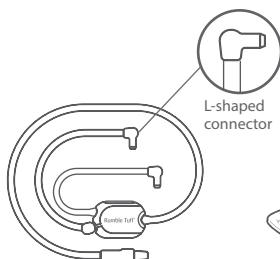


1x Tubing

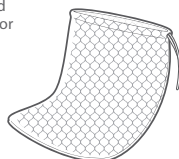Drying Bag


Note: For ordering information about accessories, please visit www.rumbletuff.com/accessories; or contact our customer service. Toll-Free: (855) 228-8388

LED Indicator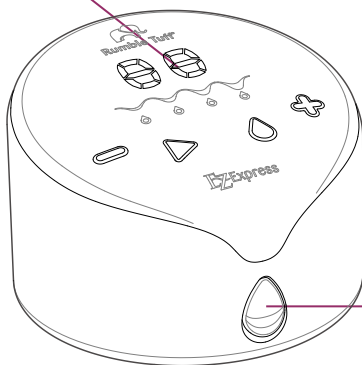

Vacuum Inlet
with Dust
Proof Cover

On/Off Button

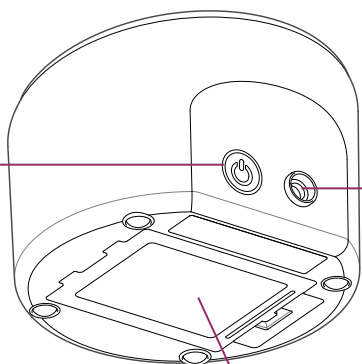AC/DC
Adapter Inlet

Battery Cover

4. Start-up Procedure and Operation

4.1 Cleaning

Sanitize Prior to First Use:

Before first use, disassemble and boil all parts of the Go-Cups in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Use tongs to remove all parts from water immediately after sterilizing. To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- Completely air-dry. Wet components may affect the performance of your Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water.
2. **Assemble.** Assemble the clean Go-Cups parts . Inspect each component for any cracks, tears, discoloration, or deterioration. If damage is found, please discontinue use and replace the part immediately. Inspect the Go-Cups parts or tubing to see if they have become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.


After Every Use:

1. **Store milk safely.** Store in refrigerator or freezer immediately.
2. **Disassemble and inspect Go-Cups.** Remove the tubing and set it aside. Separate all the parts that come into contact with breast/breast milk .
3. **Rinse Go-Cups.** To remove any milk residue, rinse the Go-Cups parts that come into contact with breast/breast milk under running water.

4. **Clean Go-Cups.** Go-Cups are hand washable or dishwasher safe:

- **Rinse and wash by hand (before first use and after each use)**

1. **Use a wash basin.** Place Go-Cups parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the Go-Cup parts.
2. **Add soap and water.** Fill wash basin with hot water and add soap.
3. **Scrub.** Scrub items using a clean brush that is used only to clean infant feeding items. Do not insert anything into the Lip Valves.
4. **Rinse.** Rinse by holding items under running water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. **Dry.** Allow to air-dry thoroughly. Place Go-Cups parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. Rumble Tuff also provides a Drying Mesh Bag for your convenience.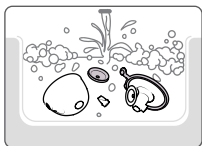

- **Wash in dishwasher (maximum once per day)**

1. Place the disassembled parts, except for the tubing, on the top rack or in the cutlery section.
2. Allow to completely dry after washing.

5. **Sanitize Go-Cups once per day after use.** Sanitize the parts according to P.8

Note:

If using a dishwasher, parts may become discolored. This will not impact the functioning of the parts.

6. Tubing

- Wipe the outside of the Tubing with a clean, damp cloth. Avoid getting water or other fluids inside the Tubing.
- If liquid is found inside the Tubing, rinse the Tubing with clean, hot water. Hang to AIR DRY COMPLETELY BEFORE USING. Do NOT insert any object into Tubing.


7. Lip Valve

- Rub gently between fingers in warm soapy water.
- Rinse thoroughly.

CAUTION:

Do not use cleaning brush inside valves as this may damage the valve and shorten its life span.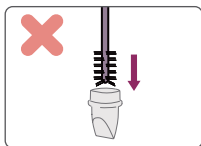

- With thumb and forefinger, gently seal any openings on the tip of the Lip valve and shape it into a straight line.

Note:

Inspect lip valves for tears or warping. Properly functioning valves are critical to performance. If damaged or unable to close, replace lip valves.

Store Safely Until Needed

Store pump and all parts in a safe dry area.


- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Go-Cups parts.
- **Store safely.** Place reassembled Go-Cups in a clean, protected area.

Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing is designed not to touch the pumped milk and does not need to be cleaned routinely.
- If the tubing has water droplets in it, disconnect it from the Go-Cups, and inspect the diaphragm for damage.
- Discard the Tubing if any mold is represent. Replace immediately.
- If the outside of your tubing is soiled, wipe it with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %).
- Do not insert any object into the Tubing.


4.2 Assembly

1-1
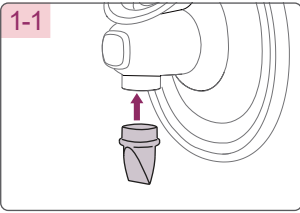


Firmly insert one-way valve into bottom of breast shield.
Place diaphragm on top of breast shield.

1-2

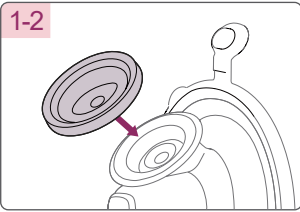
Press around edge of diaphragm completely for secure fit.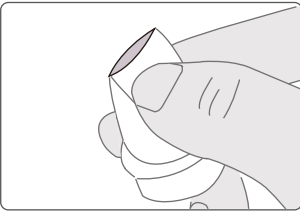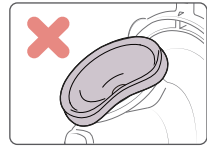



Note:

Verify the Lip Valve split is functioning normally by rubbing with fingers before pumping. This prevents milk from backing up.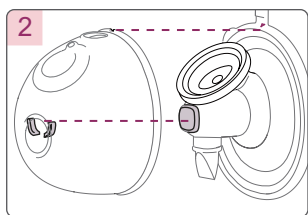


Confirm Breast Shield base is properly aligned with guides on cup.

Make sure the small triangular marks on top of them are lined up with each other.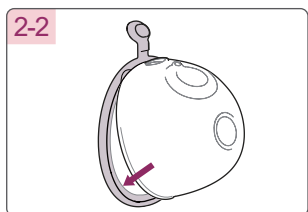
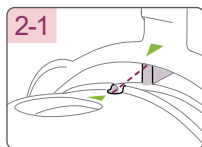


Put the Cup and the Breast Shield together

Press down the breast shield into the cup.

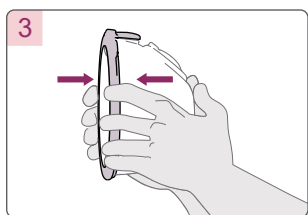
Press around edges of breast shield to ensure soft trim is snug against cup.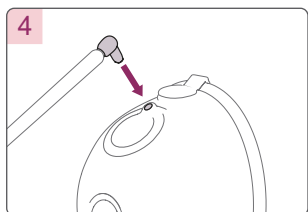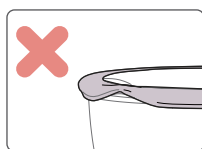


Insert smooth end of L-shaped connector into the opening directly above diaphragm.

Note:
Push L-connector down tightly for secure fit.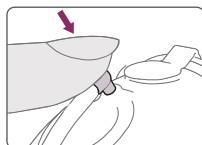


To Convert from Double Pump to a Single Pump:


Remove one tube and close the cap.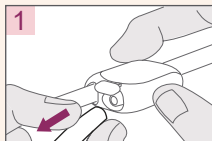

1
Disconnect the tube from the Tubing Splitter on the side with the little Cap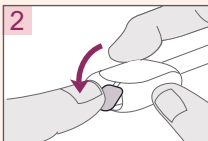


2
Cap the opening of the Tubing Splitter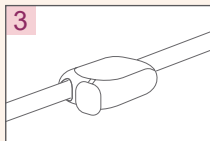




3
Ready to use as a single pump.

Note: Using the pump after converting from Single to Double or Double to Single may result in a changed of suction level. Proceed carefully.


4.3 Placement In Bra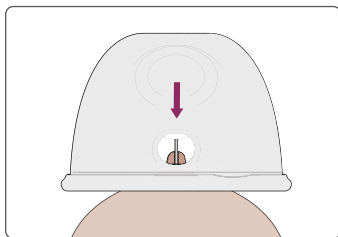

Center nipple in flange and make sure there is a complete seal between the flange and skin.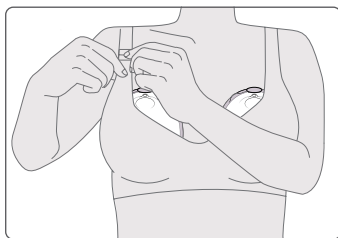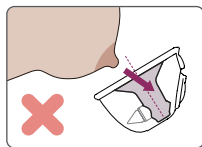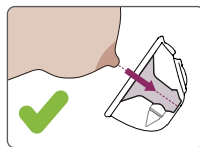

The Go-Cups can be worn under your regular nursing bra. Your bra will hold the collection cup in place. If you find it uncomfortable or painful during use, try switching to a looser, more stretchy nursing bra.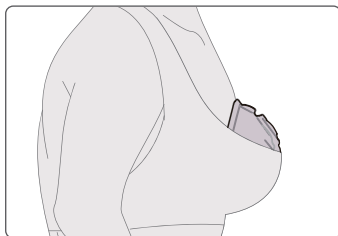


Close bra and ensure no slipping, sagging, or tilting.

Note:

Bra must not be too tight as it may cause a reduction in milk production. Insecure fit may cause loss of suction.

4.4 Correct Breast Shield Fit

- Measure nipples before pumping session.
- Please refer to rumbletuff.com/size-matters-quick-flange-breast-shield-guide/ for breast shield/flange size guidance.

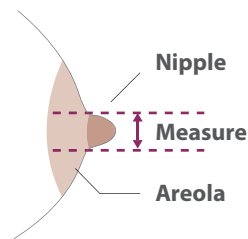The Go-Cups come with 24mm breast shield/ flange size. Optional 13mm, 15mm, 17mm, 19mm, and 21mm silicone inserts are available for purchase at rumbletuff.com.


too small **too large** **correct fit**

To allow for proper fit, breast shield / flange should be about 1 - 3mm larger than nipple diameter.


- Please refer rumbletuff.com/how-to-measure-your-nipples-for-the-proper-flange-fit-size/ options.

4.5 Positions For Expressing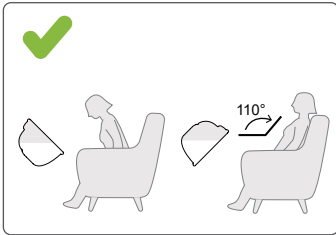


Good position

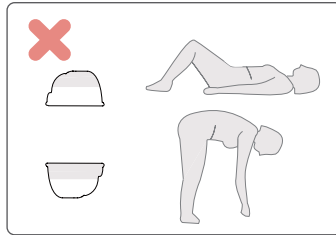
Poor position


4.6 Disconnecting Collection Cups And Milk Transfer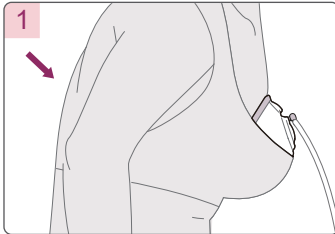



To avoid milk spilling out of the tunnel, keep the pump running & slightly lean forward to ensure all milk from the breast shield tunnel enters the collection cups.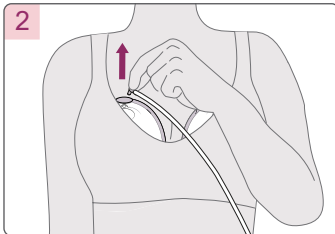


Unplug the L-shaped Connector to detach the Tubing from the Cup.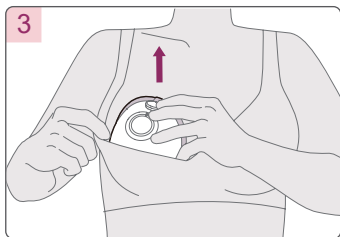

Carefully remove the Cup from the bra and place it on a firm and flat surface.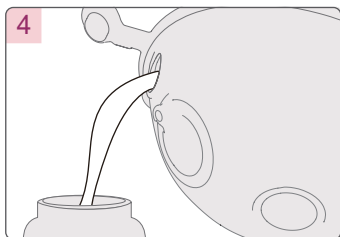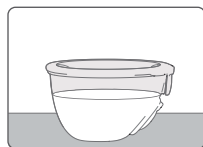

Open the spout cover and pour the breast milk into a Rumble Tuff Milk Bag or bottle.

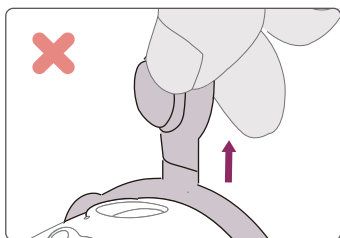


⚠ CAUTION:
Do not drag the spout cover, to prevent the spout cover from being damaged and failing.

4.7 Cup Test And Trim Tubing

Prior to placing Go-Cups in bra:


1. Connect and turn pump on.
2. Adjust suction to highest comfortable setting.
3. Observe diaphragm movement. The diaphragm will move up and down during operation when you have assembled the cups correctly.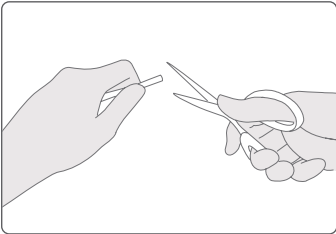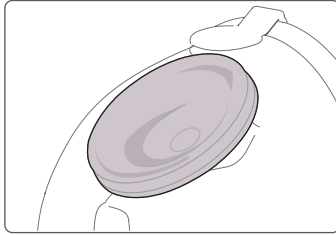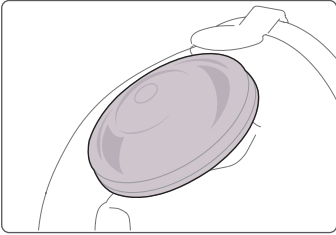

Tubing can be trimmed to desired length.

4.8 Operating EZ Express

LED Display

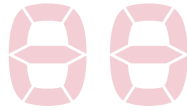
Multifunction Display

Mode Indicator


Depending on the state of the device, the multifunction display will show the following information:

- Session time in seconds (first minute of pumping), then in minutes.
- L1 to L9 for the vacuum level (along with the vacuum level display).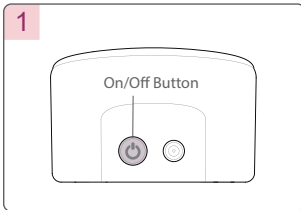


Press the On/Off Button  to begin pumping.

Note: The pump is now in Stand By Mode.

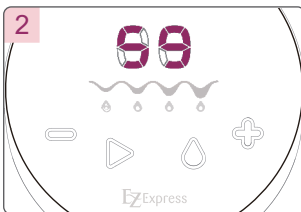
Press the Start/Pause Button  to begin pumping. At the first 1~59 seconds, the LED Indicator shows the time by second; after the 59 seconds, the LED Indicator shows the time by minute.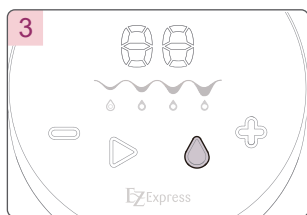


Depending on your preference and your baby's nursing habit, you can choose to express your milk in 4 different modes. To switch from one mode to the others, press the Mode Button.

**Note: Mode 1 & 2 – Stimulation;
Mode 3 & 4 – Expression.**

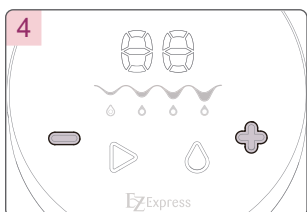
Adjust the vacuum level by pressing the Vacuum Adjusting Buttons; + for up, - for down. There are 9 suction levels for each Mode.

Note: (+) light will be dimmed on level 9, level indicator will flash twice if press (+) again; and (-) light will be dimmed on level 1, the level indicator will flash twice if press (-) again.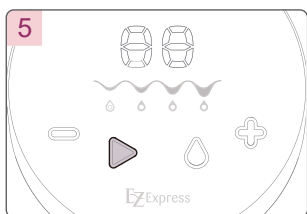


When your pumping session is over, press the Start/Pause Button ▶ to stop the pump.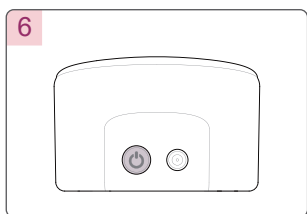


Then, press the On/Off Button  on the back to switch the pump off.

4.9 Power options

Your Electric Breast Pump may be operated under any of the following power supply modes: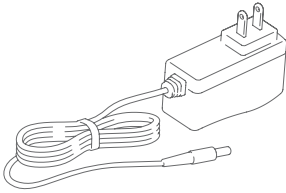

AC/DC Adapter

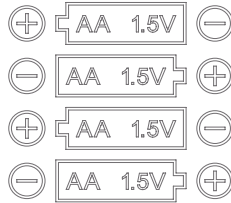4x AA Alkaline batteries

DANGER:

If the power adapters outer casing becomes loose or starts to separate, contact the Rumble Tuff customer service immediately. Separation of the casing can result in electrical shock.

CAUTION:


Electrical safety can only be achieved when using the AC/DC adapter provided with the Pump. Use of any another Adapter poses a safety risk and/or the warranty may be voided.

Note:

Rumble Tuff suggests using rechargeable AA batteries which have a rated capacity of 2000 mAh or more.


5. Special Features

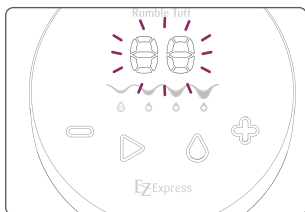Pausing

1. During the pumping process, you can press the Start/Pause Button  any time to pause, the timer will pause for two minutes and the LED Indicator will flash.
2. Press the Start/Pause Button  again to resume pumping.
3. If pumping is not resumed within two minutes, the breast pump will automatically switch off.

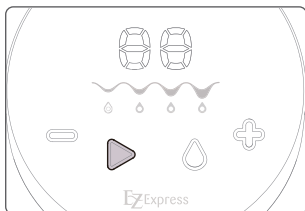Delayed Start

The pump features a 10 second delay start function, allowing user to adjust the Collection Kits. To activate the delay function, "press and hold" the Start/Pause Button  for 2 seconds in either the Standby or the Pause Mode. At the end of 10 seconds, the pump will start pumping automatically. To skip the delay, press the Start/Pause Button  once; the pump will start pumping immediately.

Auto-Off

Auto-Off

If there is no interaction with the pump, under the following conditions, it will shut off automatically:

1. On Standby Mode: 5 minutes.
2. On Pause Mode: 2 minutes.
3. Total pumping session reaches: 30 minutes.

6. Breast Milk Storage and Thawing

Storage Location and Temperatures			
TYPE OF BREAST MILK	Countertop 77°F (25°C) or colder <small>(room temperature)</small>	Refrigerator 40°F (4°C)	Freezer 0°F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1–2 Hours	Up to 1 Day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		

Source: CDC - Human Milk Storage Guidelines
(https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm)

Note:

1. Always make sure to check the date on the milk's container before using. Do not use undated milk, or milk that has been stored too long.
2. Thaw frozen milk in the refrigerator or under cool, running water.
3. Heat the milk under warm, running water and gently swirl the container to mix the milk.
4. If warm water is not available, heat a pan of water on the stove. Once the water is warm, not boiling, remove the pan from the stove and place milk container in the pan. Never warm the milk container directly on the stove.
5. Never microwave breast milk because microwaves can cause dangerous hot spots that could burn you or your baby.
6. Always test the temperature of the milk on the inside of your wrist before feeding it to a baby. The milk should feel warm, not hot.
7. After thawing, milk should be stored in the refrigerator for no more than 24 hours.
8. Never refreeze thawed milk. Throw away previously frozen milk that is not used within 24 hours.

7. Troubleshooting

Problem	Solution
No power	<ul style="list-style-type: none">• Make sure it's the correct power adaptor for Rumble Tuff model.• Try different wall outlet, if it's still not working, please call Rumble Tuff customer service at Toll-Free:(855)228-8388.
Interruption of electrical service	<p>During normal operation of your Rumble Tuff electric breast pump, if you experience an interruption of electrical service due to a power outage or other factors, try the following:</p> <ul style="list-style-type: none">• Unplug the AC/DC Adapter from the electrical outlet and wait for 5 seconds.• Plug the AC/DC Adapter back into the electrical outlet.• If the problem persists, try a different electrical outlet.
Radio frequency energy	<p>This equipment generates, uses, and can radiate radio frequency energy. If not installed properly and used in accordance with the instructions, it may cause harmful interference with other devices in the vicinity. Try the following steps if interference occurs:</p> <ul style="list-style-type: none">• Reorient or relocate the device receiving interference.• Increase the distance between the equipment.• Connect the equipment to an outlet on a circuit different from that which the other device(s) are connected.• Consult the manufacturer or a field service technician of the device receiving interference for help.• Please understand that there is no guarantee that interference will not occur.
Low battery	<p>Power LED flashes when Battery power is lower than 4.5V; When power is lower than 4V, the pump will automatically turn off.</p>

No suction
or low suction

- Make sure the tubing is securely attached to the pump and the Go-Cups.
- If problem happens in one side of the Go-Cup, try to switch the L-shaped connector to different vacuum inlet to see if the problem is now switched to the different Go-Cup.
- If yes, the problem may be at the pump unit. Contact Technical Support for further assistance
- If not, follow the following steps to troubleshoot the problem in the Go-Cups.
- Make sure the Go-Cups Kit is assembled correctly. (see pages 12 – 13)
- Center your nipple in the Flange and make sure there is a complete seal between the Flange and your skin.
- Press the Diaphragm firmly into the Cup and make sure there is no gap. (see page 18)
- Make sure the L-shaped Connector is securely fitted.
- Inspect the Lip Valve very closely for tears or warping. This component is critical to achieve proper suction; even the tiniest tear will adversely affect pump performance. If damaged, replace the Lip Valve with one of the spare valves provided with your Rumble Tuff EZ Express.

Breast milk
buildup in the
Breastshield

If breast milk doesn't appear to be draining fast enough into the Go-Cups, reduce the suction setting on your Electric Breast Pump to allow the milk to drain before gradually increasing the suction to the original setting.

The breast pump
generates only
constant vacuum
instead of
sucking rhythms

Contact the Rumble Tuff Customer Service.

The pump unit's
exterior got wet

Unplug the pump unit from the power source and turn it off. Dry off the exterior of the pump unit.

<p>The pump unit has been submerged in water</p>	<p>Unplug the pump unit from the power source and turn it off. Contact the Rumble Tuff Customer Service.</p>
<p>The breast pump is not responding as expected</p>	<p>Press the On/ Off button to switch the breast pump off, then restart the breast pump. If there is no change, contact the Rumble Tuff Customer Service.</p>
<p>Where to get replacement parts</p>	<p>If you are missing a part or if a part was damaged, please contact Rumble Tuff Customer Service.</p> <ul style="list-style-type: none"> • E-mail: customerservice@rumbletuff.com • Toll-Free: (855) 228-8388 • Website: rumbletuff.com/
<p>Where to purchase additional accessories</p>	<p>Additional accessories and replacements parts are available for purchase for use with your Rumble Tuff EZ Express.</p> <ul style="list-style-type: none"> • E-mail: sales@rumbletuff.com • Fax: 801-796-2688 • Toll-Free: (855) 228-8388 • Online Store: rumbletuff.com/shop/
<p>Condensation or milk in Tubing</p>	<p>If you find condensation in the tubing, wash and dry it immediately or replace the tubing. RumbleTuff EX Express Electric Breast Pump has anti-backflow diaphragm on top of the Go-Cups to prevent milk over flow through the tubing into main breast pump unit. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connect. Instead contact Rumble Tuff Customer Service.</p> <ul style="list-style-type: none"> • E-mail: customerservice@rumbletuff.com • Fax: 801-796-2688 • Toll-Free:(855) 228-8388 • Online Store: rumbletuff.com/shop/

- If your breasts feel uncomfortable or if you are in pain while using the Pump, try decreasing the suction level by pressing the Vacuum Adjusting Button down. 
- If these tips do not help, turn the pump off and remove the Go-Cups from your breasts for a few minutes. If you are still experiencing pain or discomfort, consult your healthcare professional or an international board certified lactation consultant IBCLC.
- Your satisfaction is our highest priority. Please contact us if you experience any discomfort while using our pump.

CAUTION:

Position the device to ensure easy disconnection of the power cord from the outlet.

8. Warranty

Limited Consumer Product Warranty (United States)

This Rumble Tuff EZ Express Electric Breast Pump is warranted to the original consumer, the purchaser, to be free from defects in material and workmanship which are not commercially acceptable for a period of 90 days from the date of purchase and an extended two year warranty on the pump motor. This warranty applies to the original owner only. Warranty coverage terminates if you sell or otherwise transfer this product to another person. This warranty gives you specific legal rights and you may also have other rights, which vary by location.

RUMBLE TUFF MAKES NO EXPRESS WARRANTY OF ANY KIND REGARDING THIS PRODUCT OTHER THAN THOSE WARRANTIES SET FORTH HEREIN. ANY IMPLIED WARRANTY, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR ANY IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE TO THE EXTENT PERMITTED BY LAW, SHALL BE LIMITED IN DURATION TO A PERIOD OF 30 DAYS FROM THE DATE OF PURCHASE BY THE ORIGINAL PURCHASER.

In the event that this product is found by Rumble Tuff to not meet the above limited warranty, as the purchaser's sole and exclusive remedy, Rumble Tuff will repair, or, at the discretion of Rumble Tuff, replace this product without charge for such replacement parts or labor. **The purchaser shall bear all expenses related to returning this product to Rumble Tuff.** This warranty does not apply to any part of the product that has been subject to misuse, abuse, or alteration. Only Rumble Tuff brand replacement parts are recommended for use with Rumble Tuff breast feeding products. Improper or incorrectly performed maintenance or repair, or use of non-Rumble Tuff brand replacement parts voids this warranty.

TO THE EXTENT PERMITTED BY LAW, Rumble Tuff SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- The entire original Rumble Tuff product.
- The original receipt showing date of purchase.
- Return Authorization Number (RA#) - Please contact Rumble Tuff customer service by calling toll-free (855) 228-8388 for a RA#
- Detailed description of the problem.

You, the purchaser, are responsible for all shipping charges.

Do NOT send product without a RA#.

To obtain an RA# and shipping address, please contact Rumble Tuff Customer Service Department by:

- Toll-free: (855) 228-8388
- E-mail: customerservice@rumbletuff.com
- In writing:
Customer Service Department
Rumble Tuff
865 N 1430 W
Orem UT 84057

Items returned without a RA# will not be processed and will be returned to sender (postage due).

9. Legend and Symbols

The following tables explain the meaning of the symbols found on the product parts and its packaging.

Symbol	Description
	Manufacturer
	Serial number
	Danger, Warning, Caution
Note	Notice or Tips
	Refer to instruction manual
	Type BF applied part
	CLASS II equipment
	DC
	This product should not be mixed with other commercial waste for disposal
	Power
	Consult instruction for use
	CE mark
IP22	Degree of protection against harmful ingress of water or particulate matter

10. Electromagnetic Compatibility (EMC)

This appliance conforms to the following standards:

IEC 60601-1-11 Medical electrical equipment – Part 1-11: General requirements for basic safety and essential performance – Collateral Standard: Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment and complies with the requirements of IEC 60601-1-2(EMC), IEC 60601-1(Safety) standards. And the manufacturer is ISO 13485 certified.

Guidance and manufacturer's declaration – electromagnetic emissions

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.

Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning:

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 2 kV, 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump requires continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery.

<p>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</p>	<p>30 A/m, 50 or 60 Hz</p>	<p>30 A/m, 50 Hz / 60 Hz</p>	<p>It may be necessary to position the breast pump farther from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.</p>
--	----------------------------	------------------------------	---

NOTE : UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30cm) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMR5 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n , RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

EMC tests have included the AC adapter.

11. Technical Data

Model	200DP
Power Source	AC/DC Adaptor: Input: 100 ~ 240V, 50/60Hz Output: 12V 1A
	4 AA Alkaline Batteries
Unit Dimensions	116 x 116 x 77 mm
Unit Weight	290 g
Suction	Approx. -70 ~ -250 mmHg
Vacuum Levels	9 levels
Operating Environment	Temperature range: 41 ~ 104°F(5 ~ 40°C) Humidity range: 15% ~ 90%
Transport/Storage Environment	Temperature range: -4 ~ 122°F(-20 ~ 50°C) Humidity range: 15% ~ 90%
Expected Service Life	Your device has been designed and developed with the greatest possible care and has an expected service life of 2000 hours.
Ingress Protection Level	IP22
Classification	Internal powered equipment, Type BF
Protection Class	Class II

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