Rumble Tuff®

Flo-1

new collection kit
Instruction for use

EN 1 - 13

ES 14 - 25

Model: PA935

Congratulations on the purchase of your new Rumble Tuff Flo-1. RumbleTuff Flo-1 provide many great features:

- NEW 2-in-1 diaphragm. Rotate to extend its life for up to three cycles before replacement.(Varies by user)
- Only Fewer parts for easy assembly and cleaning.
- Closed system.
- Allows access to your pump's motor strength.

Follow our YouTube channel for pumping tips and how-to videos

Table of Contents

1. Introduction	1
2. Important Information	3
3. Product Description	4
4. Start-up Procedure and Operation	
4.1 Cleaning	5
4.2 Assembly	8
4.3 Correct Breast Shield Fit	10
5. Breast Milk Storage and Thawing	11
6. Connecting to Breast Pump	12
7. Connecting to other Breast Pumps	13

2. Important Information




- DO NOT boil tubing or adapters. Contains small parts, which may be a choking hazard for children.
- Rumble Tuff Flo-1 are a personal care item and intended for use by only one person.
- This device is intended for use with any Rumble Tuff electric breast pump. Using it with another breast pump brand is solely at the user's discretion.
- If you experience pain or have lactation-related questions while using this device, seek professional advice from a lactation consultant to ensure you are using Rumble Tuff Flo-1 properly.
- Diaphragm is equipped with 3 valves. When one side loss of suction is detected or every 1-2 months , please use the other side. Two spare diaphragm are included.

As a Rumble Tuff user, you automatically qualify for free pumping consults from an IBCLC.

Schedule your appointment on-line:

<https://www.rumbletuff.com/lactation-consultants>

3. Product Description

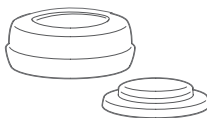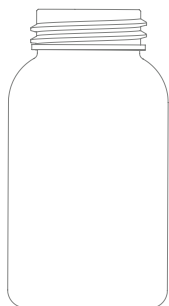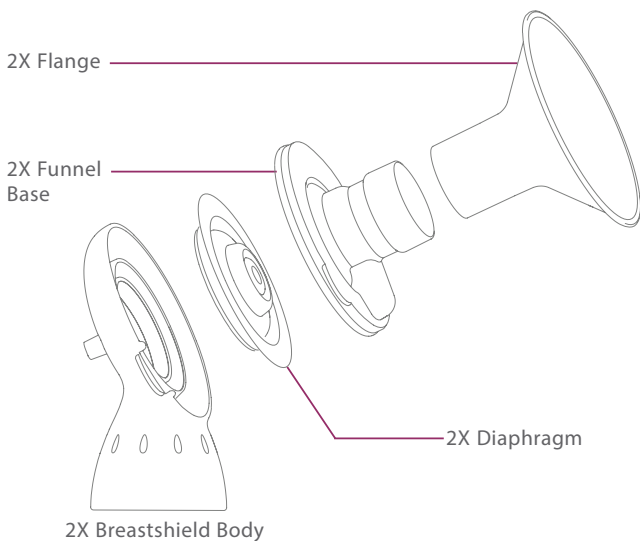Note: For ordering information about accessories, please visit www.rumbletuff.com/accessories; or contact our customer service. Toll-Free: (855) 228-8388

4. Start-up Procedure and Operation

4.1 Cleaning

Sanitize Prior to First Use:

Before first use, disassemble and boil all parts of the Flo-1 in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Use tongs to remove all parts from water immediately after sterilizing.
- To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- **Completely air-dry.**
Wet components may affect the performance of your Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water.
2. **Assemble.** Assemble the clean Flo-1 parts. Inspect each component for any cracks, tears, discoloration, or deterioration. If damage is found, please discontinue use and replace the part immediately. Inspect the Flo-1 parts or tubing to see if they have become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.


After Every Use:

1. **Store milk safely.** Store in refrigerator or freezer immediately.
2. **Disassemble and inspect Flo-1.** Remove the tubing and set it aside. Separate all the parts that come into contact with breast/breast milk.
3. **Rinse Flo-1.** To remove any milk residue, rinse the Flo-1 parts that come into contact with breast/breast milk under running water.

4. Clean Flo-1. Flo-1 are hand washable or dishwasher safe:

- **Rinse and wash by hand (before first use and after each use)**


1. **Use a wash basin.** Place Flo-1 parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the Flo-1 parts.
2. **Add soap and water.** Fill wash basin with hot water and add soap.
3. **Scrub.** Scrub items using a clean brush that is used only to clean infant feeding items.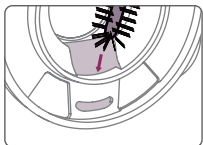

Note:

Scrub the tunnel with the brush to avoid milk residue

4. **Rinse.** Rinse by holding items under running water or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. **Dry.** Allow to air-dry thoroughly. Place Flo-1 parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. Rumble Tuff also provides a Drying Mesh Bag for your convenience.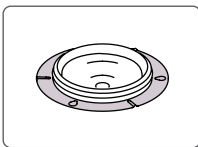

Caution:

When storing the Diaphragm, be careful not to squeeze or deform the Valve.

- **Wash in dishwasher (maximum once per day)**

1. Place the disassembled parts, except for the tubing, on the top rack or in the cutlery section.
2. Allow to completely dry after washing.

Note:

If using a dishwasher, parts may become discolored. This will not impact the functioning of the parts.

5. **Sanitize Flo-1 once per day after use.** Sanitize the parts according to P.5

6. **Tubing**

- Wipe the outside of the Tubing with a clean, damp cloth. Avoid getting water or other fluids inside the Tubing.
- If liquid is found inside the Tubing, rinse the Tubing with clean, hot water. Hang to **AIR DRY COMPLETELY BEFORE USING**. Do **NOT** insert any object into Tubing.

Store Safely Until Needed


Store pump and all parts in a safe dry area.

- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Flo-1 parts.
- **Store safely.** Place reassembled Flo-1 in a clean, protected area.




Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing is designed not to touch the pumped milk and does not need to be cleaned routinely.
- If the tubing has water droplets in it, disconnect it from the Flo-1, and inspect the diaphragm for damage.
- Discard the Tubing if any mold is represent. Replace immediately.
- If the outside of your tubing is soiled, wipe it with a low-level disinfectant such as Ethyl or Isopropyl Alcohol (70 – 90 %).
- Do not insert any object into the Tubing.


4.2 Assembly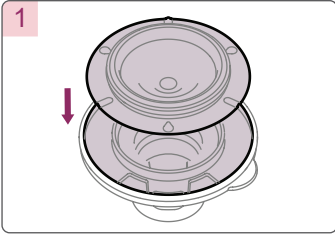

Align the waterdrop mark with the outlet. Place diaphragm on top of breast shield and press around edge completely for secure fit.

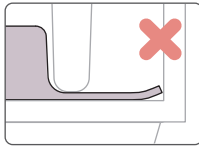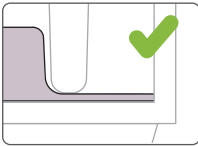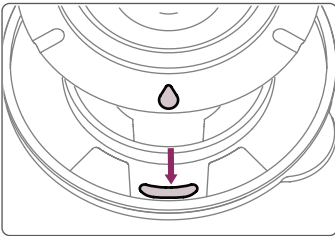
Note:
Ensure that the stem is snug against the Valve opening.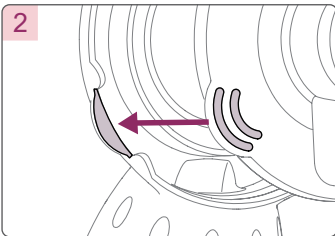


Confirm cover is properly aligned with Breastshield Body(funnel base).

Note:
Hold the tunnel to separate two parts with angle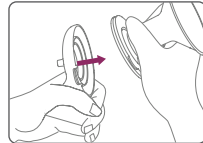


3

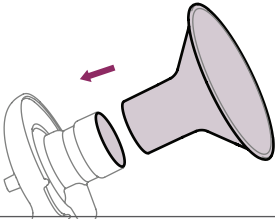Insert the Flange into the Breastshield Body (funnel base)kit.

4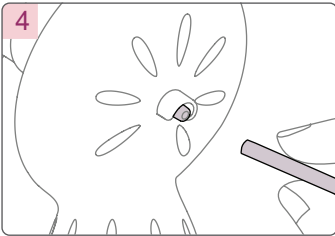


Connect the Tubing to the Tubing Connector located on the Breastshield Body(funnel base).

Note:

Make sure there is no moisture in the Tubing.

4.3 Correct Breast Shield Fit

- Measure nipples before pumping session.
- Please refer to rumbletuff.com/size-matters-quick-flange-breast-shield-guide/ for breast shield/flange size guidance.

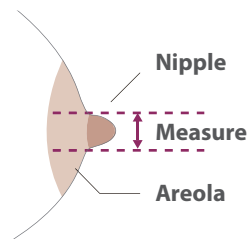The Flo-1 come with 25mm breast shield/ flange size. Optional 13mm~ 32mm breast shield/ flange are available for purchase at rumbletuff.com.

too small too large correct fit

To allow for proper fit, breast shield / flange should be about 1 - 3mm larger than nipple diameter.

- Please refer rumbletuff.com/how-to-measure-your-nipples-for-the-proper-flange-fit-size/ options.

5. Breast Milk Storage and Thawing

Storage Location and Temperatures			
TYPE OF BREAST MILK	Countertop 77°F (25°C) or colder <small>(room temperature)</small>	Refrigerator 40°F (4°C)	Freezer 0°F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1–2 Hours	Up to 1 Day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding (baby did not finish the bottle)	Use within 2 hours after the baby is finished feeding		




Source: CDC - Human Milk Storage Guidelines
(https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm)

Note:

1. Always make sure to check the date on the milk's container before using. Do not use undated milk, or milk that has been stored too long.
2. Thaw frozen milk in the refrigerator or under cool, running water.
3. Heat the milk under warm, running water and gently swirl the container to mix the milk.
4. If warm water is not available, heat a pan of water on the stove. Once the water is warm, not boiling, remove the pan from the stove and place milk container in the pan. Never warm the milk container directly on the stove.
5. Never microwave breast milk because microwaves can cause dangerous hot spots that could burn you or your baby.
6. Always test the temperature of the milk on the inside of your wrist before feeding it to a baby. The milk should feel warm, not hot.
7. After thawing, milk should be stored in the refrigerator for no more than 24 hours.
8. Never refreeze thawed milk. Throw away previously frozen milk that is not used within 24 hours.

6. Connecting to RT Breast Pump

- Insert ribbed end of connector into tubing.
- Insert smooth end of connector into the opening directly above diaphragm.

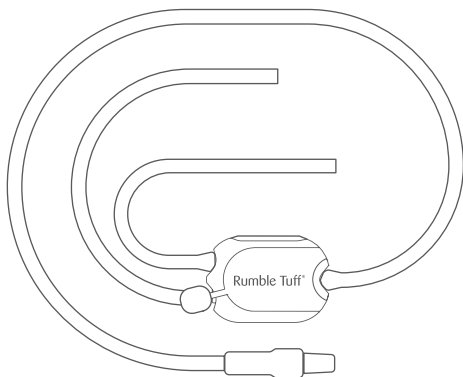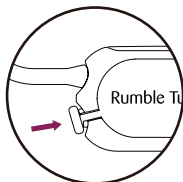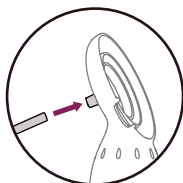
For single pumping


Rumble Tuff® Breast Pump

EZ Express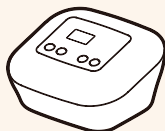

Whisper

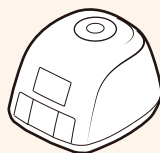Breeze

Note:

Desired suction level may vary from standard collection combo kits.

7. Connecting to other Breast Pumps

- Flo-1 tubing adaptor for different size tubing

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