Rumble Tuff®

Breeze PRO

Electric Breast Pump
Instructions & Assembly

Model: PA209DM

Congratulations on your new Rumble Tuff® Breeze Pro Electric Breast Pump (Multi-user)!

The Rumble Tuff® Electric Breast Pump Duo is perfect for many occasions.

- Collecting breast milk while you're away from your baby
 - for example: when you're at work
- Relieving engorgement
 - for example: when your baby's feeding habits change because they're eating more solids
- Maintaining your milk supply if you have to temporarily stop breastfeeding
 - for example: to express unsuitable milk while you are undergoing medical treatment

The Rumble Tuff® Electric Breast Pump Duo provides many of the features busy moms love.

- 3 cycles of expressions to mimic baby's nursing patterns.
 - Reflex Expression - Latch on & Rooting habit
 - Swift Expression - Fast but shallow swallowing habit
 - Natural Expression - Steady and strong swallowing pattern
- Milk back-flow protection
- Quiet motor for discretion
- Small and lightweight
- 2 power options to fit your needs
 - AC/DC Adapter (included)
 - Built-in Li-Ion Rechargeable Battery
- Convert to Manual Pump in seconds (parts included)

Indication for Use

This Rumble Tuff® Electric Breast Pump is intended to express and collect milk from the breasts of lactating women. This Rumble Tuff® Electric Breast Pump is intended for multiple users in a hospital setting.

It is also intended for home use by a single user.

Contraindications for use

There are no known contraindications for use with this product.

Table of Contents

Congratulations	1
Important Safeguards	4
Danger	4
Warning	4
Caution	5
Accessories and Parts	6
Included Parts and Accessories	6
Expression Collection Combo Kit	7
Bottle Assembly	8
How to Assemble the Expression Collection Combo Kit	8
Cleaning Instructions	13
Sanitize Prior to First Use	13
Before Each Use	13
After every use	14
For Extra Protection, Sanitize	16
Store Safely Until Needed	17
Tubing Cleaning Instructions	17
About Your Rumble Tuff Breeze Pro	18
Electric Breast Pump Main Unit	18
Expression Indicator Lights	18
How to Use Your Rumble Tuff Breeze Pro	19
Suction Level	21
M Button (My Favorite Pattern)	22
How to Adjust the Clock	23
How to Adjust the Alarm	24
How to mute chimes	25

Power Options	26
Using your AC/DC Adapter	26
Built-in Li-Ion Rechargeable Battery	27
Rumble Tuff Sweet Essence Assembly	28
How to Assemble the Manual-Pump Diaphragm Kit	28
How to Assemble the Expression Collection Combo Kit	29
Using your Rumble Tuff Sweet Essence	30
Expression Collection Combo Kit	30
Storage and Preparation of Breast Milk	32
Troubleshooting	34
Interruption of Electrical Service	34
Radio Frequency Energy	34
Decreased Suction	34
No response while operating Adjusting Knob and Mode button	35
Pump doesn't run when power is "on"	35
Breast Milk Buildup in the Breastshield Body	35
Where to Get Replacement Parts	35
Where to Purchase Additional Accessories	35
Condensation or Milk in Tubing	36
EMC/Technical Description	37
Warranty	43
Technical Data	44
Power Requirements	44
Technical Specifications	44
Symbols Legend	45

Important Safeguards

The Rumble Tuff® Breeze Pro Electric Breast Pump is intended to express milk and collect milk from the breasts of lactating women. The device may be used by more than one user if the proprietary Rumble Tuff Collection Kits are used and are not shared.

READ ALL INSTRUCTIONS BEFORE USING

DANGER:

To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not reach for an electrical product that has fallen into water. Unplug immediately.

WARNING:

- Multiple-user pumps are designed so that the breast milk can never touch the working parts of the pump that are shared. While the pump unit is intended for multi-user, the Collection Kits are for single-user only and should never be shared.
- Anti-Backflow is the essential performance of Rumble Tuff Breast Pumps. If any moisture/milk is found in the tubing while using or after using the pump, stop using the pump and contact Rumble Tuff Customer Service immediately.
- Use only the AC/DC Adapter that comes with the Breeze Pro Breast Pump
- Never leave plugged-in product unattended.
- Plug the AC/DC Adapter into the electrical outlet, before connecting the Power Cord into your pump.
- Make sure the voltage of the AC/DC Adapter is compatible with the power source.
- Do not operate this product if the cord or plug is damaged, if it is not working properly, if it has been dropped or damaged, or if it has been dropped in water.
- Keep the cord away from heated surfaces.
- Keep the pump out of reach of babies, children or pets. Small parts and components may pose a swallowing, choking or strangulation hazard.
- Do not use antibacterial or abrasive cleaner/detergent when cleaning breast pump or breast pump parts.
- Never put the pump unit into water or sterilizer as it can cause permanent damage to the pump.

Important Safeguards

- If you are infected with Hepatitis A, Hepatitis B, or have HIV (Human Immunodeficiency Virus), pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- Do not attempt to modify the product.
- Do not pump while driving.
- Using a breast pump on an aircraft is not recommended. The cabin pressure may affect the breast pumps performance.
- If there is any potential allergic reaction, stop using our products and find a medical advice immediately.

CAUTION:

- Do not use this product for any other purpose than as indicated in this manual.
- Make sure all of the parts are assembled correctly before operating.
- Stop using the breast pump immediately if your breasts or nipples experience pain or discomfort.
- Please wash hands before using this product.
- Do not drop or hit the product. Doing so may cause damage not covered by the warranty.
- Do not touch the Expression Collection Kits with bare hands immediately after sterilization by boiling water to avoid scalds and burns.
- Do not overboil product materials; to prevent premature aging.
- Do not use any accessories, parts, or attachments that are not recommended by Rumble Tuff. Doing so may affect the performance of your breast pump and void the warranty.
- Do not connect the pump or any of its accessories to any equipment not recommended by Rumble Tuff.
- Do not store the product in a dusty, or damp environment, or in direct sunlight.
- If irritation or discomfort occurs, discontinue use and see a doctor.

Note:


When disposing of the pump or its accessories, please consult with your local authorities for the proper method of disposal of potentially biohazardous parts and accessories, or to learn if there are any other regulations.

SAVE THESE INSTRUCTIONS


along with your receipt for future reference.

Accessories and Parts

Included Parts and Accessories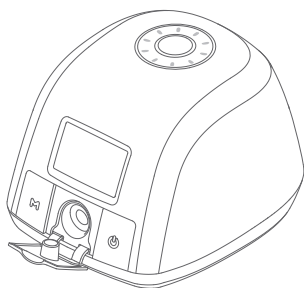


Breast Pump Main Unit

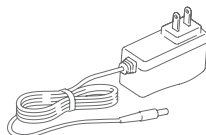
AC/DC Adapter

User Manual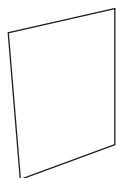

Assemble Instruction

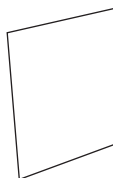
Cleaning Instruction

Warning:

Rumble Tuff Breeze Pro Electric Breast Pump is designed to be used with Rumble Tuff Expression Collection Combo Kits only. The Anti-Backflow Diaphragm on top of the Collection Kit prevents the breast milk over flow through the Tubing into the Pump Main Unit. This special feature makes the Pump suitable for multiple-users. Although the Pump Main Unit is for Multi-user, the Collection Kit is for SINGLE USER only and should never be shared. Sharing the Collection Kits will increase the risk of cross-contamination and void the warranty.

Expression Collection Combo Kit (Sold Separately)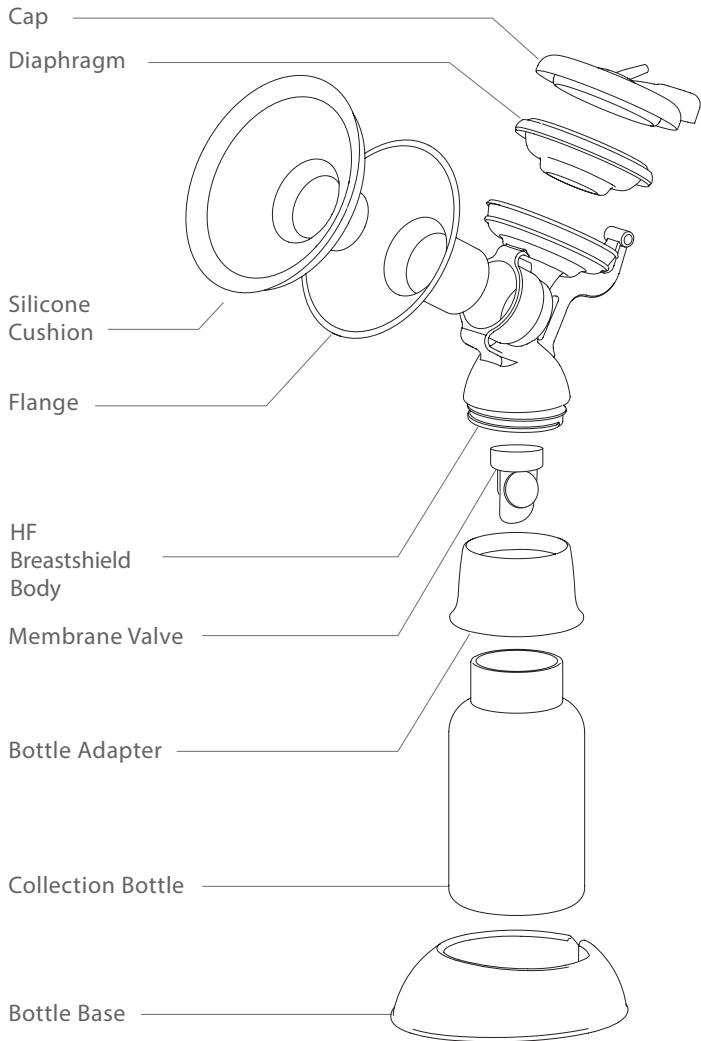

Bottle Assembly


How to Assemble the Expression Collection Combo Kit


Hold the HF Breastshield Body so you can look into the bottle thread opening.

With the membrane facing towards you, firmly insert the Membrane Valve into the HF Breastshield Body and position valve as shown in the black checked illustration seen below.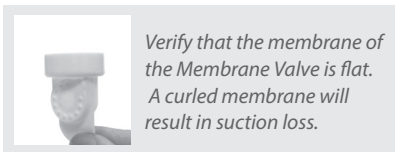

Confirm that the Membrane Valve is securely installed.

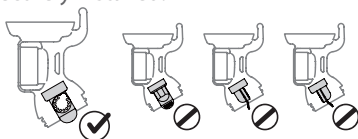
Make sure the Membrane Valve is inserted exactly as shown in the left diagram in order to maintain pumping efficiency.

How to Assemble the Membrane Valve:

Hold the Silicone Membrane and insert the small stem into the top-right opening of the Valve.

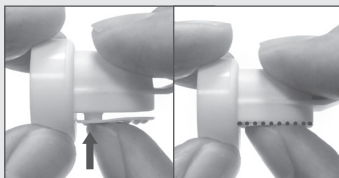
Press hard on the Membrane to ensure that the stem is completely inserted and is snug against the Valve opening.

Membrane Valve Assembly completed.

How to Assemble the Expression Collection Combo Kit (Continued)


4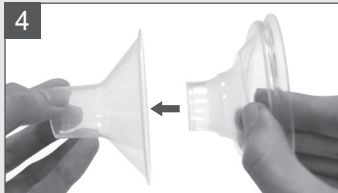

If desired, place the Silicone Cushion into the Flange of the Breastshield.

Note: Your Electric Breast Pump will work with or without the Silicone Cushion. Your comfort depends on your preference.

4-1

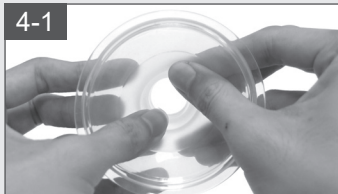Push the center of the Silicone Cushion into the Flange.

4-2

Secure the lip of the Cushion around the edge of the Flange.

4-3

Ensure that the lip of the Cushion is securely in place.

5

Insert the Flange into the HF Breastshield Body .

Bottle Assembly

How to Assemble the Expression Collection Combo Kit (Continued)

Place the Diaphragm in the Cup of the HF Breastshield Body .


Note: Do not reverse the order of steps 6 through 9. Doing so will result in suction loss when pumping.

Press around the edges of the Diaphragm for a secure fit.

Ensure that the Diaphragm is securely mounted in place. This is crucial in order to achieve proper suction and prevent milk back flow.

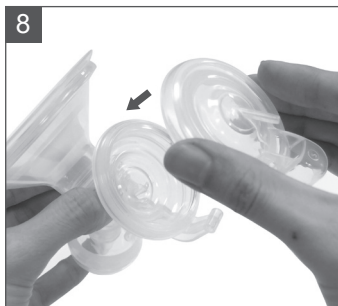Place the Cap on the Cup and Diaphragm.

Align the hinge cover over the hinge. Press tightly on the cap to seal.

The Electric Diaphragm Kit is assembled.


How to Assemble the Expression Collection Combo Kit (Continued)

Twist the Bottle Adapter onto the HF Breastshield Body .

Twist on the Collection Bottle for a secure fit.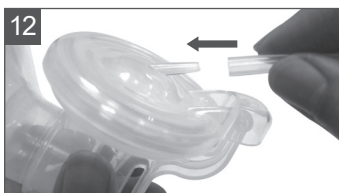

Connect the Tubing to the Tubing Connector located on the Cap.


Note: Make sure there is no moisture in the Tubing.

Bottle Assembly

How to Assemble the Expression Collection Combo Kit (Continued)

The Expression Collection Combo Kit is assembled.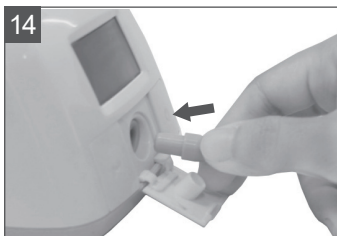

Make sure the Tubing Adapter is properly inserted into the Tubing Adapter Inlet on the Electric Pump Main Unit.

To Convert Your Double Pump Into a Single Pump:


Remove one tube and close the cap.

Disconnect the tube from the Tubing Splitter on the side with the little Cap

Cap the opening of the Tubing Splitter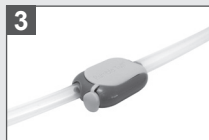

Ready to use as a single pump.

Note: Using the pump after converting from Single to Double or Double to Single may result in a temporary lack of suction or over-suction respectively. Please use caution and adjust the pump to your comfortable and effective level.

Sanitize Prior to First Use:

Before you use your Rumble Tuff® Breeze Pro Electric Breast Pump for the first time, disassemble and boil all parts of the Expression Collection Combo Kit in water for 10 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- **Do NOT boil Tubing or Tubing Adapter.**
- Remove all parts from the water immediately after sterilizing. To avoid scalds and burns, do NOT touch the product with bare hands right after sterilizing.
- Completely air-dry or towel-dry the parts before reassembling.

Wet components may affect the performance of your Electric Breast Pump. Please ensure that all parts are completely dry before use.

Before Each Use:

1. **Wash hands.** Wash your hands well with soap and water for at least 20 seconds.

2. **Assemble.** Assemble clean Express Collection Combo kits. Inspect each component for any cracks, tears, discoloration, or deterioration. Please discontinue use and replace the part immediately if damage is found. Inspect whether the kits or tubing has become moldy or soiled during storage. If your tubing is moldy, discard and replace immediately.

3. **Clean if using a share pump.** Wipe pump dials, Power Button, and pump housing with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.



Special Cleaning Instruction for the Membrane Valve:

The Membrane is specially designed with a node on the stem for your convenience to clean.

To clean the Membrane Valve, simply pull the Membrane from the Valve until it snaps slightly on the stem but does not completely detach from it.

Clean the Membrane Valve in warm, soapy water, then rinse thoroughly in clean, warm water or warm running water.

After cleaning, push the stem back into place and make sure the Membrane lays flat against the Valve opening.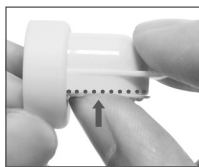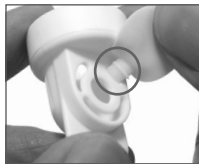

Cleaning Instructions

After every use:

1. **Store milk safely.** Cap milk collection bottle or seal milk collection bag, label with date and time, and immediately place in a refrigerator, freezer, or cooler bag with ice packs.
 - If milk collection container will be stored at a hospital or childcare facility, add name to the label.
2. **Clean pumping area.** Especially if using a shared pump, wipe the dials, power switch, and countertop with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.
3. **Take apart and inspect the Collection Kits.** Take apart breast pump tubing and leave it aside. Separate all parts that come in contact with breast/breast milk (for example, Silicone Cushions, Flanges, Membrane Valves, Diaphragm and Milk Collection Bottles).
4. **Rinse Collection Kits.** Rinse Collection Kit parts that come into contact with breast/breast milk under running water to remove remaining milk.
5. **Clean Collection Kits.** As soon as possible after pumping, clean Collection Kits that come into contact with breast/breast milk in one of the following ways.

Clean by hand

1. **Use a wash basin.** Place Collection Kit parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink because germs in sinks or drains could contaminate the pump.
2. **Add soap and water.** Fill wash basin with hot water and add soap.
3. **Scrub.** Scrub items using a clean brush that is used only to clean infant feeding items.
4. **Rinse.** Rinse by holding items under running water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. **Dry.** Allow to air-dry thoroughly. Place Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. Rumble Tuff also provides a Drying Mess Bag for your convenience to air dry all Collection Kit parts.

Clean in a dishwasher

1. **Wash.** Place disassembled Collection Kits parts in dishwasher. Be sure to place small items into a closed-top basket so they don't end up in the dishwasher filter. If possible, run the dishwasher using hot water and a heated drying cycle (or sanitizing setting); this can help kill more germs.

2. **Remove from dishwasher.** Wash your hands with soap and water before removing and storing cleaned items. If items are not completely dry, place items on a clean, unused dish towel or paper towel to air-dry thoroughly before storing or use the provided Drying Mess Bag to air dry all parts. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.

6. **Clean wash basin and bottle brush.** If you use a wash basin or bottle brush when cleaning your Collection Kits parts, rinse them well and allow them to air-dry after each use. Consider washing them every few days, either in a dishwasher with hot water and a heated drying cycle, if they are dishwasher-safe, or by hand with soap and warm water.

7. **Tubing.** Inspect the tubing. If you find condensation in the air tube, wash and dry it immediately or replace the tubing. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connect. Instead contact Rumble Tuff Customer Service.

- E-mail: customerservice@rumbletuff.com
- Toll-Free: (855) 228-8388
- Website: www.rumbletuff.com

Cleaning Instructions

For Extra Protection, Sanitize

For extra germ removal, sanitize Collection Kit parts at least once daily. Sanitizing is especially important if your baby is less than 3 months old, was born prematurely, or has a weakened immune system due to illness or medical treatment (such as chemotherapy for cancer).

Daily sanitizing of Collection Kit parts may not be necessary for older, healthy babies, if the parts are cleaned carefully after each use.

Sanitize all items (even the bottle brush and wash basin!) by using one of the following options.

Note: If you use a dishwasher with hot water and a heating drying cycle (or sanitizing setting) to clean infant feeding items, a separate sanitizing step is not necessary.

1. **Clean first.** Collection Kit parts, bottle brushes, and wash basins should be sanitized only after they have been cleaned.

2. **Sanitize.** Sanitize the Collection Kits, bottle brushes, and wash basins using one of the following options.

a. **Steam:**

Use a microwave or plug-in steam system according to the manufacturer's directions.

b. **Boil:**

Place disassembled items into a pot and cover with water.

Put the pot over heat and bring to a boil for 10 minutes.

Remove items with clean tongs.

3. **Allow to air-dry thoroughly.** Place sanitized Collection Kit parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items. You can also use the provided Drying Mesh Bag to air dry the Collection Kits.

Store Safely Until Needed

Allow the clean Collection Kits, bottle brushes, and wash basins to air-dry thoroughly before storing to help prevent germs and mold from growing. Once completely dry, the items should be stored in a clean, protected area to prevent contamination during storage.

- **Wash hands.** Wash hands well with soap and water.
- **Reassemble.** Put together the clean, dry Collection Kits.
- **Store safely.** Place reassembled Collection Kit in a clean, protected area such as inside an unused, sealable food storage bag. Store wash basins and bottle brushes in a clean area.



Tubing Cleaning Instructions:

- Rumble Tuff breast pump tubing are designed to not touch the pumped milk and does not need to be cleaned routinely.
- Keep a spare set of tubing on hand in case the set you are using gets soiled or damaged.
- If your tubing has water droplets in it at the end of a pumping session, disconnect the tubing from the Collection Kits, check to see if the Diaphragm needs to be replaced. Rinse the Tubing with clean, hot water. Hang to AIR DRY COMPLETELY BEFORE USING.
- If your tubing has mold in it, throw it away immediately because it is difficult to clean properly. Replace it with a new set of tubing, and check to see if the Diaphragm needs to be replaced.
- If the outside of your tubing is soiled, wipe it with a low-level disinfectant such as Ethyl or isopropyl alcohol (70 – 90 %) is recommended.
- Do not insert any object into the Tubing.


About Your Rumble Tuff Breeze Pro

Electric Breast Pump Main Unit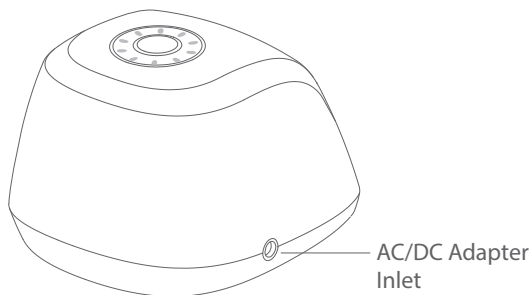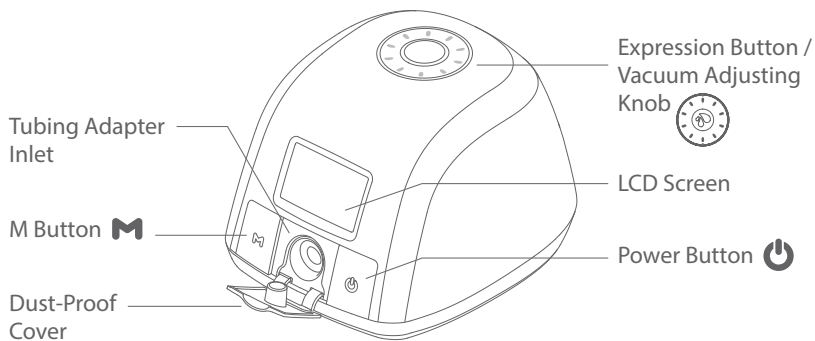


Expression Indicator Lights

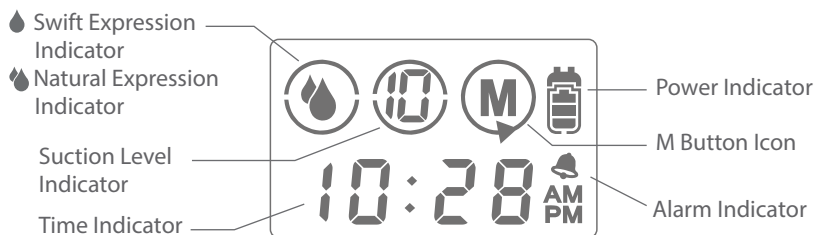About Your Rumble Tuff Breeze Pro


How to Use Your Rumble Tuff Breeze Pro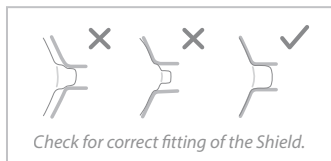

Hold the Expression Collection Combo Kit to your breast so that your nipple is centered in the Flange.

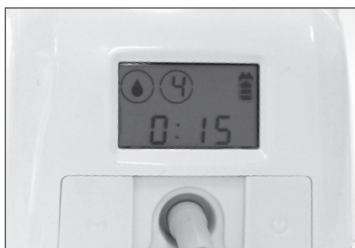
1. Open the dust-proof cover and plug in the tubing adapter.

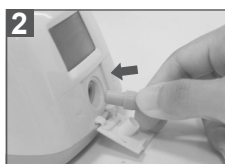2. Make sure the Tubing Adapter is properly inserted into the Tubing Adapter Inlet on the Electric Pump Main Unit.

Press the Power Button  to begin pumping.

The LCD Screen shows the time of your last pumping session for a few seconds. The pump will automatically start from the Reflex Expression for the flashing 15 seconds, then switch to the Swift Expression  automatically.

About Your Rumble Tuff Breeze Pro

How to Use Your Rumble Tuff Breeze Pro (Continued)


Adjust the suction in the Swift Expression to your comfort level during pumping by rotating the Vacuum Adjusting Knob  clockwise (up/+) or counter wise (down/-)

Depending on your preference and your baby's sucking habit, you can choose to express your milk in either the Swift Expression  or Natural Expression  settings. To switch from Swift Expression to Natural Expression, press the Expression Button  once.

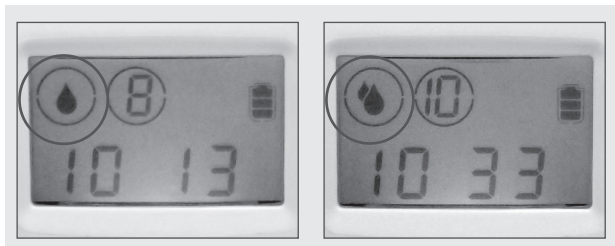To have Reflex Expression, restart the device by pressing Power Button  to turn off and turn it on again.

To check which expression setting you are on, check the LCD display for the Expression Cycle Icon. The water drop icon  indicates that you are on the Swift Expression setting. The double water drop icon  indicates that you are on the Natural Expression setting. There are 8 suction levels for Swift Expression, and 10 suction levels for Natural Expression.

When your pumping session is finished, turn the pump off by pressing the Power Button. 

About Your Rumble Tuff Breeze Pro

How to Use Your Rumble Tuff Breeze Pro (Continued)

After the pump has been turned off, the LCD Screen will remain on for a few seconds. The display will show the total amount of pumping time, then automatically switch to Standby/Clock mode, or Alarm Clock Mode if the Alarm was turned on.

After disconnecting the tubing from the Pump Unit, please make sure to cover up the Vacuum Inlets with the Dust Proof Cover.

Note:

- Make sure there is no suction on the nipples before removing the Breastshield. If necessary, break the suction by inserting a finger between the Breastshield and the breast, similar to the way you break away from your baby's suction.
- If the Rumble Tuff® Electric Breast Pump Duo is left on, but unattended for more than 30 minutes, the Pump will automatically shut-off.

Suction Level

Whenever you select a Suction Level (in any Expression setting), your Rumble Tuff Breeze Pro remembers it until you change it. This feature becomes useful after you are familiar with this pump. Your Rumble Tuff Breeze Pro retains the memory of your previous pumping level without you having to reset your preferred level each time you pump.

About Your Rumble Tuff Breeze Pro

To set your preferred suction level:

1. In Reflex / Swift Expression, adjust the suction level to the level you find most comfortable by rotating the Vacuum Adjusting Knob  clockwise (up/+) or counterclockwise (down/-)
2. In Natural Expression, adjust the suction level to the highest effective, yet still comfortable level by rotating the Vacuum Adjusting Knob  clockwise (up/+) or counterclockwise (down/-)

M Button (My Favorite Pattern)

One of the great features of your Rumble Tuff Breeze Pro is the M Button (My Favorite Pattern). You can easily set and recall your personal expression pattern by pressing the M Button. It is extremely helpful if you usually use Natural Expression setting to express your milk. If you only use Swift Expression to express your milk, you do not need to use the M Button.

To set My Favorite Pattern:

1. After a few times of pumping, you will find that your let down time is about the same in every pumping session.
2. Start pumping as you normally do. Your pump starts at the Reflex Expression for 15 seconds, then automatically switches to the Swift Expression and goes straight to the vacuum level that you last used.
3. Press the Expression Button  when your milk starts to let down to switch to the Natural Expression pattern.
Adjust the Vacuum Adjusting Knob to your desired suction level.
4. Press and hold the M Button  for three seconds, until the M icon flashes . That's it, your unique My Favorite Pattern is now set.
5. Continue to express your milk and finish your pumping session.


To recall the My Favorite Pattern:

1. Turn the Pump on. 
2. Press the M Button  (Do NOT hold more than 3 seconds).
3. Your pump automatically switches from Swift Expression to Natural Expression at the recorded let down time.

Note: You CANNOT adjust the suction level or switch between Swift Expression and Natural Expression while in My Favorite Pattern. If you have to adjust the suction level, press the M Button once to exit My Favorite Pattern first.

How to Adjust the Clock

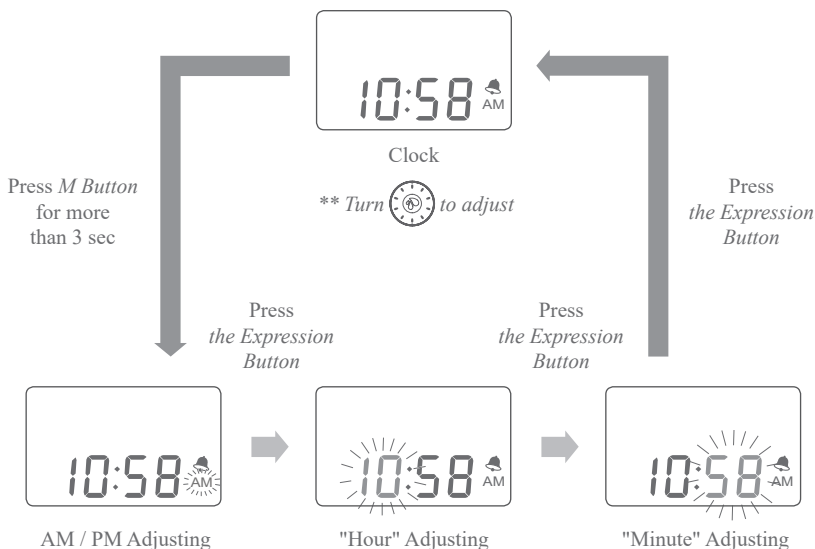1. While in the Clock/Standby Mode, press and hold the M Button **M** for 3 seconds to go into Clock Setting mode. A beep will sound and the AM/PM icon will flash.
2. To change between AM/PM, turn the Vacuum Adjustment Knob  clockwise or counterclockwise
3. Press the Expression Button  once to change from AM/PM to the hour. The hour number will flash.
4. To change the hour, turn the Vacuum Adjusting Knob  clockwise, to increase (+), or counterclockwise, to decrease(-), the hour.
5. Press the Expression Button  again to change the minutes. The minute number will flash.
6. To change the minutes, turn the Vacuum Adjusting Knob  clockwise, to increase (+), or counterclockwise, to decrease (-), the minutes.
7. Press the Expression Button  one last time to save your settings.

About Your Rumble Tuff Breeze Pro

How to Adjust the Alarm

The Alarm Clock Mode is designed to remind you when it's time for your next pumping cycle. By keeping a regular expression pattern, your body maintains a good milk supply. After you set the time interval between each pumping session, the alarm countdown will automatically start after the previous pumping cycle has ended. An alarm will sound when the pre-set time has passed.

1. In Clock/Standby Mode, press the M Button  once to go into the Alarm Adjusting Mode.
2. LCD Screen will alternately display two sets of time: Alarm Interval time (which is the time between each pumping session) and Next Alarm Time. (The time of your next pumping session). E.g. If the Power Off time of your last session is 10:58 AM and you want to do the next expression session in 2 hours and 30 minutes, then set the Alarm Interval Time as 2 hours and 30 minutes. The Next Alarm Time will be at 1: 28 PM. (As shown on P. 25)
3. Turn the Vacuum Adjustment Knob  clockwise (+) or counterclockwise (-) to adjust the Alarm Interval Time. The adjustment increases (+) or decreases (-) in 15 minute increments. The Next Alarm Time will be adjusted automatically according to the last time the pump was used.

Note: *The Alarm Interval Time is the time from the end of the last session to the next alarm set off time. It is NOT the time from the beginning of one expression session to the beginning of the next one.*


4. Press the Expression Button  to switch to the Alarm On/Off setting.
5. Turn the Vacuum Adjustment Knob  clockwise (+) or counterclockwise (-) to turn the Alarm Function On/Off.
6. Press the Expression Button  again to save your setting.

Note: *If no adjustments are made within 10 seconds in the Alarm Adjusting Mode, the setting will automatically save and the LCD Screen will return to Clock/Standby Mode.*

About Your Rumble Tuff Breeze Pro

How to Adjust the Alarm (Continued)

Alternatively shows

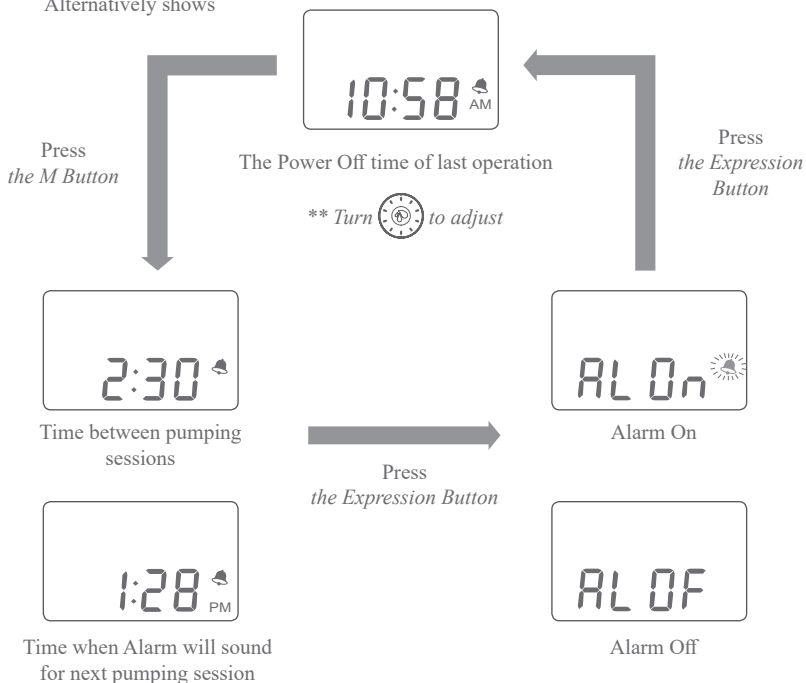How to mute chimes

Breast pump chimes can be muted in "Alarm Adjusting Mode" (see How to Adjust the Alarm P.24-25): "Alarm Off" means chimes will be muted. "Alarm On" means chimes will sound.

Note: Displayed alarm icon on display screen means pump chimes will sound.

Power Options

Your Electric Breast Pump may be operated under any of the following power supply modes:

Using your AC/DC Adapter (included)

Plug the AC/DC Adapter into an electrical outlet.

Attach the power cord to the breast pump.

Danger:

*If the power adapter outer casings become loose or are separating, contact the device manufacturer immediately. **Separation of the casing can result in electric shock.***

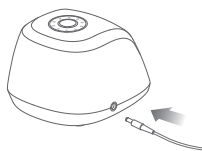
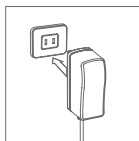
Caution:

Electrical safety can only be achieved when using the AC/DC Adapter provided with the Pump. Use of any another Adapter poses a safety risk and/or the warranty may be voided.

Built-in Li-Ion Rechargeable Battery

Your Rumble Tuff Breeze Pro contains a built-in Li-Ion rechargeable battery.

- Fully charge the battery for 12 hours before the first use on battery power.
- If needed, you may use the pump with the power cord plugged in to an external power source. The battery will charge as soon as the pump is plugged in, but not while the pump is in use.
- After the initial charge, each subsequent charge will take approximately 6 hours.
- For best results, keep your battery fully charged and recharge the battery before it completely drains.
- Keep the pump in a cool place.
- When the battery low  is light, recharge the battery as soon as possible by plug in the AC/DC Adapter to the wall outlet and other end of the power cord to the pump.

Built-in Li-Ion Rechargeable Battery (Continued)

- Your pump may not run immediately on battery power if it has been stored in a heated location. If this is the case, you can allow the battery to cool down while still using it on an external power source.

Note :

The Li-Ion Rechargeable Battery used in this device may present a risk of fire or chemical burn if mistreated.

Do NOT attempt to open the pump unit to replace the battery or for any other reason, doing so will void the warranty.

Do NOT disassemble the battery.

Do NOT place battery near a heat source or fire.

Dispose of used battery properly.

Do NOT dispose of battery in fire.

Keep away from children.

Battery Charge Status Codes

External power source connected

Battery charge status

(flashing)

Low battery, needs charging

(flashing)

Battery drained.

Fully charge the battery for 6 hours.

Operating time remaining (approx.):

Full Bar: 2hours

2 Bar: 1.5hours

1 Bar: 1hour

(flashing)


1 Bar Flash: 30minutes

- Keep the battery charged at 50% capacity and at room temperature (20-25°C/68-77°F) for long-term storage.
- We recommend batteries be charged once every 3 months to prevent over-discharge.
- Battery performance will deteriorate after being used or stored for a long period of time
- Battery life: Continuous charge and discharge for 500 cycles, the capacity will be $\geq 70\%$ of initial capacity.

Rumble Tuff Sweet Essence Assembly

How to Assemble the Manual-Pump Diaphragm Kit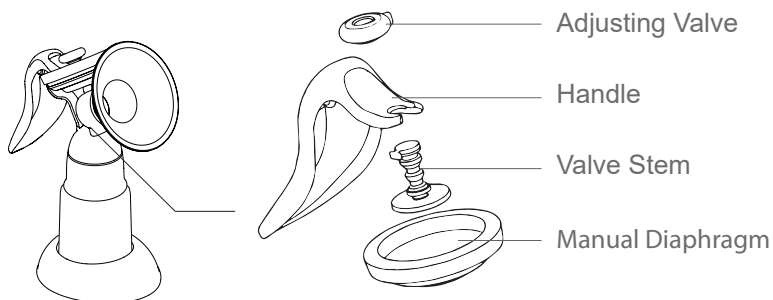

For the installation of the Membrane Valve and Silicone Cushion, please refer to Bottle Assembly on P. 8 step1 ~ P.9 step 4-3 .

With the Cup in one hand, place the Manual Silicone Diaphragm in the Cup (matching the shape of the Cup).


Press around the edges of the Diaphragm for a secure fit. Make sure the Diaphragm is evenly mounted on the Cup and there is no gap in between them.

Rumble Tuff Sweet Essence Assembly

How to Assemble the Expression Collection Combo Kit


3-1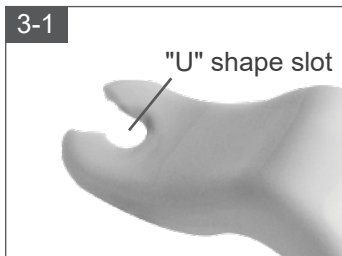

"U" shape slot

Place the "U" shape slot of the Handle onto the recess of the Adjusting Valve Stem.

3-2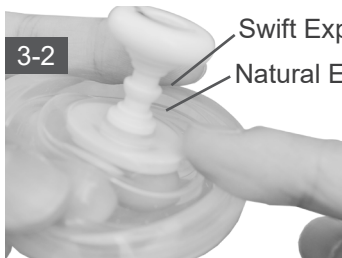

Swift Expression


Natural Expression

There are 2 recesses on the Adjusting Valve Stem. The upper recess replicates the Swift Expression setting, and the bottom recess replicates the Natural Expression setting.

3-3

4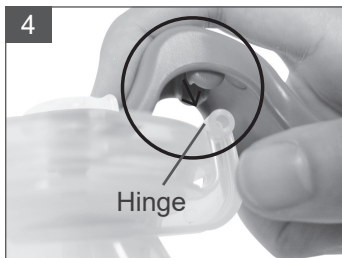

Hinge

Push the Handle down onto the hinge of the Cup until it "snaps" into place.

Using your Rumble Tuff Sweet Essence

Expression Collection Combo Kit

Note: There are two recesses for attaching the handle to the Manual Pump. You can toggle between these two settings by pushing the Adjusting Valve Stem away from the "U" shape tip of the Handle to dislocate the connection. Reattach the tip of the Handle using a different recess on the Adjusting Valve Stem.

Swift Expression Setting:

This setting is for fast, shallow suction. Often when a baby is hungry or has just "Latched-On," their nursing pattern is fast and shallow to start the mother's milk flow.

Natural Expression Setting:

This setting is for longer, deeper suction. When the mother's milk begins to flow into the baby's mouth, they often naturally change to a longer deeper suction pattern.

Note: You can use the Manual Pump in either Expression pattern. To achieve the best results, use an expression pattern or combination of expression patterns, that closely match your baby's sucking and swallowing habits.

Using your Rumble Tuff Sweet Essence

Expression Collection Combo Kit(Continued)

1

Hold the Manual Breast Pump to your breast so that your nipple is centered inside the Breast Shield, then squeeze the handle toward the bottle to test the suction.

Note: If an adjustment is necessary, turn the round knob of the Adjusting Valve to a comfortable suction level.

2

Increase

Decrease

You can adjust the suction level by turning the round knob on the top of the Adjusting Valve Stem. Turn clockwise to increase the suction and counter-clockwise to decrease it.

Note: Do not over-turn the Adjusting Valve, as it will cause the Adjusting Valve to bulge and leak air, creating a suction problem. Make sure the gap opening on top of the Adjusting Valve is lined up properly on the top of the Valve Stem so the Adjusting Valve turns smoothly.

If necessary, re-adjust the Adjusting Valve by turning it back and forth a few times.

Storage and Preparation of Breast Milk

BEFORE EXPRESSING/PUMPING MILK

Wash your hands well with soap and water.

Inspect the pump kit and tubing to make sure it is clean.
Replace moldy tubing immediately.

Clean pump dials and countertop.

STORING EXPRESSED MILK

Use breast milk storage bags or clean food-grade containers with tight fitting lids.

Avoid plastics containing bisphenol A (BPA) (recycle symbol #7).

HUMAN MILK STORAGE GUIDELINES

TYPE OF BREAST MILK	STORAGE LOCATIONS AND TEMPERATURES		
	Countertop 77°F (25°C) or colder <i>(room temperature)</i>	Refrigerator 40 °F (4°C)	Freezer 0 °F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1–2 Hours	Up to 1 Day <i>(24 hours)</i>	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding <i>(baby did not finish the bottle)</i>	Use within 2 hours after the baby is finished feeding		

Storage and Preparation of Breast Milk

STORE

Label milk with the date it was expressed and the child's name if delivering to childcare.

Store milk in the back of the freezer or refrigerator, not the door.

Freeze milk in **small amounts of 2 to 4 ounces** to avoid wasting any.

When freezing leave an inch of space at the top of the container; breast milk expands as it freezes.

Milk can be stored in an insulated cooler bag with frozen ice packs for **up to 24 hours** when you are traveling.

If you don't plan to use freshly expressed milk **within 4 days**, freeze it right away.

THAW

Always thaw the oldest milk first.

Thaw milk under lukewarm running water, in a container of lukewarm water, or overnight in the refrigerator.

Never thaw or heat milk in a microwave. Microwaving destroys nutrients and creates hot spots, which can burn a baby's mouth.

Use milk **within 24 hours** of thawing in the refrigerator (*from the time it is completely thawed, not from the time when you took it out of the freezer*).

Use thawed milk **within 2 hours** of bringing to room temperature or warming.

Never refreeze thawed milk.

FEED

Milk can be **serviced cold, room temperature, or warm.**

To heat milk, place the sealed container into a bowl of warm water or hold under warm running water.

Do not heat milk directly on the stove or in the microwave.

Test the temperature before feeding it to your baby by putting a few drops on your wrist. It should feel warm, **not hot.**

Swirl the milk to mix the fat, which may have separated.

If your baby did not finish the bottle, leftover milk should be used **within 2 hours.**

CLEAN

Wash disassembled pump and feeding parts in a clean basin with soap and water. **Do not wash directly** in the sink because the germs in the sink could contaminate items.

Rinse thoroughly under running water. Air-dry items on a clean dishtowel or paper towel.

Using clean hands, store dry items in a clean, protected area.

For extra germ removal, sanitize feeding items daily using one of these methods:

- clean in the dishwasher using hot water and heated drying cycle (*or sanitize setting*),
- boil in water for 5 minutes (*after cleaning*),
- steam in a microwave or plug-in steam system according to the manufacturer's directions (*after cleaning*).

June 2019

Centers for Disease Control and Prevention
National Center for Chronic Disease Prevention and Health Promotion

FOR MORE INFORMATION, VISIT:

<https://bit.ly/2dxVYLU>

296657-B

Interruption of Electrical Service

If you experience a power interruption due to a power surge or other factors, while operating your pump using the AD/DC Adapter, try the following:

- Unplug the AC/DC Adapter from the electrical outlet and wait at least for 5 seconds or until the power has resumed.
- Plug the AC/DC Adapter back into the electrical outlet.
- If the problem persists, try a different electrical outlet.

Radio Frequency Energy

This equipment generates, uses, and can radiate radio frequency energy. If not installed properly and used in accordance with the instructions, it may cause harmful interference with other devices in the vicinity. Try the following steps if interference occurs:

- Reorient or relocate the device receiving interference.
- Increase the distance between the equipment.
- Connect the equipment to an outlet on a circuit different from that which the other device(s) are connected.
- Consult the manufacturer or a field service technician of the device receiving interference for help.
- Please understand that there is no guarantee that interference will not occur.

Decreased Suction

- Make sure the tubing is securely attached to the pump and the Collection Kits.
- If problem happens in one side of the collection bottle, try to switch the tubing to different vacuum inlet to see if the problem is now switched to the different collection bottle.
- If yes, the problem may be at the pump unit. Contact Technical Support for further assistance.
- If not, follow the following steps to troubleshoot the problem in the Expression Collection Combo Kit.
- Make sure the Expression Collection Combo Kit is assembled correctly (see pages 8– 12).
- Center your nipple in the Flange and make sure there is a complete seal between the Flange and your skin.
- Press the Diaphragm firmly into the Cup and make sure there is no gap (see page 10 step 6-7).

- Check the Membrane Valve and make sure that it is properly sealed and securely seated (see page 8).
- Inspect the Membrane and Valve very closely for tears or warping. This component is critical to achieve proper suction; even the tiniest tear will adversely affect pump performance. If damaged, replace the Membrane Valve with one of the spare valves provided with your Rumble Tuff Breeze Pro.

No response while operating Adjusting Knob and Mode button

You CANNOT adjust the suction level or switch between Swift Expression and Natural Expression while in My Favorite Pattern. If you have to, stop My Favorite Pattern by pressing the M Button.

Pump doesn't run when power is "on"

If the Power Symbol flashes  or you see "Err1" and hear 4 Beeps (under Alarm ON), contact the local DME dealer/retailer where you purchased the unit from.

Breast Milk Buildup in the Breastshield Body

If breast milk doesn't appear to be draining fast enough into the Collection Bottle, reduce the suction setting on your Electric Breast Pump to allow the milk to drain before gradually increasing the suction to the original setting.

Where to Get Replacement Parts

If you are missing a part or if a part was damaged, please contact Rumble Tuff Customer Service.

- E-mail: customerservice@rumbletuff.com
- Toll-Free: (855) 228-8388
- Website: www.rumbletuff.com

Where to Purchase Additional Accessories

Additional accessories and replacements parts are available for purchase for use with your Rumble Tuff® Breeze Pro.

- E-mail: sales@rumbletuff.com
- Fax: 801-796-2688
- Toll-Free: 855-228-8388
- Online Store: www.rumbletuff.com/store

Condensation or Milk in Tubing:

If you find condensation in the air tube, wash and dry it immediately or replace the tubing. Rumble Tuff Breeze Pro Electric Breast Pump has anti-backflow diaphragm on top of the Collection Kit to prevent milk over flow through the tubing into main breast pump unit. If you see breast milk inside the tubing or tubing connector do not attempt to wash, clean or sterilize the tubing or tubing connector. Instead contact Rumble Tuff Customer Service.

- E-mail: customerservice@rumbletuff.com
- Fax: 801-796-2688
- Toll-Free: 855-228-8388
- Online Store: www.rumbletuff.com/store

Tips:

- *If your breasts feel uncomfortable or are in pain while using the Pump, try decreasing the suction level by adjusting the Vacuum Adjusting Knob counter wise (down/-).* 
- *You may also try pumping with/without the Silicone Cushion.*
- *If these tips do not help, turn the pump off and remove the Collection Bottles from your breasts for a few minutes. If you are still experiencing pain or discomfort, consult your healthcare professional or an international board certified lactation consultant.*
- *Your satisfaction is our highest priority. Please contact us if you experience any discomfort while using our pump.*

Position the device to ensure easy disconnection of the power cord from the supply mains.

To learn more about your pump, please visit www.rumbletuff.com or scan this QR Code with your smartphone or tablet.

The breast pump needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the instructions for use. Portable and mobile RF communications can affect the breast pump.

Note:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Guidance and manufacturer's declaration – electromagnetic emissions		
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The breast pump uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment .
RF emissions CISPR 11	Class B	The breast pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Warning :

This breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, this breast pump should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacturer's declaration – electromagnetic immunity

This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment. The Breeze Pro Breast Pump has anti-backflow as essential performance, which is not influenced by electromagnetic. Further details describe on Page 4 Warning under Safeguard. This device was tested for immunity to electromagnetic disturbances and passed using the following criteria:

1. Change of operation mode except reset into standby mode (pump is switched off) .
2. Change of pressure value more than 10%, when the device is operated as intended.
3. Change in programmable parameters, including mode and suction level.
4. Error indicated and is not restartable.
5. Permanent hardware failure is induced in the device.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	± 8 kV contact discharge ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.

Electrical fast transient/burst IEC 61000-4-4	± 2 kV, 100 kHz repetition frequency	± 2 kV, 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV Line-to-line	± 0.5 kV, ± 1 kV Line-to-line	Mains power quality should be that of a typical commercial or hospital environment .
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	0 % UT ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% UT;1 cycle At 0° 70% UT; 25 cycles At 0° 0% UT; 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the breast pump require s continued operation during power mains interruptions, it is recommended that the breast pump be powered from an uninterruptible power supply or a battery .
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50 or 60 Hz	30 A/m, 50 Hz / 60 Hz	It may be necessary to position the breast pump further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.

NOTE : UT is the a.c. mains voltage prior to application of the test level .

Guidance and manufacturer's declaration – electromagnetic immunity			
This breast pump is intended for use in the electromagnetic environment specified below. The customer or the user of the breast pump should assure that it is used in such an environment .			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	3 V 0.15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol : 
Radiated RF IEC 61000-4-3	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	10 V/m (minimum) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	

Note 1 : At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 : These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the breast pump is used exceeds the applicable RF compliance level above, the breast pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the breast pump.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the breast pump including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications device

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380 to 390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
450	430 to 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720						
1845	1700 to 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
1970						
2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n , RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

EMC tests have included the AC adapter.

Limited Consumer Product Warranty (United States)

This Rumble Tuff® Breeze Pro Electric Breast Pump includes a 3 years limited warranty (Collection Kits not included). It is warranted to the registered owner that the product to be free from defects in material and workmanship not commercially acceptable for a period of 3 years from the date of first registration of the product. This warranty and the registration are transferable. Warranty coverage terminates at the end of the third anniversary date of the product registration. This warranty gives you specific legal rights and you may also have other rights, which vary by location.

In the event that this product is found by Rumble Tuff to not meet the above limited warranty standard, as the purchaser's sole and exclusive remedy, Rumble Tuff will repair, or, at the discretion of Rumble Tuff, replace this product without charge for such replacement parts or labor. **The owner shall bear all expenses related to returning this product to Rumble Tuff.** This warranty does not apply to any part of the product that has been subject to misuse, abuse, or alteration. Only Rumble Tuff brand replacement parts are recommended for use with Rumble Tuff breastfeeding products. Improper or incorrectly performed maintenance or repair, or use of non-Rumble Tuff brand replacement parts voids this warranty.

To ensure the pump always performs at its peak performance, we suggest once per year maintenance service on the pump. Maintenance service is not part of the warranty and can be obtained from Rumble Tuff certified technician. All maintenance fee and related shipping expenses are sole responsibility of the registered owner.

TO THE EXTENT PERMITTED BY LAW, Rumble Tuff® SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES INCLUDING, BUT NOT LIMITED TO, REPLACEMENT COSTS RESULTING FROM THE BREACH OF ANY WRITTEN OR IMPLIED WARRANTY.

If you wish to make a claim under this warranty, you will need the following:

- The entire original Rumble Tuff® product
- The original receipt showing date of purchase
- Return Authorization Number (RA#) - Please contact Rumble Tuff® customer service by calling toll-free (855) 228-8388 for a RA#
- Detailed description of the problem

You, the purchaser, are responsible for all shipping charges.

Do NOT send product without a RA#.

To obtain an RA# and shipping address, please contact Rumble Tuff® Customer Service Department by:

- Toll-free: (855) 228-8388
- E-mail: customerservice@rumbletuff.com
- In writing:

Customer Service Department
Rumble Tuff
865 N 1430 W
Orem UT 84057

Items returned without a RA# will not be processed and will be returned to sender (postage due).

Power Requirements:

- AC/DC Adapter (included) North American Plug Configuration:
100V~240V AC, 50/60 Hz, 0.4A, wall
- **Plug type AC/DC Adapter with DC 9V/1.3A output.**
- Battery
Non-replaceable li-Ion battery; 7.4V 2200mAh

Technical Specifications:

- Vacuum: Swift Expression -75~-179mmHg (-10~-24kpa)
Natural Expression -75~-250mmHg (-10~-33.3kpa)
- Ambient Temperature range: +10°C to 40°C (50°F to 104°F)
- Relative Humidity range: 20% to 80%
- Transportation / Storage Environment: -10 to 50°C / 20 to 90% relative humidity
- Estimated Use-life: 1800 Hours

The meaning of the symbols found on the product and packaging are explained in below.

Symbols found on this instruction	
	Caution. Please consult the instruction for use for important cautionary information. ¹
	Non-ionizing electromagnetic radiation. Indicates that the equipment includes RF transmitters. ²
Symbols found on the main unit	
	Indicates On/Off button. ³
	Dispose the device in accordance with local regulation. ⁴
	Consult instruction for use. ⁵
	Type BF equipment (Breastshield). ⁶
	Indicates direct current. ⁷
	Polarity of power supply.

¹ ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1 : General requirements, Clause 5.4.4 Caution

² IEC 60878 Graphical symbols for electrical equipment in medical practice

³ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 29 Stand-by

⁴ EN 50419, Marking of Electrical and Electronic Equipment in accordance with Article 11(2) of Directive 2002/96/EC (WEEE)

⁵ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 11 Operating instructions

⁶ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 20 TYPE BF APPLIED PART

⁷ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 4 Direct current

⁸ Code of Federal Regulations, Title 47, Part 15b/ 15c

⁹ IEC 60417-5957, Graphical symbols use on equipment, for indoor use only

¹⁰ ISO 60601-1, Medical electrical equipment – Part 1: General requirement for basic safety and essential performance, Table D.1 Symbol 9 CLASS II equipment

Symbols Legend

Symbols found on the power adaptor	
	Indicates the product compliance with the requirements of the Federal Communications Commission. ⁸
	Indicates the device for indoor use only. ⁹
	Dispose the device in accordance with local regulation. ⁴
	The UL LISTED mark indicates that the product is manufactured in compliance with UL safety requirements for USA and Canada).
	Indicates direct current. ⁷
	Polarity of power supply.
	CLASS II Equipment. ¹⁰
Symbols found on the packaging	
	Dispose the device in accordance with local regulation. ⁴
	Type BF equipment (Breastshield). ⁶
	Consult instruction for use. ⁵
	CE mark (made in compliance with 93/42/EEC Directive on class II medical devices).*

*These symbols are not applicable for the US, Canada, Mexico market.

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Inspect the pump kit and tubing to make sure it is clean.

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HUMAN MILK STORAGE GUIDELINES

TYPE OF BREAST MILK	STORAGE LOCATIONS AND TEMPERATURES		
	Countertop 77°F (25°C) or colder <i>(room temperature)</i>	Refrigerator 40 °F (4°C)	Freezer 0 °F (-18°C) or colder
Freshly Expressed or Pumped	Up to 4 Hours	Up to 4 Days	Within 6 months is best Up to 12 months is acceptable
Thawed, Previously Frozen	1-2 Hours	Up to 1 Day <i>(24 hours)</i>	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding <i>(baby did not finish the bottle)</i>	Use within 2 hours after the baby is finished feeding		

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